## GRADIA™ gum shades

LIGHT-CURED GUM SHADE COMPOSITE SYSTEM

For use by a dental professional in the recommended indications.

**INDICATIONS FOR USE**  
 Reproduction of gum tissue for crown restorations, such as implant superstructures and removable denture frames.

**CONTRAINDICATIONS**  
 Avoid use of this product in patients with known allergies to methacrylate monomer or methylacrylate polymer.

**DIRECTIONS FOR USE**

For the following steps (No.1-5) of the procedure, refer to GRADIA Instructions for use.

1. Production of Metal Framework
2. Wax up frame in usual manner.
3. Cast in wax using master.
- 3.1 Using a carbide bur, adjust casting in usual manner.
- 3.2 Treatment of Metal Framework
- 3.3 Sandblast retention area using 50 to 100 micron aluminum oxide. Clean with dry syringe air.
- 3.4 Application of METALPRIMER II / METALPRIMER Z. Apply one or two thin coats of METALPRIMER II / METALPRIMER Z to the retention area.

Note: After application of the METALPRIMER II / METALPRIMER Z, immediately start to apply GRADIA OPAQUE to avoid contamination of the retention area.

4. Application and light curing of GRADIA FOUNDATION OPAQUE  
 Apply a layer of GRADIA FOUNDATION OPAQUE to the retention area to mask the metal. Light cure for 1 minute using the LABLIGHT LV-III / LABLIGHT DUO.

Note: The STEPLIGHT SL-I / LABLIGHT DUO cannot be used for light curing of GRADIA FOUNDATION OPAQUE.

5. Build-up of veneer crown  
 Use GRADIA composite and light cure according to GRADIA instructions for use.

6. Application and Pre-light curing of Gum Opaque (GO)  
 Apply two layers of Gum Opaque (GO) to the gum part of the metal framework. Temporarily light cure each layer for 1 minute using the LABLIGHT LV-III / LABLIGHT DUO.

Note: Light curing refer to the following chart of Irradiation time and Depth of cure.

2. For depth adjustment, apply Gum Opaque Modifier (GOM5) to the light cured Gum Opaque. Light cure for 1 minute using the LABLIGHT LV-III / LABLIGHT DUO.

3. The STEPLIGHT SL-I / LABLIGHT DUO cannot be used for light curing Gum Opaque and Gum Opaque Modifier.

4. Final light curing of Gum Opaque  
 Apply Gum (G). Temporarily light cure for 30 seconds using the STEPLIGHT SL-I / LABLIGHT DUO. Finally light cure for 10 minutes using the LABLIGHT LV-III / LABLIGHT DUO.

Note: If gum application exceeds the depth of cure mentioned in the chart, apply in layers and light cure respectively in the LABLIGHT LV-III / LABLIGHT DUO. Finally light cure for 10 minutes using the LABLIGHT LV-III / LABLIGHT DUO.

2. In case of long span and full arch, apply in sections to each side of one tooth and carry out final cure in respectively in order to minimize color shrinkage.

3. Apply Gum Modifier (GM) or Gum Translucent (GT) as necessary. If it is desired to reproduce tooth shades, incorporate Gum Fiber (GF) in translucent Gum Modifier (GM3) or material translucent Gum Translucent (GT4).

4. When using Gum Fiber, cover with Gum Translucent. Light cure and polish.

5. Remove air inhibited layer and Shape adjustment  
 Remove air inhibited layer and adjust contour using a carbide bur, diamond point or carbide bur.

6. Finish  
 Use a margin tracher than desired and progressively finish using a rubber point or silicone point.

7. Polish and buff  
 Polish and buff in usual manner to complete the gum restoration.

| Irradiation time for Gum Opaque, Gum Opaque Modifier | Pre-cure | Final cure |
|--|----------|------------|
| Light curing device                                  | 1 min.   | —          |
| Halogen / High power LED                             | 20 sec.  | 1 min.     |

Note: 1) The STEPLIGHT SL-I / LABLIGHT DUO cannot be used to light cure the above materials.

2) When using a hand held type light curing device, apply light in all directions for complete polymerization.

**Indications for Gum, Gum Modifier, Gum Translucent**

| Light curing device            | Pre-cure | Final cure |
|--------------------------------|----------|------------|
| LABLIGHT LV-III / LABLIGHT DUO | 30 sec.  | 3 min.     |
| STEPLIGHT SL-I / LABLIGHT DUO  | 10 sec.  | —          |
| Halogen / High power LED       | 20 sec.  | 1 min.     |

Note: 1) The STEPLIGHT SL-I / LABLIGHT DUO cannot be used for final curing. 2) When using a hand held type light curing device, apply light in all directions for complete polymerization.

**Depth of cure (using the LABLIGHT LV-III / LABLIGHT DUO)**

| Shades                | Irradiation time (30 sec) | Pre-cure (1 min.) | Final cure (3 min.) |
|-----------------------|---------------------------|-------------------|---------------------|
| Gum Opaque            | 0.2 mm                    | —                 | —                   |
| Gum Opaque Modifier   | —                         | 0.2 mm            | —                   |
| Gum                   | 1.1 mm                    | —                 | 2.5 mm              |
| Gum Modifier          | 0.8 mm                    | —                 | 1.5 mm              |
| Gum Translucent (GT4) | 3.0 mm                    | —                 | 5.0 mm              |

Note: 1) The STEPLIGHT SL-I / LABLIGHT DUO cannot be used for final curing. 2) When using a hand held type light curing device, apply light in all directions for complete polymerization.

**STORAGE**  
 Recommended for optimal performance, store in a cool place away from direct sunlight.

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