## GRADIA™ CORE CARTRIDGE DISPENSER

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### RECOMMENDED INDICATIONS

Dispenser gun for precise and easy dispensing of Gradia Core double barrel cartridges.

### DIRECTIONS FOR USE

- Lift the release lever (a) of the dispenser and pull the piston plunger (b) all the way back into the Dispenser.
- Load the double cased cartridge into the cartridge holder (c) of the dispenser.
- Push the piston plunger forward until it engages into the cartridge. The Dispenser is now ready for use.

### STORAGE

Store away from direct sunlight.

### PACKAGES

GRADIA CORE CARTRIDGE DISPENSER x 1 piece

### CAUTION

- GRADIA CORE Cartridge is to be used always with GRADIA CORE CARTRIDGE DISPENSER.
- GRADIA CORE CARTRIDGE DISPENSER can be autoclaved. The operating temperature should not exceed 135°C (275°F). For chemical cleaning, do not use sodium hypochlorite, sodium percarbonate, phenol, or cresol.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

### Undesired effects- Reporting:

If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:

[https://ec.europa.eu/growth/sectors/medical-devices/contacts\\_en](https://ec.europa.eu/growth/sectors/medical-devices/contacts_en)

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UK Responsible Person  
GC UNITED KINGDOM Ltd.  
Coopers Court Newport Pagnell  
Buckinghamshire  
MK16 8JS  
United Kingdom

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IFU GRADIA CORE  
CARTRIDGE DISPENSER  
WEP/EEP  
116265 - T4503

Prior to use, carefully read the instructions for use.

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</

## GRADIAT™ CORE CARTRIDGE DISPENSER

For use only by a dental professional in the recommended indications.

**RECOMMENDED INDICATIONS**  
Dispenser gun for precise and easy dispensing of Gradia Core double barrel cartridge.

**DIRECTIONS FOR USE**  
1. Lift the release lever (a) of the dispenser and pull the piston plunger (b) all the way back into the Dispenser.  
2. Load the double cased cartridge into the cartridge holder (c) of the dispenser.  
3. Push the piston plunger forward until it engages into the cartridge. The Dispenser is now ready for use.

**STORAGE**  
Store away from direct sunlight.

**PACKAGES**  
GRADIA CORE CARTRIDGE DISPENSER x 1 piece

**CAUTION**  
1. GRADIA CORE Cartridge is to be used always with GRADIA CORE CARTRIDGE DISPENSER.  
2. GRADIA CORE CARTRIDGE DISPENSER can be autoclaved. The operating temperature should not exceed 135°C (275°F). For chemical cleaning, do not use sodium hypochlorite, sodium percarbonate, phenol, or cresol.  
3. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

**Undesired effects Reporting:**  
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:  
[https://ec.europa.eu/growth/sectors/medical-devices/contacts\\_en](https://ec.europa.eu/growth/sectors/medical-devices/contacts_en)

as well as to our internal vigilance system: [vigilance@gc.dental](mailto:vigilance@gc.dental)

In this way you will contribute to improve the safety of this product.

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