For use only, carefully read
the instructions for use.
GRADIA™ DIRECT Flo

LIGHT-CURED FLOWABLE COMPOSITE

For use only by a dental professional in the
indications for use.

PRODUCT DESCRIPTION

GRADIA DIRECT Flo is a light-cure, radio-opaque, flowable restorative material to be used intra-orally and classified as a Type 1 and Class 2 (Group 1) per ISO standard 4049.

The material has a radiopacity equivalent to L5 - 2.0 mm of aluminum (density = 1 mm, enamel = 2 mm). The particle size of inorganic filler is 0.01 - 0.9 µm.

The total amount of inorganic filler is approximately 45 vol-%.

COMPOSITION

Fluoro-alumino-silicate glass, dimetacrylate, silicon dioxide, initiator, pigment, stabilizer

INDICATIONS FOR USE

1. Restoration of Class I, II, III, IV, V cavities for small Class I cavities (shallow Class V caries) and Class V cavities).

2. Restoration of root face cavities.

3. Restorations in deciduous teeth.

4. Filling tunnel shaped cavities.

5. Sealing hypersensitive areas.

6. Line/base filling in cavity undercuts.

7. Sealants.

8. Splitting mobile teeth.

9. Additions to composite restorations. (Fig. 1)

CONTRAINDICATIONS

1. Direct pulp capping.

2. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

DIRECTIONS FOR USE

1. Preparations

1) Hold the syringe upright and remove the wing cap by turning counter-clockwise. Take care not to expose the material to direct light from the dental lamp or natural light (Fig. 2).

2) Promptly and securely attach the dispensing tip (plastic or needle type) to the syringe by turning clockwise (Fig. 3).

Note:

Take care not to attach the dispensing tip too tightly. This may damage its screw.

3) After attaching the dispensing tip, protect it with the wing cap to ready for use in order to avoid exposing to light (Fig. 2).

2. Cavity Preparation

Prepare cavity using standard techniques.

Note:

For pulp capping, use calcium hydroxide.

3. Shape Selection

Select shade from 7 shades of A1, A2, A3, A5, A3, CV and BW.

A shade is based on Vita** Shade.

*A shade is a registered trademark of VITA Zahnfabrik, Bad Säckingen, Germany.

4. Use of Light-cured Bond System

For bonding to dentin, tooth structure, use light-cured bonding system (Fig. 4).

G-aenial Bond, G-Premio BOND or G-BOND which contain adhesive monomer (4-MET) for superior bondability to the structure are recommended.

Note:

When using light-cured bonding system, follow manufacturer's instructions for use.

5. Placement of GRADIA DIRECT Flo

1) Remove cover from the dispensing tip on the syringe.

2) Prior to extraction of material, gently trial push the syringe plunger outside the mouth to make sure that the dispensing tip is securely attached to the syringe. Take care not to expose the dispensing tip with the tip pointing upwards gently push forward the syringe plunger until material reaches the mouth of the tip (Fig. 6).

Note:

If there is air inside the dispensing tip, air bubbles may be formed at the time of injection.

3) Place the dispensing tip as close as possible to the tooth. Hold the dispensing tip and gently dispense the pulp until material reaches the mouth of the tip (Fig. 7). Alternatively dispense material into a mixing pad and transfer to the cavity using a suitable instrument.

Note:

When attaching the dispensing tip, make sure that no material is sticking to the joint between the tip and the syringe in order to ensure a tight seal.

b. If the dispensing tip does not extrude smoothly, remove the dispensing tip and dispense material directly from the syringe to make sure that the dispensing tip is securely attached to the syringe. Take care not to expose the dental light or ambient light. Be sure to protect it from light when working from a mixing pad.

d. After use, immediately remove the dispensing tip and tightly close the syringe with the wing cap.

e. When attaching the dispensing tip, make sure that the material is sticking to the joint between the tip and the syringe in order to ensure a tight seal.

f. If the dispensing tip does not extrude smoothly, remove the dispensing tip and dispense material directly from the syringe to make sure that the dispensing tip is securely attached to the syringe. Take care not to expose the dental light or ambient light. Be sure to protect it from light when working from a mixing pad.

g. After use, immediately remove the dispensing tip and tightly close the syringe with the wing cap.

h. When attaching the dispensing tip, make sure that the material is sticking to the joint between the tip and the syringe in order to ensure a tight seal.

i. If the dispensing tip does not extrude smoothly, remove the dispensing tip and dispense material directly from the syringe to make sure that the dispensing tip is securely attached to the syringe. Take care not to expose the dental light or ambient light. Be sure to protect it from light when working from a mixing pad.

j. After use, immediately remove the dispensing tip and tightly close the syringe with the wing cap.

k. When attaching the dispensing tip, make sure that the material is sticking to the joint between the tip and the syringe in order to ensure a tight seal.

l. If the dispensing tip does not extrude smoothly, remove the dispensing tip and dispense material directly from the syringe to make sure that the dispensing tip is securely attached to the syringe. Take care not to expose the dental light or ambient light. Be sure to protect it from light when working from a mixing pad.

m. After use, immediately remove the dispensing tip and tightly close the syringe with the wing cap.

n. When attaching the dispensing tip, make sure that the material is sticking to the joint between the tip and the syringe in order to ensure a tight seal.

o. If the dispensing tip does not extrude smoothly, remove the dispensing tip and dispense material directly from the syringe to make sure that the dispensing tip is securely attached to the syringe. Take care not to expose the dental light or ambient light. Be sure to protect it from light when working from a mixing pad.

p. After use, immediately remove the dispensing tip and tightly close the syringe with the wing cap.

q. When attaching the dispensing tip, make sure that the material is sticking to the joint between the tip and the syringe in order to ensure a tight seal.

r. If the dispensing tip does not extrude smoothly, remove the dispensing tip and dispense material directly from the syringe to make sure that the dispensing tip is securely attached to the syringe. Take care not to expose the dental light or ambient light. Be sure to protect it from light when working from a mixing pad.

s. After use, immediately remove the dispensing tip and tightly close the syringe with the wing cap.

t. When attaching the dispensing tip, make sure that the material is sticking to the joint between the tip and the syringe in order to ensure a tight seal.

u. If the dispensing tip does not extrude smoothly, remove the dispensing tip and dispense material directly from the syringe to make sure that the dispensing tip is securely attached to the syringe. Take care not to expose the dental light or ambient light. Be sure to protect it from light when working from a mixing pad.

v. After use, immediately remove the dispensing tip and tightly close the syringe with the wing cap.

w. When attaching the dispensing tip, make sure that the material is sticking to the joint between the tip and the syringe in order to ensure a tight seal.

x. If the dispensing tip does not extrude smoothly, remove the dispensing tip and dispense material directly from the syringe to make sure that the dispensing tip is securely attached to the syringe. Take care not to expose the dental light or ambient light. Be sure to protect it from light when working from a mixing pad.

y. After use, immediately remove the dispensing tip and tightly close the syringe with the wing cap.

z. When attaching the dispensing tip, make sure that the material is sticking to the joint between the tip and the syringe in order to ensure a tight seal.

aa. If the dispensing tip does not extrude smoothly, remove the dispensing tip and dispense material directly from the syringe to make sure that the dispensing tip is securely attached to the syringe. Take care not to expose the dental light or ambient light. Be sure to protect it from light when working from a mixing pad.

bb. After use, immediately remove the dispensing tip and tightly close the syringe with the wing cap.

cc. When attaching the dispensing tip, make sure that the material is sticking to the joint between the tip and the syringe in order to ensure a tight seal.

dd. If the dispensing tip does not extrude smoothly, remove the dispensing tip and dispense material directly from the syringe to make sure that the dispensing tip is securely attached to the syringe. Take care not to expose the dental light or ambient light. Be sure to protect it from light when working from a mixing pad.

ee. After use, immediately remove the dispensing tip and tightly close the syringe with the wing cap.

ff. When attaching the dispensing tip, make sure that the material is sticking to the joint between the tip and the syringe in order to ensure a tight seal.

gg. If the dispensing tip does not extrude smoothly, remove the dispensing tip and dispense material directly from the syringe to make sure that the dispensing tip is securely attached to the syringe. Take care not to expose the dental light or ambient light. Be sure to protect it from light when working from a mixing pad.

hh. After use, immediately remove the dispensing tip and tightly close the syringe with the wing cap.

ii. When attaching the dispensing tip, make sure that the material is sticking to the joint between the tip and the syringe in order to ensure a tight seal.

jj. If the dispensing tip does not extrude smoothly, remove the dispensing tip and dispense material directly from the syringe to make sure that the dispensing tip is securely attached to the syringe. Take care not to expose the dental light or ambient light. Be sure to protect it from light when working from a mixing pad.

kk. After use, immediately remove the dispensing tip and tightly close the syringe with the wing cap.

ll. When attaching the dispensing tip, make sure that the material is sticking to the joint between the tip and the syringe in order to ensure a tight seal.

mm. If the dispensing tip does not extrude smoothly, remove the dispensing tip and dispense material directly from the syringe to make sure that the dispensing tip is securely attached to the syringe. Take care not to expose the dental light or ambient light. Be sure to protect it from light when working from a mixing pad.

nn. After use, immediately remove the dispensing tip and tightly close the syringe with the wing cap.

oo. When attaching the dispensing tip, make sure that the material is sticking to the joint between the tip and the syringe in order to ensure a tight seal.

pp. If the dispensing tip does not extrude smoothly, remove the dispensing tip and dispense material directly from the syringe to make sure that the dispensing tip is securely attached to the syringe. Take care not to expose the dental light or ambient light. Be sure to protect it from light when working from a mixing pad.

qq. After use, immediately remove the dispensing tip and tightly close the syringe with the wing cap.

rr. When attaching the dispensing tip, make sure that the material is sticking to the joint between the tip and the syringe in order to ensure a tight seal.

ss. If the dispensing tip does not extrude smoothly, remove the dispensing tip and dispense material directly from the syringe to make sure that the dispensing tip is securely attached to the syringe. Take care not to expose the dental light or ambient light. Be sure to protect it from light when working from a mixing pad.

tt. After use, immediately remove the dispensing tip and tightly close the syringe with the wing cap.

uu. When attaching the dispensing tip, make sure that the material is sticking to the joint between the tip and the syringe in order to ensure a tight seal.

vv. If the dispensing tip does not extrude smoothly, remove the dispensing tip and dispense material directly from the syringe to make sure that the dispensing tip is securely attached to the syringe. Take care not to expose the dental light or ambient light. Be sure to protect it from light when working from a mixing pad.

ww. After use, immediately remove the dispensing tip and tightly close the syringe with the wing cap.

xx. When attaching the dispensing tip, make sure that the material is sticking to the joint between the tip and the syringe in order to ensure a tight seal.

yy. If the dispensing tip does not extrude smoothly, remove the dispensing tip and dispense material directly from the syringe to make sure that the dispensing tip is securely attached to the syringe. Take care not to expose the dental light or ambient light. Be sure to protect it from light when working from a mixing pad.

zz. After use, immediately remove the dispensing tip and tightly close the syringe with the wing cap.

aa. When attaching the dispensing tip, make sure that the material is sticking to the joint between the tip and the syringe in order to ensure a tight seal.

bb. If the dispensing tip does not extrude smoothly, remove the dispensing tip and dispense material directly from the syringe to make sure that the dispensing tip is securely attached to the syringe. Take care not to expose the dental light or ambient light. Be sure to protect it from light when working from a mixing pad.

cc. After use, immediately remove the dispensing tip and tightly close the syringe with the wing cap.

dd. When attaching the dispensing tip, make sure that the material is sticking to the joint between the tip and the syringe in order to ensure a tight seal.

ee. If the dispensing tip does not extrude smoothly, remove the dispensing tip and dispense material directly from the syringe to make sure that the dispensing tip is securely attached to the syringe. Take care not to expose the dental light or ambient light. Be sure to protect it from light when working from a mixing pad.

ff. After use, immediately remove the dispensing tip and tightly close the syringe with the wing cap.

gg. When attaching the dispensing tip, make sure that the material is sticking to the joint between the tip and the syringe in order to ensure a tight seal.

hh. If the dispensing tip does not extrude smoothly, remove the dispensing tip and dispense material directly from the syringe to make sure that the dispensing tip is securely attached to the syringe. Take care not to expose the dental light or ambient light. Be sure to protect it from light when working from a mixing pad.

ii. After use, immediately remove the dispensing tip and tightly close the syringe with the wing cap.

jj. When attaching the dispensing tip, make sure that the material is sticking to the joint between the tip and the syringe in order to ensure a tight seal.

kk. If the dispensing tip does not extrude smoothly, remove the dispensing tip and dispense material directly from the syringe to make sure that the dispensing tip is securely attached to the syringe. Take care not to expose the dental light or ambient light. Be sure to protect it from light when working from a mixing pad.

ll. After use, immediately remove the dispensing tip and tightly close the syringe with the wing cap.

mm. When attaching the dispensing tip, make sure that the material is sticking to the joint between the tip and the syringe in order to ensure a tight seal.

nn. If the dispensing tip does not extrude smoothly, remove the dispensing tip and dispense material directly from the syringe to make sure that the dispensing tip is securely attached to the syringe. Take care not to expose the dental light or ambient light. Be sure to protect it from light when working from a mixing pad.

oo. After use, immediately remove the dispensing tip and tightly close the syringe with the wing cap.

pp. When attaching the dispensing tip, make sure that the material is sticking to the joint between the tip and the syringe in order to ensure a tight seal.

qq. If the dispensing tip does not extrude smoothly, remove the dispensing tip and dispense material directly from the syringe to make sure that the dispensing tip is securely attached to the syringe. Take care not to expose the dental light or ambient light. Be sure to protect it from light when working from a mixing pad.

rr. After use, immediately remove the dispensing tip and tightly close the syringe with the wing cap.

ss. When attaching the dispensing tip, make sure that the material is sticking to the joint between the tip and the syringe in order to ensure a tight seal.

tt. If the dispensing tip does not extrude smoothly, remove the dispensing tip and dispense material directly from the syringe to make sure that the dispensing tip is securely attached to the syringe. Take care not to expose the dental light or ambient light. Be sure to protect it from light when working from a mixing pad.

uu. After use, immediately remove the dispensing tip and tightly close the syringe with the wing cap.

vv. When attaching the dispensing tip, make sure that the material is sticking to the joint between the tip and the syringe in order to ensure a tight seal.

ww. If the dispensing tip does not extrude smoothly, remove the dispensing tip and dispense material directly from the syringe to make sure that the dispensing tip is securely attached to the syringe. Take care not to expose the dental light or ambient light.

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