IMPRESSION SEPARATION WAFERS

- EN** Polyethylene spacer wafer for double impression techniques
- DE** Polyethylen-Separierfolie für die Zweiphasentechnik
- FR** Feuillets espaceurs en polyéthylène pour techniques double empreinte
- IT** Cialda spaziatrice in polietilene per tecnica della doppia impronta
- ES** Oblea de separación de polietileno para técnicas de doble impresión
- NL** Polyethyleen wafels voor dubbele afdruktechnieken
- DA** Polyætylen warfer til dobbel aftryksteknik
- SV** Polyetylenfolie för separering vid dubbelavtrycksteknik
- PT** Folha separadora de polietileno para técnicas de dupla impressão
- EL** Λιαχωριστικές μεμβράνες πολυαιθυλενίου λήψης αποτυπωμάτων με την τεχνική της διπλής αποτύπωσης
- NO** Polyetylen separasjons skive for doble avtrykksteknikker
- FI** Polyeteeni erotuslevy kaksoisjäljennöstechniikoihin

, 'GC', '

EN

Prior to use, carefully read
the instructions for use.

IMPRESSION SEPARATION WAFERS

For use only by a dental professional in the recommended indications.

INDICATIONS FOR USE

Separation wafers, consisting of 1 mm thick polyethylene foam, are intended to create space in a first GC EXA'lence or GC Exaflex Putty impression to be used later as a customtray. The even space of 1 mm throughout guarantees an equal distribution of the wash impression material. This results in full advantage being taken of the elasticity of the impression material, allowing easy removal of the final impression with no permanent deformation. The separation wafers are suitable for both upper and lower arch impressions.

CONTRAINDICATIONS

Patients with known allergy for polyethylene. In case of allergy, seek medical advice.

DIRECTIONS FOR USE

Prior to preparation of the tooth:

- Select a suitably sized rigid metal GC/COE impression tray(fig.1).
- Take a separation wafer and check the size against the tray. Where necessary parts of the dorsal ends of the wafer are removed with scissors (fig.2), leaving about 2 mm of the dorsal side of the tray uncovered. This will guarantee a good fit at the end of the tooth arch, thereby limiting the chance of surplus wash material escaping dorsally when taking the final impression.

- c. GC EXA'lence or GC Exaflex Putty is mixed and distributed uniformly in the tray (fig.3). A separation wafer is placed onto the impression material and lightly pressed down so that it sticks to the surface (fig.4). The impression tray assembly is brought into the mouth to obtain the first impression. As soon as the first hardening of the material occurs, the impression is removed (fig.5).

After preparation of the tooth:

- d. The separation wafer is removed from the first impression. It can be seen that the wafer has imparted a certain texture onto the surface of the impression material, thereby offering extra retention (fig.6). Check the fit of the 'custom tray' in the mouth. Excess material may be removed with a scalpel.
- e. After cleaning and drying the first impression, the final impression can be taken with one of the following syringable impression materials: GC EXA'lence Light or Extra Light Body or GC Exaflex, GC Examix or GC Exafast NDS in either Injection or Regular grades.
- f. Apply the wash material to the preparation(s), the occlusal surfaces and the first impression, and seat the 'custom tray' slowly over the tooth arch in such a way that air and excess material can escape without the introduction of any high stresses. When seated, the tray must be held in place passively without compression.
- g. After the material has set remove the tray from the mouth and continue in the usual way.

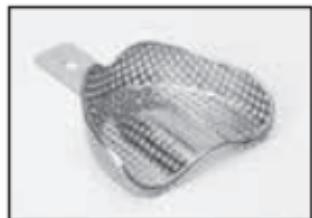
Fig.1

Fig.2

Fig.3

Fig.4

Fig.5


Fig.6

STORAGE

Recommended for optimal performance, store at room temperature (15-25°C/59-77°F), 50±10% relative humidity

CAUTION

Before use, please remove all excess material from the separation wafer with scissors.

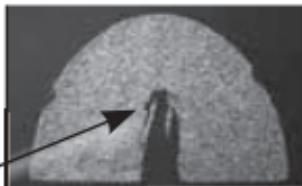PACKAGES

30 separation wafers

For infection control reasons, separation wafers are for single use only.
After use, discard the wafer in order to prevent cross-contamination.

Some products referenced in the present IFU may be classified as hazardous according to GHS.
Always familiarize yourself with the safety data sheets available at: <http://www.gceurope.com>
or for The Americas <http://www.gcamerica.com>
They can also be obtained from your supplier.

Undesired effects- Reporting:

If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilancesystem, by selecting the proper authority of your country accessible through the following link:

https://ec.europa.eu/growth/sectors/medical-devices/contacts_en
as well as to our internal vigilance system:

vigilance@gc.dental

In this way you will contribute to improve the safety of this product.

For the Summary of Safety and Clinical Performance (SSCP) please see EUDAMED database
(<https://ec.europa.eu/tools/eudamed>) or contact us at Regulatory.gce@gc.dental

UK Responsible Person
GC UNITED KINGDOM Ltd. Coopers Court Newport Pagnell, Buckinghamshire, MK16 8JS
United Kingdom

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3737 West 127th Street, Alsip, IL 60803 U.S.A.
GC ASIA DENTAL PTE. LTD.
3737 West 127th Street, Alsip, IL 60803, U.S.A.