Prior to use, carefully read the instructions **EN**

GC Initial™ INmetalbond

Buffer between GC Initial metal ceramics and dental alloys.

INTENDED USE
INmetalbond is used as a thin layer between the alloy and the first Opaque layer. The INmetalbond blocks the escaping metal oxides and neutralizes differences in the expansion coefficient.

It can be used on all precious and non-precious PFM alloys improving and securing the chemical bond on various types of alloys and allowing a wider span of CTE compatibility. Compatible with Initial MC and Initial LF ceramic systems.

INDICATIONS FOR USE
CAN be used on all PFM alloys, precious as well as non-precious.

CONTRAINDICATIONS
Not to be used on titanium frameworks

COMPOSITION
SiO₂ 22-26%, Al₂O₃ 3-5%, K₂O 5-8%, Na₂O 2-4%, BaO 3-5%, B₂O₃ 4-6%, TiO₂ 27-32%

INSTRUCTION FOR USE

- Metal framework needs to be prepared according to manufacturer’s instructions.
- Oxidation firing again according to manufacturer’s instructions.
- Stir paste before using.
- Apply the Bonder in a thin layer but mask the metal completely.

- IMPORTANT:**
 - To change the consistency of the INmetalbond paste, use the “GC Initial Paste Opaque Thinner”.
 - Only use it in very small quantities.
 - To avoid drying out, close the cap after using.
 - The fired Bonder should have a yellowish, slightly shiny surface.

- NOTE:**
 - The colour of the fired bonder can vary depending on the composition of the alloys.

- When using NPA (non-precious alloys) with the INmetalbond, the final temperature of the Opaque Washbake does not need to be increased by 20°C anymore. e.g.: 1st Opaque Firing on NPA: GC Initial MC = 940°C GC Initial LF = 830°C

FIRING INSTRUCTIONS

Preheating Temp.	Drying Time	T° increase	Vacuum	Final Temp.	Holding Time
550°C	6min	80°C/min	Yes	980°C	1 min no vacuum

STORAGE
Recommended for optimal performance, store in a cool, dry place.

PACKAGES

GC Initial INmetalbond, 2x4gr.

CAUTION

Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IUF may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: http://www.gc.dental/europe. They can also be obtained from your supplier.

For the Summary of Safety and Clinical Performance (SSCP) please see EUDAMED database (https://ec.europa.eu/tools/eudamed) or contact us at Regulatory.gce@gc.dental
Undesired effects - Reporting:
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gc.dental
In this way you will contribute to improve the safety of this product.

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