Master all your challenges

initial™ Ti

- EN Technical Manual
- DE Technisches Handbuch
- FR Manuel Technique
- IT Manuale tecnico

APRIL 2024

EN	DE	FR	IT	
Table of Contents	Inhaltsverzeichnis	Table des matières	Indice	Page
Intended Use	Bestimmungsgebrauch	Utilisation	Campo di impiego	4-5
True-to-nature ...	Naturidentisch ...	Proche du naturel ...	Estetica naturale ...	6
Colour Combination Chart	Farbtabelle	Tableau des teintes	Tabella di combinazione delle masse	6
GC Initial Ti System Colour Chart	GC Initial Ti System Farbkarte	Tableau des teintes du système GC Initial Ti	GC Initial Ti Tabella Colori	7
GC Initial Ti Bonder	GC Initial Ti Bonder	GC Initial Ti Bonder	GC Initial Ti Bonder	8-9
Standardized build up procedure	Standardisierte Schichttechnik	Procédure classique de montage	Procedura di stratificazione standardizzata	
Anterior tooth	Frontzahn	Dent antérieure	Dente anteriore	10-14
Molar	Molar	Molaire	Molare	15-19
GC Initial – Colour combination chart	GC Initial - Farbuordnungstabelle	GC Initial – Tableau de combinaison des teintes	GC Initial – Schemi delle combinazioni cromatiche	20-22
Multi-chromatic build up procedure	Polychrome Schichttechnik	Procédure de montage multi chromatique	Procedura di stratificazione policromatica	23-31
Firing instructions	Brennanleitung	Tableau de cuisson	Istruzioni per la cottura	32
Physical properties & Shelf Life	Physikalische Eigenschaften & Haltbarkeitsdauer	Propriétés physiques & Péréemption	Proprietà fisiche & Scadenza	33-34
Related products	Verwandte Produkte	Produits associés	Prodotti collegati	35
Medical Device Regulation Information	Informationen zur Medizinprodukteverordnung	Informations sur la réglementation des dispositifs médicaux	Informazioni sul regolamento dei dispositivi medici	36-37

EN Intended Use

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EN Thank you for choosing our GC Initial Ti veneering ceramic.

GC Initial Ti is a veneering ceramic powder, with low fusing temperature, especially adapted for titanium frameworks.

To bond the ceramic securely to the framework use GC Initial Titanium Bonder.

This instruction for use will give you an impression of how easy it is to get a convincing aesthetic result with a minimum workload that highlights the excellent features of this veneering ceramic.

Indications

For characterisation and veneering of dental alloy frameworks:

- Crown and Bridge work (incl. reproduction of gum tissue)
- Implant superstructures

Contraindications

- For patients with parafunctions (e.g. bruxism)
- Multi-unit framework free restorations
- Veneering of alloys in the non-approved CTE range (CTE= coefficient of thermal expansion of the ceramic veneering material or the used alloys. The respective CTE must be coordinated with one another)
- It may be contra-indicated in patients with known allergies to specific metal alloys.

Warnings

Inhalation of ceramic dust released during processing must be avoided by using suitable suction devices and, if necessary, protective masks. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

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Description	No. of Bottles	Colour Code
Cervical Translucent	5	CT-21: Light
		CT-22: Yellow Soft
		CT-23: Pale Orange
		CT-24: Yellow
		CT-25: Brown
Shoulder Transpa	7	ST-30: Neutral White
		ST-31: Cream
		ST-32: Light Yellow
		ST-33: Yellow
		ST-34: Pink/Beige
		ST-35: Olive Brown
		ST-36: Light Brown
Shoulder Opaque	3	SO-37: Ivory
		SO-38: Orange
		SO-39: Brown
INside	11	IN-41: Flamingo
		IN-42: Terracota
		IN-43: Sun
		IN-44: Sand
		IN-45: Havanna
		IN-46: Brasil
		IN-47: Sienna
		IN-48: Kurkuma
		IN-49: Maracuja
		IN-50: Curry
		IN-51: Olive
Glaze	1	GL
Correction Powder	1	COR
Bleach	4	BLD-1: Light
		BLD-2: White
		BLD-3: Xwhite
		BL-E: Bleach enamel
Gum	2	GU: Gum Universal
		GM-23: Base light

EN GC Initial Ti Bonder
 DE GC Initial Ti Bonder
 FR GC Initial Ti Bonder
 IT GC Initial Ti Bonder

EN Finishing is carried out with special, cross cur burrs. These burrs should only be used for grinding titanium.

- grind with low pressure
- the framework should be grinded in one direction only
- clean the burrs using the steam cleaner
- sharp edges and overlapping should be avoided
- slow speed grinding (approx. 15.000 rpm)

Sandblast with noble corundum only.

- grain size 120 - 150 μm /2bar pressure
- blast in angle of 45° to object
- steam clean the surface
- the surface needs a passivating time of 5 minutes

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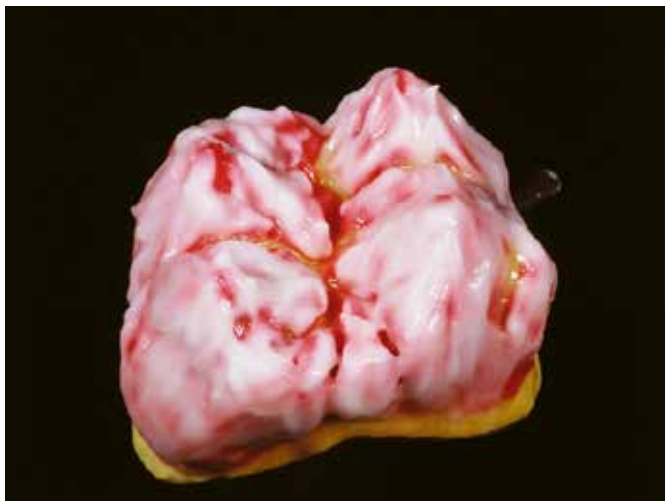
EN This Dentin base is now covered with Clear Fluorescence material, CL-F. This layer is 0,2 mm in depth and is a copy of human dentition.

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
EN Enamel and / or Translucent are applied on top of the Clear Fluorescence layer up to the occlusal contacts of the antagonists.

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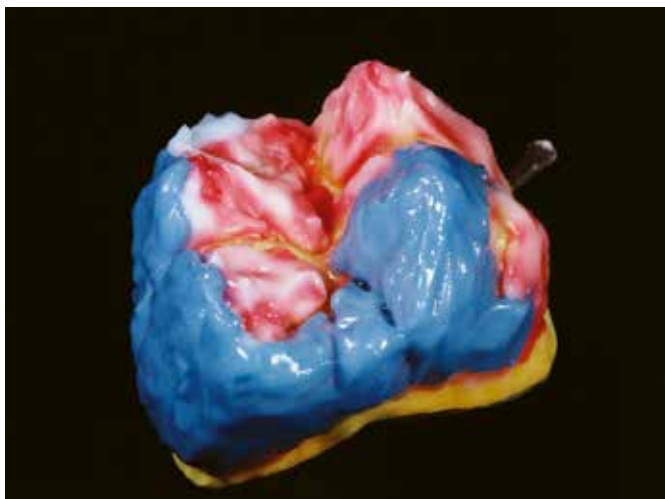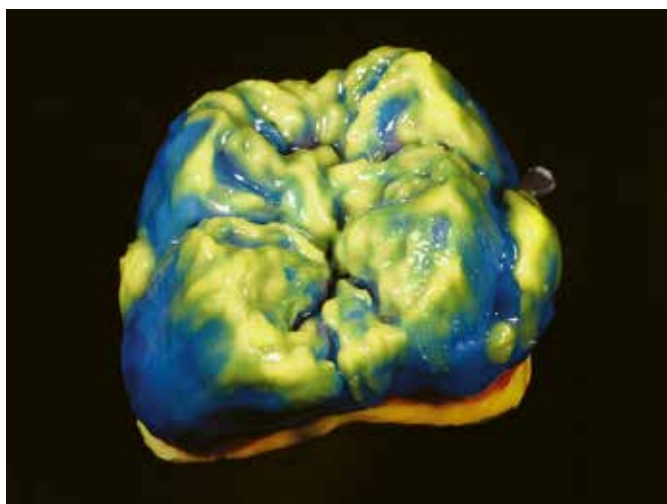
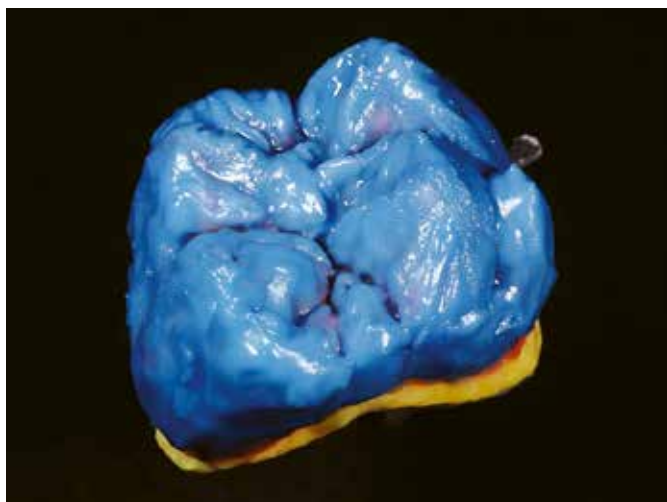
EN The Shoulder Opaque SO-37 should be used as a first basic layering at the border frame / tooth shoulder. The percentages used in this list can vary depending on the thickness of the total ceramic shoulder. **BASIC RULE:** The thinner the total ceramic shoulder, the more high chroma shoulder powders should be used. Furthermore, all shoulder powders can be mixed with the INside powders to intensify or to individualize the shoulder shades until a maximum of 20% INside powders of this total mixture. In this case the final temperature of shoulder firing should be reduced by 10°C to 20°C, depending on the amount of INside powders mixed with the shoulder porcelain.

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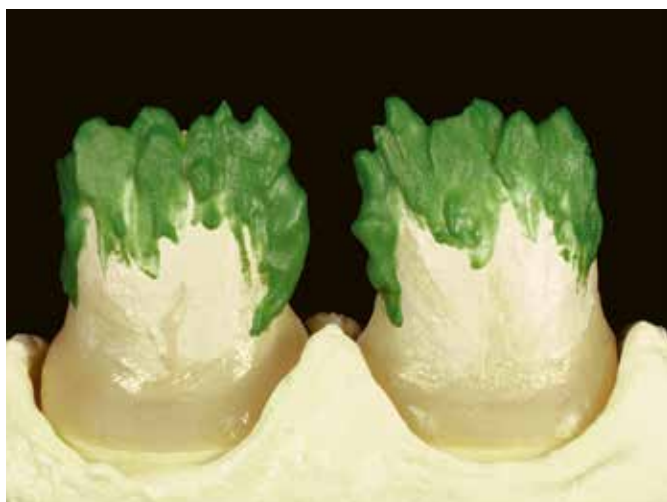
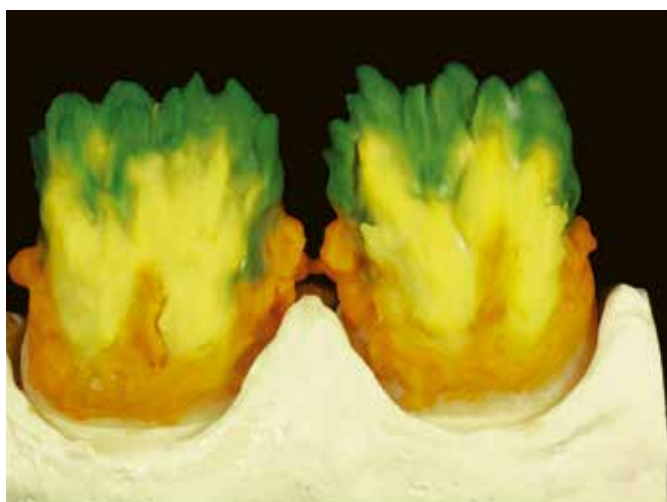
GC Initial Spectrum Stains

GC Initial Ti

EN Medical Device Regulation Information

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EN COMPOSITION

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GC Initial Ti

Powders: SiO₂ 65-70%, Al₂O₃ 6-9%, K₂O 5-7%, Na₂O 7-10%, CaO 5-7%, B₂O₃ 1-2%, ZrO₂ < 1%, Pigments 0.1-3.0%

Liquids: H₂O + Propylenglykol

GC Initial Ti Bonder

Powder: B₂O₃ 40-48%, SrO 13-18%, La₂O₃ 36-43%

Liquid: H₂O + Propylenglykol

EN Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <http://www.gceurope.com>

or for The Americas: <http://www.gcamerica.com>

They can also be obtained from your supplier.

For the Summary of Safety and Clinical Performance (SSCP) please see EUDAMED database (<https://ec.europa.eu/tools/eudamed>) or contact us at Regulatory.gce@gc.dental.

Undesired effects - Reporting: If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:

https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gc.dental.

In this way you will contribute to improve the safety of this product.

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Notes

Notes

Notes section with horizontal dotted lines for writing.

 EU: GC EUROPE N.V.
Researchpark Haasrode-Leuven 1240
Interleuvenlaan 33, B-3001 Leuven
Tel. +32.16.74.10.00, Fax. +32.16.40.02.14
[info.gce@gc.dental](https://www.gc.dental/europe), <https://www.gc.dental/europe>

RESPONSIBLE MANUFACTURER IN CANADA
GC AMERICA INC.
3737 W. 127th Street, Alsip, IL 60803 U.S.A.

GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
Tel: +1-708-597-0900
www.gc.dental/america

GC Germany GmbH
Seifgrundstraße 2, D-61348 Bad Homburg
Tel. +49.61.72.99.59.60, Fax. +49.61.72.99.59.66.6
[info.germany@gc.dental](https://www.gc.dental/europe/de-DE), <https://www.gc.dental/europe/de-DE>

GC ITALIA S.r.l.
Via Luigi Cadorna 6, I-20090 Vimodrone (MI)
Tel.: +39 02 98282068
[info.italy@gc.dental](https://www.gc.dental/europe/it-IT), <https://www.gc.dental/europe/it-IT>

GC UNITED KINGDOM Ltd.
Coopers Court, Coopers Court, Newport Pagnell,
UK-Bucks. MK16 8JS
Tel. +44.1908.218.999, Fax. +44.1908.218.900
[info.uk@gc.dental](https://www.gc.dental/europe/en-GB), <https://www.gc.dental/europe/en-GB>

GC FRANCE s.a.s.
8 rue Benjamin Franklin, F-94370 Sucy en Brie Cedex
Tel. +33.1.49.80.37.91, Fax. +33.1.45.76.32.68
[info.france@gc.dental](https://www.gc.dental/europe/fr-FR), <https://www.gc.dental/europe/fr-FR>

GC IBÉRICA - Dental Products, S.L.
Edificio Codesa 2
Playa de las Américas, 2, 1º, Of. 4, ES-28290 Las Rozas, Madrid
Tel. +34.916.364.340, Fax. +34.916.364.341
[comercial.spain@gc.dental](https://www.gc.dental/europe/es-ES), <https://www.gc.dental/europe/es-ES>

GC AUSTRIA GmbH
Tallak 124, A-8103 Gratwein-Strassengel
Tel. +43.3124.54020, Fax. +43.3124.54020.40
[info.austria@gc.dental](https://www.gc.dental/europe/de-AT), <https://www.gc.dental/europe/de-AT>

UK RESPONSIBLE PERSON
GC UNITED KINGDOM LTD.
Coopers Court Newport Pagnell
Buckinghamshire, mk16 8js, United Kingdom

GC EUROPE NV - Benelux Sales Department
Researchpark Haasrode-Leuven 1240
Interleuvenlaan 33, B-3001 Leuven
Tel. +32.16 74.18.60
[info.benelux@gc.dental](https://www.gc.dental/europe/fr-BE), <https://www.gc.dental/europe/fr-BE>

GC EUROPE N.V. - East European Office
Siget 19b, HR-10020 Zagreb
Tel. +385.1.46.78.474, Fax. +385.1.46.78.473
[info.eeo@gc.dental](https://www.gc.dental/europe/hr-HR), <https://www.gc.dental/europe/hr-HR>

GC NORDIC AB - Finnish Branch
Lemminkäisenkatu 46, FIN-20520 Turku
Tel. +358.40.900.07.57
[info.finland@gc.dental](https://www.gc.dental/europe/fi-FI), <https://www.gc.dental/europe/fi-FI>

GC NORDIC AB - GC Nordic Danish Branch
c/o Andersen Partners - Advokatpartnerselskab
Buen 11 6, DK-6000 Kolding
Tel. +45 51 15 03 82
[info.denmark@gc.dental](https://www.gc.dental/europe/da-DK), <https://www.gc.dental/europe/da-DK>

GC NORDIC AB
c/o Lundin Revisionbyrå
Erik Dahlbergsgatan 11B, SE-411 26 Göteborg
Tel. +46.768.54.43.50
[info.nordic@gc.dental](https://www.gc.dental/europe/sv-SE), <https://www.gc.dental/europe/sv-SE>

GC AUSTRIA GmbH - Swiss Office
Zürichstrasse 31, CH-6004 Luzern
Tel. +41.41.520.01.78, Fax. +41.41.520.01.77
[info.switzerland@gc.dental](https://www.gc.dental/europe/de-CH), <https://www.gc.dental/europe/de-CH>

GC AUSTRALASIA DENTAL PTY LTD
1753 Botany Rd, Banksmeadow NSW 2019, Australia
Tel: +61-2-9301 8200, Fax: +61-2-9316 4196

GC SOUTH AMERICA
Rua Heliadora, 399, Santana - São Paulo, SP, BRASIL
CEP: 02022-051 - TEL: +55-11-2925-0965
CNPJ: 08Initial.279.999/0001-61
RESP. TÊC: Erick de Lima - CRO/SP 100.866

GC ASIA DENTAL PTE. LTD.
5 Tampines Central 1, #06-01 Tampines Plaza 2, Singapore 529541
Tel: +65 6546 7588

z LMA TI EN/DE & FR/IT 7 29 04/2024
NT6305 - 890080 - 10003201

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- EN V-Shade is referring to the Vita® Shade Guide. Vita® is a registered trademark of VITA Zahnfabrik H. Rauter GmbH & Co. KG, Bad Säckingen, Germany.