Master all your challenges

initial[™] Zirconia Coloring Liquid

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3

One Liquid – two techniques

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Painting technique – individual & detailed procedure
Immersion technique – fast & easy procedure

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4 - 5

Painting technique
individual & detailed procedure

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6 - 12

Individual coloring procedure
& chart using the painting technique

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13

Immersion technique
fast & easy procedure

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14 - 19

Dedicated dental instruments for processing
zirconium oxide frameworks

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Verwandte Produkte

24

Medical Device
Regulation Information

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GC Initial Zirconia Coloring Liquid

EN Intended Use

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EN Water-based coloring liquids for the complete or partial coloring of pre-sintered zirconia restorations

„GC Initial Zirconia Coloring Liquid“ are intended for use by professional dental technicians for coloring pre-sintered zirconia restorations made from zirconia milling blanks for the manufacturing of permanent dental restorations for adults.

Indications

Individualization of anatomically reduced and fully anatomical (monolithic) crowns and bridges made of zirconium oxide before sintering.

Contraindications

Intolerance to contained components.

Adverse effects

We are not aware of any risks or adverse effects related to the GC Initial Zirconia Coloring Liquid.

Warnings

1. Causes serious eye damage.
2. May cause an allergic skin reaction.
3. Processing of GC Initial Zirconia Disk produces dust which can lead to eye and skin irritation and may cause lung damage. Always ensure that the extraction unit of the machine is in good working order. Personal Protective Equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
4. Do not use autoclave to sterilize the framework.
5. Dispose of contents/container in accordance with local/regional/national/international regulations.

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
GC Initial Zirconia Coloring Liquid

EN One liquid – two techniques

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EN 1. Painting technique – individual & detailed procedure

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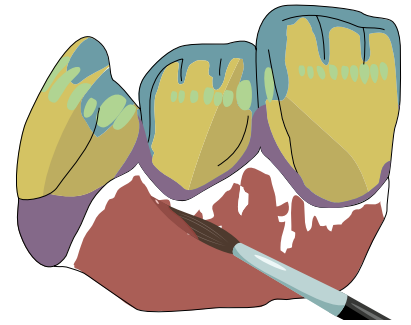EN Application of the different Body - Cervical - Effect colors by painting

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EN Drying

DE Trocknen

EN Final aesthetics with IQ ONE SQIN concept

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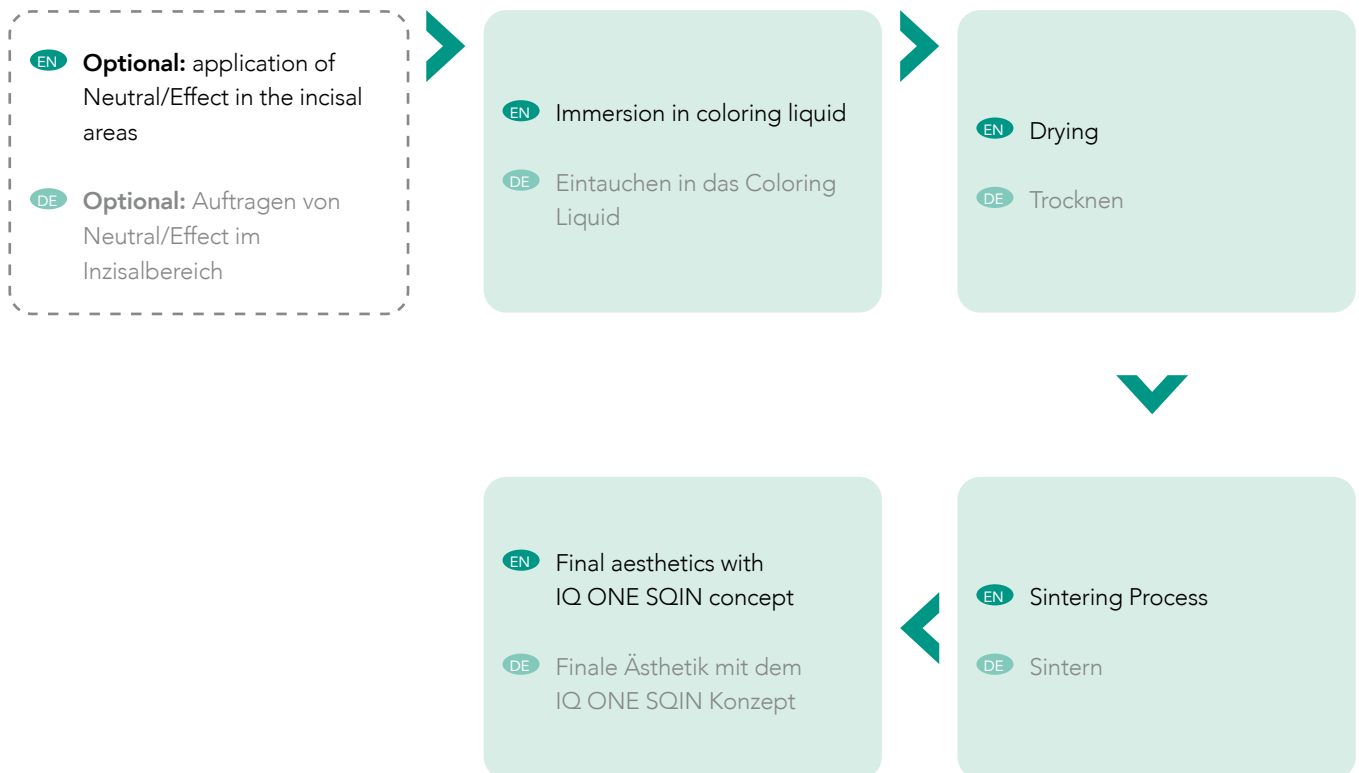
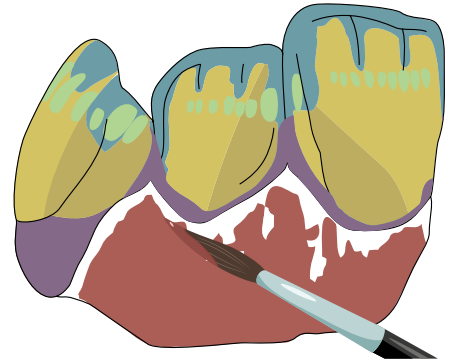
EN Procedure for A2-shade

As an example, the procedure for coloring fully anatomical crowns in the "A2-shade" (V-Classic) is described below. For "B/C/D" shades, the corresponding "B/C/D-Body" and "B/C/D-Cervical" coloring liquids are used, see table page 13.

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EN Tip:

It is recommended to create color samples with the appropriate zirconia wall thickness before coloring the objects. Depending on the wall thickness of the restoration, the coloring liquid must be applied more or less often.

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EN A. Preparation of the zirconia restorations after the milling process

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Optional use of Marking Liquid

- To facilitate the visualization during application, Marking Liquids – red, blue, yellow, green – can be used.
- Add 1 drop of the desired Marking Liquid to 10 ml Initial Zirconia Coloring Liquid to visualize the area to be colored.
Note:
Use a dedicated jar/container and keep it closed when not in use to avoid contamination.
- It might be possible that the Marking Liquid loses its effect after a few days. Re-add it to the Coloring Liquid.
Do not exceed five times.

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
EN 2. Application of the Body A Liquid

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EN Completely dip the brush into the coloring liquid.
Wipe off the excess drop from the brush tip.
Use brush size 3.

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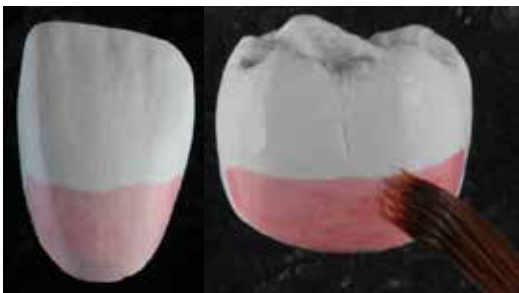EN Stroke the brush over the crown area to be colored as often as indicated in table page 9.
Before each new brush stroke, dip the brush (again) in the liquid.
Wipe off the excess liquid from the brush tip.

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EN **IMPORTANT:**

Clean the brush under running water before each color change.
Then dry the brush before immersing it in the new coloring liquid.

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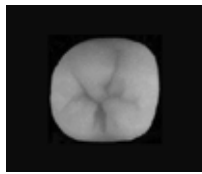
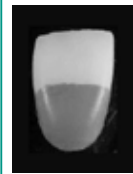
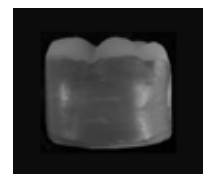
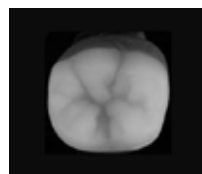
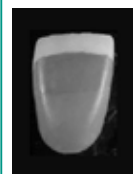
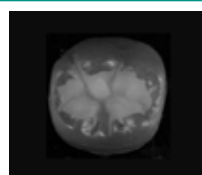
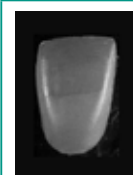
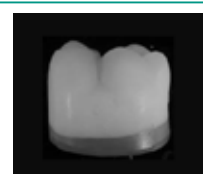
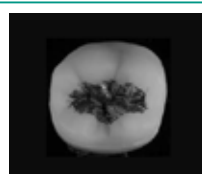
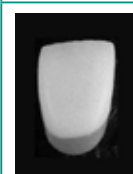
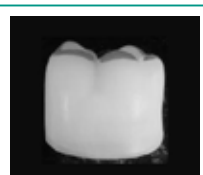
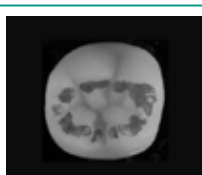
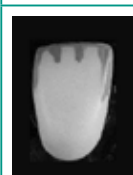
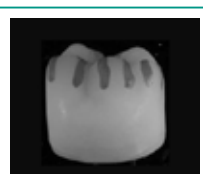
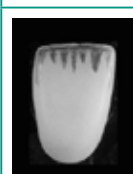
EN General advice for the gingival area

If parts of the gingival area need coloring, we recommend coloring these parts of the gingiva after having colored the tooth-colored parts.

This prevents the gum coloring liquid from diffusing into the tooth-colored parts.

Depending on the zirconia wall thickness and the desired color intensity, the gingiva parts can be coloured 3x - 5x with the brush.

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EN 3. Drying


Dry the colored zirconia restoration before the sintering process at 80 °C for 60 minutes in a dedicated small oven with a heating lamp.

To ensure a uniform color result, the dental restoration should be dried immediately after the color application.

Position the crowns and bridges on their occlusal surface while drying.

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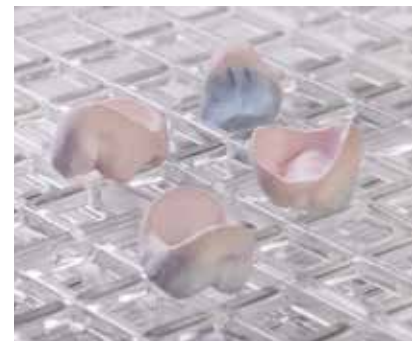EN 4. Sintering of the zirconia restorations

Sinter the colored and dried zirconia restoration according to the zirconia manufacturer's recommendation.
For GC Initial Zirconia disk: see https://europe.gc.dental/sites/europe.gc.dental/files/products/downloads/initialzrdisk/ifu/IFU_Initial_Zr_Disk_W.pdf

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EN Final aesthetical outcome

After the coloring and sintering of the zirconia restoration according to this IFU, the final aesthetical outcome can be reached by using the "GC Initial IQ ONE SQIN" concept.

Please refer to the dedicated Instruction For Use:
https://europe.gc.dental/sites/europe.gc.dental/files/products/downloads/initialiqonesqin/manual/MAN_Initial_IQ_One_Sqin_technical_manual.pdf

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Optional use of Marking Liquid

- To facilitate the visualization during application, Marking Liquid – red, blue, yellow, green – can be used.
- Add 1 drop of the desired Marking Liquid to 10 ml Initial Zirconia Coloring Liquid to visualize the area to be colored.
Note:
Use a dedicated jar/container and keep it closed when not in use to avoid contamination.
- It might be possible that the Marking Liquid loses its effect after a few days. Re-add it to the Coloring Liquid.
Do not exceed five times.

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2. Immersion in Body A Liquid

2. Eintauchen in das Body A Liquid

EN For a color gradation within the restoration, we recommend immersing the incisal third of the restoration in the CL-N liquid for 2-4 seconds and immediately thereafter completely immersing it in the corresponding Body coloring liquid for 10 seconds (see dilution chart page 14 for V-shades).

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EN During the immersion process, the Body coloring liquid and the Neutral liquid (CL-N) blend together, which creates an overlapping area in between. Immersion should take place immediately after the incisal liquid has been applied! Applying too much Neutral liquid (CL-N) will cause all the dentine color to become brighter. Carefully dab away any excess liquid with a paper tissue – in particular the inner side of the restoration and the occlusal area. Alternative is to blow away the excess liquid using compressed air.

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
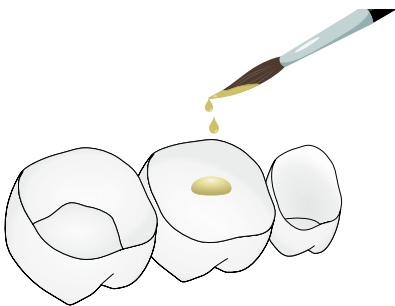
EN 3. Drying

Dry the colored zirconia restoration before the sintering process at 80 °C for 60 minutes in a dedicated small oven or with a heating lamp. To ensure a uniform color result, the dental restoration should be dried immediately after the color application. Position the crowns and bridges on their occlusal surface while drying.

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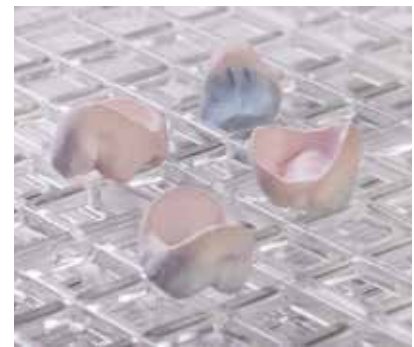
EN 4. Sintering of the zirconia restorations

Sinter the colored and dried zirconia restoration according to the zirconia manufacturer's recommendation.

For GC Initial Zirconia Disk: see https://europe.gc.dental/sites/europe.gc.dental/files/products/downloads/initialzrdisk/ifu/IFU_Initial_Zr_Disk_W.pdf

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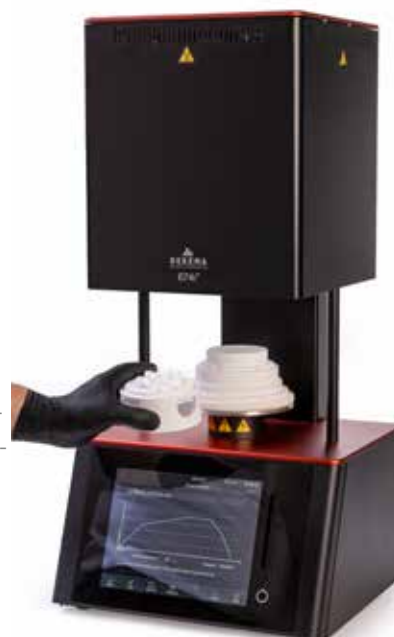EN Final result after the sintering process

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EN Final aesthetic outcome

After the coloring and sintering of the zirconia restoration according to this IFU, the final aesthetic outcome can be reached by using the "GC Initial IQ ONE SQIN" concept.

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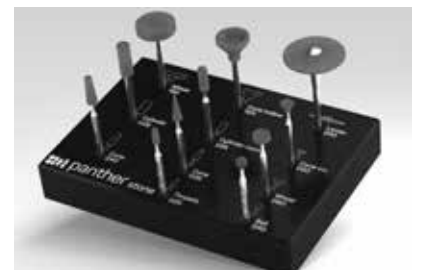
EN GC Initial Zirconia Coloring Liquid Set

Contents

- 4 Body colors: Body A / Body B / Body C / Body D
- 4 Cervical colors: Cervical A / Cervical B / Cervical C / Cervical D
- 3 Effect colors: Blue, Grey, Pink
- 1 Neutral
- 4 Marking Liquids: Red, Blue, Yellow, Green

DE GC Initial Zirconia Coloring Liquid Set

Inhalt

- 4 Body colors: Body A / Body B / Body C / Body D
- 4 Cervical colors: Cervical A / Cervical B / Cervical C / Cervical D
- 3 Effect colors: Blue, Grey, Pink
- 1 Neutral
- 4 Marking Liquids: Red, Blue, Yellow, Green

EN Liquid can only be stored in the original bottle. Keep it separated from edibles and ensure the bottle is tightly sealed. Avoid storing it with alkalis (lyes).

Close all bottles immediately after use. For reproducible results, shake each liquid before use.

Note:

The "Initial Zirconia Coloring Liquid" are sensitive to a slight color shift within their shelf life during storage or use. This however does not affect the color of the final sintered restoration.

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GC Initial Zirconia Disk Multilayer Elite

GC Initial Zirconia Coloring Liquid

EN Medical Device Regulation Information

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EN COMPOSITION

Erbiumchloride Hexahydrate < 8%, Ironchloride Hexahydrate < 6%, Praseodymiumacetate Hydrate < 6%, Manganesesulfate Monohydrate < 0.3%, Neodymiumchloride < %8, Copper (II) sulfate Pentahydrate < 0.01, Chromium acetate < 0.1%, Polyethylenglycol 6-9%, Hydrochloric Acid 0.2-0.4%, Water > 80%

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Erbiumchlorid-Hexahydrat < 8%, Eisen(III)-chlorid-Hexahydrat < 6%, Praseodymacetat-Hydrat < 6%, Mangansulfat-Monohydrat < 0,3%, Neodymchlorid < 8%, Kupfer(II)-sulfat-Pentahydrat < 0,1%, Chromacetat < 0,1%, Polyethylenglykol 6-9%, Salzsäure 0,2-0,4%, Wasser > 80%

EN Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <https://www.gc.dental/europe> or for The Americas <https://www.gc.dental/america>. They can also be obtained from your supplier.

For the Summary of Safety and Clinical Performance (SSCP) please see EUDAMED database (<https://ec.europa.eu/tools/eudamed>) or contact us at klema.qm@klema.at.

Undesired effects - Reporting: If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:

https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: klema.qm@klema.at.

In this way you will contribute to improve the safety of this product.

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GC NORDIC AB
GC AUSTRIA GmbH - Swiss Office
GC AUSTRALASIA DENTAL PTY LTD
GC ASIA DENTAL PTE. LTD.