Prior to use, carefully read the instructions for use.

ISO FUNCTIONAL

REBASING AND RELINING IMPRESSION MATERIAL

For use only by a dental professional in the indications for use.

INDICATIONS FOR USE

Because of its unique slow setting characteristics ISO FUNCTIONAL enables production of extremely accurate impressions. This makes it particularly useful as an impression material for rebasing and muscle trimming. May also be used for general impressions such as dentures, root surfaces and posts.

DIRECTIONS FOR USE

- Remove the flange and tissue contact surface from the denture to a depth of 2-3 mm using steel burrs or abrasive points.
- Cover the relieved surface of the denture with ISO FUNCTIONAL which has been softened over a flame.
- Soften again by passing through hot water at 45°C (113°F). Insert the denture into the mouth and instruct the patient to bite and hold the denture in centric occlusion. Perform muscle trimming movements.
- The denture should be retained in the mouth for 1 day to allow a truly functional impression to be formed under normal chewing conditions.

Note:

Advise patients to avoid hot drinks during this period.
- The impression formed is used in standard rebasing procedures.

NOTES

- To avoid ISO FUNCTIONAL becoming too sticky do not overheat.
- If ISO FUNCTIONAL adheres to the denture, let it set hard and remove with laboratory burs and hand instruments. Remove residual traces using solvent.

STORAGE

Recommended for optimal performance, store in a cool and dark place (4-25°C / 39-77°F).

PACKAGES

ISO FUNCTIONAL Sticks 120 g (15 pieces)

CAUTION

- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
- In case of contact with eyes, remove and flush with water.
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:

https://www.gc.dental/europe

or for The Americas

https://www.gc.dental/america

They can also be obtained from your supplier.

Undesired effects- Reporting:
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:

https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gc.dental
In this way you will contribute to improve the safety of this product.

UK Responsible Person
GC UNITED KINGDOM Ltd.
Coopers Court Newport Pagnell, Buckinghamshire,MK16 8JS, United Kingdom

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