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COMPOZIȚIE

Etanol, MDP, MDPF

EXTRA-ORAL application in all cases

1. Sandblast metal surfaces with alumina (aluminum oxide), and blow clean with air gun. Avoid contamination with oil, separator or moisture.
2. Dispense 2-3 drops of METAL PRIMER Z into a clean dispensing dish. Using a brush, apply a thin layer to the bonding surface. Allow to dry. Immediately apply a resin material to the treated surface according to the instructions of the manufacturer.
INTRA-ORAL application in case of repair of prostheses.
1. Place rubber dam.
Note:
Be sure to use rubber dam and avoid direct contact with oral tissue.
2. Roughen the metal surface with a sandblaster or a coarse diamond bur. Avoid contamination with oil, separator or moisture.
3. Dispense 2-3 drops of METAL PRIMER Z into a clean dispensing dish. Using a brush, apply a thin layer to the bonding surface. Allow to dry. A second coat may be applied if necessary.
4. Immediately apply a resin material to the treated surface according to the instructions of the manufacturer.

STORAGE
Recommended for optimal performance, store at room temperature (4-25°C) (39-77.0°F).

PACKAGE
METAL PRIMER Z 5 ml (1)
CAUTION
1. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
2. METAL PRIMER Z is highly flammable liquid and vapour. Do not use near open flame. Keep away from sources of ignition. Keep away from direct sunlight. Use in a well ventilated place.
3. Take care to avoid ingestion of the material.
4. In case of contact with oral tissues, immediately remove with a cotton pellet or a sponge. Flush immediately after cementing.
5. In case of contact with skin, remove immediately with a cotton pellet or a sponge. Rinse with soap and water.
6. In case of contact with eyes, flush immediately with water and seek medical attention.
7. Replace the cap immediately after use. If dew condensation is present on the bottle of METAL PRIMER Z, immediately wipe off.
8. Do not mix with any other primers.
9. Store in a well ventilated place.
10. Dispose of all waste according to local regulations.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:
http://www.gceurope.com
They can also be obtained from your supplier.

CLEANING AND DISINFECTING:
MULTI-USE DELINEAR SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.
DO NOT IMMERSE. Thoroughly clean device to prevent infection and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

For the Summary of Safety and Clinical Performance (SSCP) please see EUDAMED database (https://ec.europa.eu/tools/eudamed/) or contact us at Regulatory.gce@gc.dental

Undesired effects- Reporting:
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:
https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gc.dental
In this way you will contribute to improve the safety of this product.

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PACKAGING
METAL PRIMER Z 5 ml (1)

CAUTION
1. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
2. METAL PRIMER Z is highly flammable liquid and vapour. Do not use near open flame. Keep away from sources of ignition. Keep away from direct sunlight. Use in a well ventilated place.
3. Take care to avoid ingestion of the material.
4. In case of contact with oral tissues, immediately remove with a cotton pellet or a sponge. Flush immediately after cementing.
5. In case of contact with skin, remove immediately with a cotton pellet or a sponge. Rinse with soap and water.
6. In case of contact with eyes, flush immediately with water and seek medical attention.
7. Replace the cap immediately after use. If dew condensation is present on the bottle of METAL PRIMER Z, immediately wipe off.
8. Do not mix with any other primers.
9. Store in a well ventilated place.
10. Dispose of all waste according to local regulations.

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