Prior to use, carefully read the instructions for use.

GC Modeling Liquid

LIQUID TO MODEL COMPOSITE MATERIALS FOR DIRECT RESTORATIONS

For use only by a dental professional in the indications for use.

WHAT IS GC MODELING LIQUID?

GC Modeling Liquid is a liquid developed to ease the placement and the modeling of composite materials for direct restorations when using the brush technique. It is compatible with all direct composites. It can also be used to pre-treat fibres in direct and indirect procedures.

INDICATIONS FOR USE

- Modeling of composite materials for direct restorations using the brush technique.
- Pre-treatment, reactivation, repair and bonding of fibres.

CONTRAINDICATIONS

Avoid use of this product in patients with known allergies to methacrylate monomer or methacrylate polymer.

COMPOSITION

Dimethacrylate, methacrylate, initiator, silicon dioxide, stabilizer

USE OF GC MODELING LIQUID FOR DIRECT RESTORATIONS

- Dispense one drop of GC Modeling Liquid on a pad.

NOTE:

Use immediately after dispensing, or protect the product from the light using a suitable light cover. Replace the bottle cap immediately after use.

Touch the drop of GC Modeling Liquid with your brush in order to moisturise the brush.

NOTE: Do not over-wet the brush as this could modify the physical properties of the composite material and hinder the longevity of the restoration.

- Remove thoroughly the excesses from the brush using a paper tissue.

NOTE: The brush will then look dry, but the small amount of GC Modeling Liquid still on the brush will be sufficient to ease the placement of composites.
- Apply the composite material (as per manufacturer’s instructions) onto the tooth preparation.
- Shape the composite using the brush moisturised with GC Modeling Liquid. The GC Modeling Liquid will prevent the composite from sticking to the brush, allowing it shape it efficiently and obtain a smooth surface.
- When necessary, the brush can be moisturised again with GC Modeling Liquid and thoroughly dried on a paper tissue before touching the restoration again.
- Refer to manufacturer’s instructions related to curing, finishing & polishing of the composite restoration.

PRE-TREATMENT, REACTIVATION, REPAIR AND BONDING OF FIBRES
Bonding and positioning fibres

When placing fibres on each other (as in the case of everStickNET) apply a thin layer of Modeling Liquid on top of the cured fibre strip. Then press the next fibre piece tightly in place using a StickREFIX D silicone instrument, and light-cure through the silicone for 10 seconds. Repeat the procedure for a third fibre, if necessary.

Adjusting the length of fibres

If the fibre (such as everStick POST, everStick C&B, everStick Perio) needs to be shortened during the finishing phase of the splint / bridge, cut the fibre with a diamond bur, apply Modeling Liquid onto the fibre surface to activate it. Blow it into a thin layer to remove excess resin and light cure for 10 seconds.

Pre-treatment of prosthetic fibre constructions before cementation
Use a carbundum bur to lightly roughen the surfaces to be cemented. Rinse with water and air-dry the surfaces. Apply Modeling Liquid to the roughened bonding surfaces to activate it, protect it from light and let it take effect for 3 to 5 minutes (you can use, for instance, a metal cap as protection against light). Blow it into a thin layer to remove excess resin and light cure for 10 seconds.

Direct fibre-reinforced temporary bridge with everStickC&B
After cutting the fibre to the desired length apply a thin layer of Modeling Liquid on top of the abutments. Do not light-cure the Modeling Liquid yet. Place the fibre bundle on top of the abutments. Use a StickREFIX D silicone instrument to press the fibre into the correct position and shape. Initially light-cure the fibre through the silicone for 10 seconds. Finalise the polymerisation by light-curing the entire fibre frame for 40 seconds per area.

STORAGE

Recommended for optimal performance, store in a cool, dark place (4-25°C / 39.2-77.0°F).

PACKAGES

GC Modeling Liquid: Bottle of 6 mL
Modeling brushes: GRADIA Brush holder (x 2), GRADIA Brush Flat (x 10), GRADIA Brush Round (x 10)

CAUTION

- In case of contact with oral tissues, immediately remove with a sponge or cotton pellet. Flush with water after treatment is completed. To avoid contact, a rubber dam and/or cocoa butter can be used to isolate the operation field from the skin or oral tissue.
- In case of contact with skin, immediately remove with a sponge or cotton pellet. Rinse with soap and water.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- Take care to avoid ingestion of the material.
- Do not mix with any other similar products.
- Do not reuse a brush with other products.
- Dispose of all wastes according to local regulations.
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <http://www.gceurope.com>. They can also be obtained from your supplier.

CLEANING AND DISINFECTING:
MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.
DO NOT IMMERSE: Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional / national guidelines.

For the Summary of Safety and Clinical Performance (SSCP) please see EUDAMED database (<https://ec.europa.eu/tools/eudamed/>) or contact us at Regulatory.gce@gc.dental

Undesired effects – Reporting:
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gc.dental. In this way you will contribute to improve the safety of this product.

UK Responsible Person
GC UNITED KINGDOM Ltd.
Coopers Court Newport Pagnell, Buckinghamshire, MK16 8JS, United Kingdom

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- Tragen Sie das Compositematerial (gemäß Herstellerangaben) auf die Zahnpräparation auf.
NOTE: Use immediately after dispensing, or protect the product from the light using a suitable light cover. Replace the bottle cap immediately after use. Touch the drop of GC Modeling Liquid with your brush in order to moisturise the brush. NOTE: Do not over-wet the brush as this could modify the physical properties of the composite material and hinder the longevity of the restoration.
- Remove thoroughly the excesses from the brush using a paper tissue. NOTE: The brush will then look dry, but the small amount of GC Modeling Liquid still on the brush will be sufficient to ease the placement of composites.
- Apply the composite material (as per manufacturer’s instructions) onto the tooth preparation.
- Shape the composite using the brush moisturised with GC Modeling Liquid. The GC Modeling Liquid will prevent the composite from sticking to the brush, allowing it shape it efficiently and obtain a smooth surface.
- When necessary, the brush can be moisturised again with GC Modeling Liquid and thoroughly dried on a paper tissue before touching the restoration again.
- Refer to manufacturer’s instructions related to curing, finishing & polishing of the composite restoration.

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