Prior to use, carefully read the instructions for use. **EN**

OPTIGLAZE™

NANOFLUENT LIGHT CURED PROTECTIVE COATING FOR COMPOSITE AND ACRYLIC INDIRECT RESTORATIONS

For use only by a dental professional in the recommended indications.

INDICATIONS FOR USE

- Coating of indirect composite surfaces (fissures and interproximal areas)
- Coating of denture base surfaces
- Coating of provisional materials

CONTRAINDICATIONS

Avoid use of this product in patients with known allergies to acrylate or methacrylate monomer.

PRECAUTIONS

- It is not recommended to use OPTIGLAZE on high stress bearing areas (occlusal surfaces).
- The use of OPTIGLAZE does not replace the mechanical polishing of aesthetic indirect composite restorations such as GRADIA.

DIRECTIONS FOR USE

Preparation
After contouring the resin surface, finish with carbide bur or coarse silicone point. Clean the surface with a steam cleaner or ultrasonic cleaner and then dry. If surface still contains oil residue, the surface can be further cleaned with ethanol (alcohol).

NOTE:

- OPTIGLAZE does not adhere to a polished surface. Make sure surfaces are to be coated are rough.
- The air-inhibited layer of the resin surface to be coated should be removed before applying OPTIGLAZE as it may inhibit the proper curing of OPTIGLAZE.
- The use of ethanol (alcohol) on acrylic resin may cause micro-cracks. A mild detergent can be used to clean the acrylic resin.

Application

- Dispense a few drops into a dispensing dish. Replace bottle cap immediately after dispensing

2. Dip the brush in (within 1 minute after dispensing) to the surface with a brush. DO NOT AIR BLOW.

Light Curing

Light cure with a suitable light curing device.

LABLIGHT DUO	90 sec.
LABLIGHT LV-III_II	3 min.
STEPLIGHT SL-I	Distance from light source <3 cm 10 sec. Distance from light source >3 cm 20 sec.
Halogen Light	40 sec.
LED Light (with light source having wavelength lower than 430 nm)	40 sec.

NOTE:

- If surface is tacky, repeat light curing.
- If necessary, rotate the prosthesis a half-turn or turn-over and light cure again to ensure all surfaces are completely cured.
- Additional build-up cannot be performed on surfaces coated with OPTIGLAZE. Remove the coated layer with a bur and then build-up with OPTIGLAZE.
- OPTIGLAZE cannot be cured by a conventional LED light curing device with blue LED only.
- When using other manufacturers' light curing devices, use final curing mode. Refer to the respective manufacturers' instructions for use.

STORAGE

Recommended for optimal performance, store at temperature of 4-25°C (39,2-77,0°F).

PACKAGE

OPTIGLAZE 15 mL (1)

CAUTION

- OPTIGLAZE is flammable. Do not use near open flame. Keep away from sources of ignition. Do not store large quantities in one area. Keep away from open flame.
- OPTIGLAZE is volatile. Use in a well ventilated area. Replace cap immediately after dispensing.
- In case of contact with eyes, flush immediately with water and seek medical attention
- In case of contact with skin, immediately remove with a sponge or cotton pellet. Flush with water.
- Wear protective gloves. In particular, avoid contact of uncured material with skin.
- Avoid getting material on clothing.
- Avoid inhalation or ingestion of material.
- Do not mix with other dental products.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
- In rare cases the product may be irritating to some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <http://www.gceurope.com>

They can also be obtained from your supplier.

CLEANING AND DISINFECTING MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged. DO NOT IMMERGE. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

For the Summary of Safety and Clinical Performance (SSCP) please see attached GHS database (<https://ec.europa.eu/tools/eudamed/>) or contact us at Regulatory.gce@gdc.dental

Undesired effects: Reporting:
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system by selecting the proper authority of your country accessible through the following link:
https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gdc.dental. In this way you will contribute to improve the safety of this product.

Last revised: 10/2022

MANUFACTURED BY:
GDC DENTAL PRODUCTS CORP.
2-285 Torimatsu-cho, Kasugai, Aichi 486-0844, Japan

DISTRIBUTED BY:
GCE CORPORATION
76-1 Hásunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

EU: GCE EUROPE N.V.
Researchpark Haardroev-Luven 1240, Interlevenlaan 33,
B-3001 Leuven, Belgium TEL: +32 16 74 10 00

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NOTE:
1) If surface is tacky, repeat light curing.

2) If necessary, rotate the prosthesis a half-turn or turn-over and light cure again to ensure all surfaces are completely cured.

3) Additional build-up cannot be performed on surfaces coated with OPTIGLAZE. Remove the coated layer with a bur and then build-up with OPTIGLAZE.

4) OPTIGLAZE cannot be cured by a conventional LED light curing device with blue LED only.

5) When using other manufacturers' light curing devices, use final curing mode. Refer to the respective manufacturers' instructions for use.

STORAGE
Recommended for optimal performance, store at temperature of 4-25°C (39,2-77,0°F).

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