Prior to use, carefully read the instructions for use.

OPTIGLAZE™ color
LIGHT-CURED CHARACTERIZATION MATERIAL

For use only by a dental professional in the indications for use.
INDICATIONS FOR USE
1. Characterization of direct & indirect composite restorations, acrylic denture base and artificial acrylic teeth.
2. For obtaining surface smoothness and wear resistance of restorations made of composite resin, acrylic denture base and artificial acrylic teeth.
Note:
Not intended for characterization of natural teeth.

CONTRAINDICATIONS
Avoid use of this product in patients with known allergies to acrylate / methacrylate monomers or acrylate / methacrylate polymers.

COMPOSITION
Methacrylate, multifunctional monomer, silicon dioxide, initiator, pigment

DIRECTIONS FOR USE
SURFACE CHARACTERIZATION
1. After contouring the resin surface, finish with a carbide bur or coarse silicone point. The oxygen inhibition layer must be removed as it inhibits the polymerization of OPTIGLAZE color. Laboratory use: Sandblast with 25-50 µm alumina (0.15 MPa/1.5 bar), clean and dry.
2. If the surface still contains oil residue, the surface can be cleaned further with ethanol (alcohol). The use of ethanol (alcohol) on acrylic resin may cause micro cracks. A mild detergent can be used to clean the acrylic resin.
3. When applying OPTIGLAZE color on hybrid ceramic blocks such as CERAMART, treat the surface with CERAMIC PRIMER II.
4. Shake the bottle of OPTIGLAZE color well. Dispense a few drops in a dispensing dish. Dilute by adding OPTIGLAZE color Clear to the desired shade. Apply a thin layer to the resin surface using the brush provided. Do not air blow.
5. Light cure with a suitable light curing device. (See table 1)
Light cure tip as close as possible to the surface.
Laboratory use:
Set a 2 cm platform in the curing device and place the prosthesis on it with the coated surface facing the light. Reverse the prosthesis and repeat the light curing to fully cure the undercut or area in shadow.

INTERNAL CHARACTERIZATION
1. Build up the dentin layer with composite and light cure. Follow the instructions of your composite system.
Note:
Do not remove the oxygen inhibition layer. Apply OPTIGLAZE color on the oxygen inhibition layer. If the oxygen inhibition layer is removed, treat the surface with CERAMIC PRIMER II (blow dry) and COMPOSITE PRIMER (light cure) before applying OPTIGLAZE color.

2. Shake the bottle of OPTIGLAZE color well. Dispense a few drops in a disposable dispensing dish. Apply a thin layer of OPTIGLAZE color of choice to the dentin surface with an instrument or brush (e.g. the convex and concave of dentin mamelon or incisal area). Mix multiple colors to obtain the desired shade if needed. Do not air blow.
3. Light cure with a suitable light curing device. (See table 1)
4. Continue building up with enamel shade on the oxygen inhibition layer. Follow the instructions of your composite system.
5. Finish and polish using standard technique or apply OPTIGLAZE color Clear or Clear HV on the entire surface.

REPAIR OF RESTORATION (e.g. when the gloss is lost).
1. Roughen the surface of restoration using a carbide bur and paper cone.
2. Clean the surface with a steam cleaner or ultrasonic cleaner and then dry with oil free air.
Note:
If surface is contaminated by oil residue, the surface can be cleaned further with ethanol (alcohol). The use of ethanol (alcohol) on acrylic resin may cause micro cracks. A mild detergent can be used to clean the acrylic resin.

3. Apply OPTIGLAZE color Clear or Clear HV shade to the surface. Do not air blow.
4. Light cure with a suitable light curing device. (See table 1)
Table 1: Light Curing Time

GC LABOLIGHT DUO	90 sec.
GC LABOLIGHT LV-III II	5 min.
GC STEPLIGHT SL-I	Distance from light source <3 cm 10 sec. Distance from light source >3 cm 20 sec.
Halogen Light	40 sec.
Plasma Arc	8 sec.
LED Light (with light source having wavelength lower than 430 nm)	40 sec.
Table 2: Depth of Cure	
Orange, Yellow, Redbrown, Red	0.07 mm
A Plus, B Plus, C Plus, White, Ivory White, Pink Orange, Pink, Olive, Lavender, Grey, Blue	0.15 mm
Clear, Clear HV	0.5 mm

STORAGE
Recommended for optimal performance, store at room temperature of (+25°C) (39.2-77.0°F).

PACKAGE:
Complete set:
A Plus 2.6 mL, B Plus 2.6 mL, C Plus 2.6 mL, White 2.6 mL, Ivory White 2.6 mL, Yellow 2.6 mL, Orange 2.6 mL, Pink Orange 2.6 mL, Pink 2.6 mL, Red Brown 2.6 mL, Olive 2.6 mL, Lavender 2.6 mL, Grey 2.6 mL, Blue 2.6 mL, Red 2.6 mL, Clear 5 mL, Clear HV 5 mL, Disposable Dispensing Dish, Flat Brush, Round Brush

CAUTION
1. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
2. In particular avoid skin contact with the material.
3. Avoid getting material on clothing.
4. OPTIGLAZE color is highly flammable liquid and vapor. Do not use near open flame. Keep away from source of ignition.
5. OPTIGLAZE color is volatile and may cause respiratory irritation. Use in a well ventilated space. Replace cap immediately after dispensing.
6. It is not recommended to use OPTIGLAZE color on high pressure bearing area.
7. In case of contact with eyes, flush immediately with water and seek medical attention.
8. In case of contact with skin, immediately remove with a sponge or cotton pellet. Flush with water.
9. The oxygen inhibited layer of the resin surface to be coated should be removed before applying OPTIGLAZE color as it may hinder the proper curing of OPTIGLAZE color.
10. Dispense a few drops of OPTIGLAZE color into a disposable dispensing dish immediately before applying as the material contains a volatile solvent.
11. Store locked up.
12. Dispose all waste according to local regulation.
13. In rare cases the product may cause respiratory discomfort in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

CLEANING AND DISINFECTING
MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.
DO NOT IMMERS: Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.
Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:
<http://www.gecorpae.com> or <http://www.gcameric.com>
They can also be obtained from your supplier.
For the Summary of Safety and Clinical Performance (SSCP) please see EU/AMED database (<https://ec.europa.eu/tools/euamed/>) or contact us at Regulatory.gcc@dc.dental

Undesired effects - Reporting:
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gcc.dental in this way you will contribute to improve the safety of this product.

UK Representative Person
GC UNITED KINGDOM Ltd.
Coopers Court Newport Pagnell, Buckinghamshire, MK16 8JS United Kingdom
Last revised: 02/2024

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