

Placement



Clean the tooth surface with pumice and water.



Apply the non-runny ORTHO Gel for 10 sec. Rinse but do not desiccate.



Express Fuji ORTHO LC onto the bracket and press firmly against moist tooth.



Remove excess with scaler, adjust and tack-cure if needed (2-5 sec). Continue with quadrant or full arch.



Light-cure each bracket for 20 seconds.

Dr.A. Komori, Japan

Removal



Grasp the bracket with a ligature cutter and give a sharp clockwise rotation to the bracket.



If resistance is met, try again twisting in the opposite direction after desiccating the tooth around the bracket (using an air syringe).



Remove any residual cement with a scaler or rotary instrument.

Placement



Clean the tooth surface with Pumice and Water. Rinse. Do not dessicate. Coat the inner surface of the band.



Seat the band.



Remove the excess (with scaler).



Light-cure for 20 sec. from occlusal surface.



Final results.

Removal



Remove bonded bands from the buccal or lingual side using band removing pliers.



Remove any residual material with adhesive removing pliers, scaler or rotary instrument.