Prior to use, carefully read the instructions for use.

EN

GC PLIERS PLIERS FOR REMOVAL OF PROSTHETIC APPLIANCES

For use only by a dental professional in the indications for use.

INDICATIONS FOR USE

- Removal of luted temporary prosthetic appliances.
- Removal of all types of prosthetic appliances during trial fitting.

DIRECTIONS FOR USE

- REMOVAL OF TEMPORARY PROSTHETIC APPLIANCES
 - Use the GC PLIERS without the rubber pads.
 - Hold the cervical area of the crown with the tips of the pliers.
 - Twist the crown alternatively towards the mesial and distal direction.
 - Remove the crown in a labiolingual direction.
- REMOVAL OF FINAL PROSTHETIC APPLIANCES DURING TRIAL FITTING
 - Use the GC PLIERS with the rubber pads by pushing the pads on the tips of the pliers.
 - Place emery powder on the surface of the pads in order to improve the grip.
 - Continue with 1 b).

Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

CLEANING

Cleaning of the GC Pliers made out of stainless steel, can be accomplished by hand scrubbing, ultrasonic cleaning or thermodesinfection with a non-corrosive, neutral detergent. For further details consult the respective manufacturer's instructions.

STERILISATION

GC Pliers may be routinely sterilized by the following methods : autoclave, chemiclave or dry heat.

STORAGE

Recommended for optimal performance, store in a dry place.

PACKAGES

- Introductory set: one pair of pliers, 50 rubber pads, 4 g emery powder
- 50 rubber pads
- 4 g emery powder

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <http://www.gceurope.com> They can also be obtained from your supplier.

Undesired effects- Reporting: If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gc.dental

In this way you will contribute to improve the safety of this product.

UK Responsible Person
GC UNITED KINGDOM Ltd.
Coopers Court Newport Pagnell Buckinghamshire MK16 8JS United Kingdom

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