## GC RELINE BONDING AGENT

GC RELINE BONDING AGENT is a specially designed methacrylate base bonding agent that enhances the bond between GC Reline Material and denture base materials.

For use only by a dental professional in the recommended indications.

### RECOMMENDED INDICATIONS:

For use as a bonding agent when processing GC RELINE Hard Denture Chairside Reline Material.

### CONTRAINDICATIONS:

- Direct pulp capping.
- Avoid use of this product in patients with known allergies to glass ionomer cement, methacrylate monomer or methacrylate polymer.

### DIRECTIONS FOR USE:

*Note: Consult the instructions for use for the GC RELINE Hard Denture Chairside Reline Material for complete processing information.*

- Roughen the area of the denture to be relined. Clean and dry the denture thoroughly.
- Coat the artificial teeth, labial and buccal surfaces of the denture with COE LUBRICANT or Vaseline using a cotton swab being careful to stay at least 3 mm (1/8 inch) from the peripheral border*Do not contaminate the tissue side of the denture with lubricant.*

- Using the brush provided, coat the area to be relined with a uniform coating of the GC RELINE BONDING AGENT. Air-dry the bonding agent for approximately 10 seconds to allow the solvent to evaporate.

Note: GC RELINE BONDING AGENT is highly volatile; therefore be sure to replace and tightly close the cap after each use.

- Continue processing the GC RELINE Hard Denture Chairside Reline Material following the recommended procedures.

### CAUTIONS:

- The solvent in GC RELINE BONDING AGENT is highly volatile and flammable. Avoid sources of ignition and keep tight capped when not in use.
- In case of contact with skin, wash the affected area thoroughly.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- Wash the brush with GC RELINE liquid or ethanol after using the GC RELINE BONDING AGENT.
- Personal Protective Equipment (PPE) such as gloves, face mask and safety eyewear should always be worn.

### STORAGE:

Recommended for optimal performance, store at normal temperature 15-25 °C (60 - 77 °F) CLEANING AND DISINFECTING MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged. DO NOT IMMERSE. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional / national guidelines. Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the Safety Data Sheets available at: <https://www.gc.dental/europe> or for the Americas: <https://www.gc.dental/america> They can also be obtained from your supplier. Undesired effects-Reporting: If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: [https://ec.europa.eu/growth/sectors/medical-devices/contacts\\_en](https://ec.europa.eu/growth/sectors/medical-devices/contacts_en) as well as to our internal vigilance system: [vigilance@gc.dental](mailto:vigilance@gc.dental) In this way you will contribute to improve the safety of this product.

UK Responsible Person  
GC UNITED KINGDOM Ltd.  
Coopers Court Newport Pagnell,  
Buckinghamshire,  
MK16 8JS, United Kingdom

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