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Pror to use, carefully read the instructions for use.

**EN**

## CERAMIC PRIMER II

PRIMER FOR CERAMIC AND COMPOSITE BONDING

CERAMIC PRIMER II is a priming agent designed to treat the adherend surface of ceramic and composite prostheses to promote a strong bond to resin-based materials as resinased cements in case of luting procedures or direct composites in case of repairs.

For use only by a dental professional in the indications for use.

**INDICATIONS FOR USE**

1. Priming of the adherend surfaces of indirect restorations made of ceramics (glass ceramics, zirconia and alumina), hybrid ceramics (e.g. CERASMART) and composite before luting procedures.
2. Intra-oral and extra-oral repair of fractured indirect restorations made of ceramics, hybrid ceramics and composites.

**CONTRAINDICATIONS**

Avoid use of this product in patients with known allergies to methacrylate monomer, methacrylate polymer or ethanol.

**COMPOSITION**

Ethanol, dimethacrylate, phosphoric acid ester monomer, silane coupling agent

**DIRECTIONS FOR USE**

I. For luting procedure
1. Treat the bonding surfaces of prosthesis according to the respective manufacturer's instructions and then clean it using a suitable cleaning method. We recommend cleaning in an ultrasonic bath or with alcohol, then rinse and dry with an oil-free air syringe.
2. Bottle type: Dispense one drop of CERAMIC PRIMER II into a dispensing dish. Unit dose type: Unscrew cap of unit dose counter-clockwise. Make sure to identify the top and bottom of the unit dose capsule before opening.
3. Apply a thin layer to the bonding surface of the ceramic or composite prosthesis using a micro-tip applicator and dry with an air syringe.
4. Apply suitable resin-based luting cement to the primed surface of the prosthesis. For these steps, refer to the manufacturer's instructions for use of the corresponding luting cement.

II. For Intra-oral Repair

1. Roughen the bonding surface of ceramic or composite using a coarse diamond bur or a carbondium point.
2. Treat the bonding surfaces of the prosthesis according to the respective manufacturer's instructions and then clean it using a suitable cleaning method. Rinse and dry with an oil-free air syringe.
3. Bottle type: Dispense one drop of CERAMIC PRIMER II into a dispensing dish. Unit dose type: Unscrew cap of unit dose counter-clockwise. Make sure to identify the top and bottom of the unit dose capsule before opening.
4. Apply a thin layer to the fractured surface of the ceramic or composite prosthesis using a micro-tip applicator and dry with an oil-free air syringe.
5. Continue the repair by using a light-cured adhesive (e.g. G-Premio BOND) and a light-cured composite (e.g. G-ænial Universal Flo), referring to their respective manufacturer's instructions for use.
Note:
If the treated adherend surface is contaminated with saliva or blood, rinse with water and dry. Re-treat the surface according to the manufacturer's instructions and re-apply CERAMIC PRIMER II according to the instructions above.

If the treated adherend surface is contaminated with saliva or blood, rinse with water and dry. Re-treat the surface according to the manufacturer's instructions and re-apply CERAMIC PRIMER II according to the instructions above.

**STORAGE**
Recommended for optimal performance, store at room temperature (4-25°C, 39.2-77.0°F).
Keep away from direct sunlight.

**PACKAGING**

1. CERAMIC PRIMER II, 1 x bottle of 3 mL
2. CERAMIC PRIMER II, 10 x unit dose of 0,1 mL

(All packaging types may not be available in all countries.)

**CAUTION**

1. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
2. Avoid inhalation or ingestion of the material.
3. CERAMIC PRIMER II is highly flammable liquid and vapor. Do not use near open flame. Keep away from sources of ignition. Keep away from direct sunlight. Use in a well ventilated place.
4. In case of contact with oral tissues, immediately remove with a cotton pellet or a sponge. Rinse immediately after luting or repairing.
5. In case of contact with skin, remove immediately with a cotton pellet or a sponge. Rinse with soap and water.
6. In case of contact with eyes, flush immediately with water and seek medical attention.
7. Replace the cap of bottle immediately after use. If dew condensation is present on the bottle of CERAMIC PRIMER II, immediately wipe it off.
8. Unit dose is disposable. Do not recap or reuse to avoid cross contamination between patients.
9. If the cap of the unit dose is too tight to open, do not try to open it with force to avoid accidental contact of the liquid with skin and eyes. Use another unit dose.
10. Do not mix with any other primers.
11. Dispose of all waste according to local regulations.
12. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:
http://www.gceurope.com
or for The Americas
http://www.gcamerica.com
They can also be obtained from your supplier.

**CLEANING AND DISINFECTING:**
**MULTI-USE DELIVERY SYSTEMS:**
to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.
**DO NOT IMMERGE.**
Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional / national guidelines.

For the Summary of Safety and Clinical Performance (SSCP) please see EUDAMED database (https://ec.europa.eu/tools/eudamed/) or contact us at Regulatory.gce@gdc.dental

Undesired effects- Reporting:
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:
https://ec.europa.eu/growth/sectors/medical-devices/contacts\_en
as well as to our internal vigilance system: vigilance@gdc.dental
In this way you will contribute to improve the safety of this product.

UK Responsible Person
GC UNITED KINGDOM Ltd.
Coopers Court Newport Pagnell, Buckinghamshire, MK16 8JS, United Kingdom

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Ethanol, Dimethacrylat, Phosphorsäureester-Monomer, Silan-Haftvermittler

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