# ROCLEAN

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## OPAKOWANIE

Butelka 1000 ml

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MANUFACTURED by  
GC CORPORATION  
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

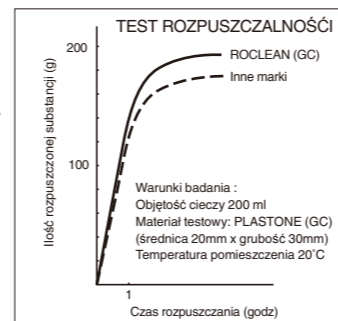
DISTRIBUTED by  
GC CORPORATION  
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

PRINTED IN JAPAN

GC EUROPE N.V.  
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33,  
B-3001 Leuven, Belgium TEL: +32 16 74 10 00

GC ASIA DENTAL PTE. LTD.  
11 Tampines Concourse, #03-05, Singapore 528729  
TEL: +65 6546 7588

190266SK

# ROCLEAN

DISSOLVING AGENT FOR DENTAL PLASTERS

Prior to use, carefully read the instructions for use.

EN

ROCLEAN (GC) is a cleaning agent for dissolving and removing the residual dental plasters and the gypsum-bonded investments from the prosthetic appliances.

## USAGE

1. Removal of the residual dental plasters from dentures, impression trays, dowel pins, and so forth.
2. Removal of the gypsum-bonded investments from surface casts.

## ADVANTAGES

1. Offers exceptional power in dissolving the dental plasters and the gypsum-bonded investments.
2. Produces bubbles during dissolving, so its working effectiveness can be visually monitored.
3. Never deteriorates resins and alloys.

## DIRECTIONS FOR USE

1. Pour ROCLEAN (50 mL - 100 mL) into a proper container.
2. Soak the objects in ROCLEAN after removing the bulk of plasters or investments from them.
3. Rinse the objects thoroughly in running water after plasters or investments are dissolved.

## NOTE

1. Replace with a new solution when the one in use does no longer emit bubbles.
2. For more efficient work, it is desirable to remove the bulk of plasters or investments as much as possible from the objects before soaking.
3. The use in ultrasonic makes the procedure faster.
4. Use ROCLEAN at room temperature and store it in a cool place.
5. Causes serious eye irritation. Wear protective gloves / eye protection. Wash thoroughly after handling. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention.
6. Precipitates may be observed at low temperature, however, they will not affect quality.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:  
<http://www.gceurope.com>  
They can also be obtained from your supplier.

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