Prior to use, carefully read the instructions for use.

GC TISSUE CONDITIONER

TISSUE CONDITIONING AND RELINING MATERIAL

For use by or under the supervision of a dental professional in the recommended indications.

This product is a Type B (Extra soft), Class 1 (medium flow) material according to classification of ISO 10139-1. Chemical nature of system is plasticized acrylics.

INDICATIONS FOR USE

- Soft relining material for short-term use, including functional impression taking using existing removable dentures.
- Tissue conditioning.
- Temporary relining.
- Functional impression taking.

CONTRAINDICATIONS

Avoid use of this product in patients with known allergies to methacrylate monomer, methacrylate polymer, ethanol or aliphatic acid ester-based plasticizer.

DIRECTIONS FOR USE

- PREPARATION**
 - Check the patient's oral conditions and the denture in use. Adjust mistfitting areas with a carbide bur, clean and dry. Create sufficient space so GC TISSUE CONDITIONER can be applied at a minimum thickness of 1 mm in order to maintain the denture. Apply COCOA BUTTER or petroleum jelly to the areas where the denture should not be relined.
- POWDER LIQUID DISPENSING**
 - The standard powder/liquid ratio is 2.4 g / 2 mL (1 unit of powder to 2 units of liquid). Dispense the required amounts of powder and liquid into the rubber cup. Note:
 - Using less liquid will result in a shorter gel phase and a more viscous mixture. More liquid will result in a longer gel phase and a less viscous mixture.
- MIXING**
 - Mix powder and liquid for 30-60 seconds until the mixture reaches a creamy consistency.
- APPLICATION**
 - The coating agent can be used as a bonding agent to improve the bond strength and to avoid possible delamination. Apply the coating agent with the brush only on the areas where a stronger bond is needed. In this case, GC TISSUE CONDITIONER cannot be peeled off by hand. C. Apply the mixture evenly to the fitting surfaces of the prepared denture. Note:
 - To avoid sticking of the material, lubricate fingertips or instruments with water. Do not use alcohol or petroleum jelly.
 - Do not allow the material to flow into the throat.
 - Seat the denture in the mouth. Ask the patient to gently occlude in the central occlusion and move the lips and mandibles to muscle firm. After keeping the denture for 5 minutes in this position, remove and inspect for excess and deficiencies.
 - Remove any excess using a sharp instrument.
- FINISHING**
 - Wash away saliva from the relined denture under running water. Dry with an air syringe.
 - Apply the coating agent to the relined surfaces of denture. Dry the coating agent thoroughly with an air syringe or leave it undisturbed for 10 minutes. Note:
 - To improve the marginal seal, coating agent can also be applied to the interface between the denture and denture interface.
 - Repeat the denture in the mouth. Note:
 - Intension of the denture in water before seating in the mouth will reduce unwanted odor.
 - Instruct the patient to clean the relined areas with fingers under running water. The use of a tooth brush may damage the relined areas. Care should be taken not to use denture cleaning agents for a short period of time.

- REMOVAL**
 - It is recommended to remove the material or replace it with a new one after a few days or when the denture is damaged. GC TISSUE CONDITIONER can be peeled off by hand. Use a sharp instrument or bur to remove the area where the coating agent was used as a bonding agent.
- STORAGE**
 - Recommended for optimal performance, store in a cool, dark place (4-25°C) (39-77.0°F) away from direct sunlight and high temperature.
- COLORS**
 - White
 - Pink
 - White

- PACKAGES**
 - 1-1 Package
 - Kit contents: Powder 90g (1), Liquid 90g (101mL), Coating agent 12g (15mL), Plastic liquid (1), Plastic liquid Refill cap (Large) (1), Plastic spatula (1), Plastic spatula (1).
 - 2 Package
 - Kit contents: Powder 90g (1), Liquid 90g (101mL), Coating agent 12g (15mL), Plastic liquid (1), Plastic spatula (1), Plastic spatula (1), Refill cap (Large) (1).

- CAUTION**
 - The liquid is flammable. Avoid sources of ignition. Do not keep or use near fire or other heat sources.
 - Use in a well ventilated place. Replace cap immediately.
 - Avoid contact with eyes. In case of contact, immediately flush with water and refer to a physician.
 - Avoid direct contact of liquid, mixture or coating agent on skin.
 - In case of contact, wipe with a cotton pellet or sponge soaked in alcohol and flush with water.
 - Coating agent may turn yellow with time, but this will not affect its function. It can be used with other products.
 - In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
 - Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present F.U. may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:

<http://www.gceurope.com>

They can also be obtained from your supplier.

CLEANING AND DISINFECTING: MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use respect device and label for decontamination. Do NOT IMMERS. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

Undesired effects: Reporting:

If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system by following the proper authority of your country accessible through the following link:

https://ec.europa.eu/growth/sectors/medical-devices/contacts_en

or by contacting the manufacturer directly. In this way you will contribute to improve the safety of this product.

Last revised: 08/2022

MANUFACTURED BY GDC DENTAL PRODUCTS CORP.

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