

# GC TISSUE CONDITIONER TECHNIQUE GUIDE



**1**  
Check the patient's mouth and denture. Trim off misfitting areas with a bur, clean and dry.



**2**  
**Optional technique:**  
Apply the coating agent on the areas where a stronger bond is needed.



**3**  
Dispense the powder (1 unit) and liquid (2 units) into the rubber cup. Mix for 30-60 sec. until the mixture becomes creamy.



**4**  
Apply the mixture to the prepared denture and seat in the mouth.



**5**  
Remove the denture from the mouth after about 5 min. Trim off excess with a sharp instrument. Rinse and dry.



**6**  
Coat the relined surface with the coating agent to increase the longevity. Dry the coating.



**7**  
**Optional technique:**  
To improve the marginal seal, coating agent can also be applied to the GC TISSUE CONDITIONER /denture interface.



**8**  
**Removal:**  
GC TISSUE CONDITIONER can be peeled off by hand. Use a sharp instrument or bur to remove the area where the coating agent was used as a bonding agent.