GC Tooth Mousse™

TOPICAL CREME WITH BIO-AVAILABLE CALCIUM AND PHOSPHATE

For use by or under the supervision of a dental professional in the recommended indications.

Description
GC Tooth Mousse is a water based creme containing RECALDENT™ (CPP-ACP: Casein Phosphopeptide – Amorphous Calcium Phosphate). When CPP-ACP is applied in the oral environment, it will bind to biofilms, plaque, bacteria, hydroxyapatite and soft tissue localising bio-available calcium and phosphate.

Important:
Saliva will enhance the effectiveness of CPP-ACP and the flavour will help stimulate saliva flow. The longer CPP-ACP and saliva are maintained in the mouth, the more effective the result.

A topical paste containing bio-available calcium and phosphate

1. Provides extra protection for teeth.
2. Helps neutralize acid challenges from acidogenic bacteria in plaque.
3. Helps neutralize acid challenges from other internal and external acid sources.

CONTRAINDICATIONS

RECALDENT™ (CPP-ACP) is derived from milk casein. Do not use this material on patients with a proven or suspected milk protein allergy and/or with a sensitivity or allergy to benzoate preservatives.

DIRECTIONS FOR USE

INDICATIONS

- a. Prior to and following in-office bleaching.
- b. After ultrasonic, hand scaling or root planing.
- c. Following professional tooth whitening (P.T.C.).
- d. After application of topical fluoride.
- e. To provide a topical coating for patients suffering from erosion, xerostomia or Sjögrens syndrome.
- f. During orthodontic treatment.
- g. For high risk caries patients.
- h. For special needs patients.

In-office application:

I. Custom tray application

1. Before use rinse the custom tray thoroughly under running water.
2. Extrude a generous layer of GC Tooth Mousse into the tray and apply to the upper and / or lower teeth.
3. Leave the tray undisturbed in the mouth for a minimum of 3 minutes.
4. Remove the tray.
5. Instruct the patient to use the tongue to spread the remaining GC Tooth Mousse throughout the mouth. Instruct the patient to retain for as long as possible (1- 2 minutes) avoiding excretion and delaying swallowing. The longer GC Tooth Mousse and saliva are maintained in the mouth, the more effective the result.
6. Ask the patient to expectorate and if possible avoid rinsing. Any GC Tooth Mousse remaining on the surface can be left to gradually dissipate. Advise the patient not to eat or drink for 30 minutes following application.
7. Any residual GC Tooth Mousse in the tray should be rinsed or brushed off under running water immediately after use.

II. Non tray application:

1. If necessary, remove any excess saliva on the tooth surface with a cotton roll or pellet. However, it is NOT necessary to dry the teeth with compressed air.
2. Apply a sufficient amount of GC Tooth Mousse to the tooth surfaces using an application swab, gloved finger or in difficult interproximal areas using an Interproximal Tooth Cleaning Brush.
3. Leave GC Tooth Mousse undisturbed for a minimum of 3 minutes.
4. Then instruct the patient to use the tongue to spread the remaining GC Tooth Mousse throughout the mouth. Request the patient to hold in the mouth for as long as possible (an additional 1 – 2 minutes) avoiding excretion and delaying swallowing. The longer GC Tooth Mousse and saliva are maintained in the mouth, the more effective it results.
5. Ask the patient to expectorate and if possible avoid rinsing. Any GC Tooth Mousse remaining on the surface can be left to gradually dissipate. Advise the patient not to eat or drink for 30 minutes following application.

At home application:

III. Day time application after tooth brushing as recommended by a dental professional

1. Apply a sufficient amount of GC Tooth Mousse to upper and lower teeth. A pea size amount for each arch is the minimum required. The material should be applied to the tooth surfaces using a clean dry finger or cotton tip. For difficult areas (between the teeth) use an Interproximal Tooth Cleaning Brush or dental floss coated with GC Tooth Mousse.
2. Leave GC Tooth Mousse on the teeth undisturbed for a minimum of 3 minutes.
3. Then use your tongue to spread the remaining GC Tooth Mousse throughout the mouth. Hold in the mouth for as long as possible (a further 1 - 2 minutes) avoiding excretion (spitting out) and delaying swallowing. The longer GC Tooth Mousse and saliva are maintained in the mouth, the more effective the result.
4. Expectorate thoroughly and if possible avoid rinsing. Any GC Tooth Mousse remaining on the surface can be left to gradually dissipate. Do not eat or drink for 30 minutes following application.

IV. Night time application after tooth brushing as recommended by a dental professional

1. Apply a sufficient amount of GC Tooth Mousse to upper and lower teeth. A pea size amount for each arch is the minimum required. The material should be applied to the tooth surfaces using a clean dry finger or cotton tip. For difficult areas (between the teeth) use an Interproximal Tooth Cleaning Brush or dental floss coated with GC Tooth Mousse.
2. Leave GC Tooth Mousse on the teeth undisturbed for a minimum of 3 minutes.
3. Then use your tongue to spread the remaining GC Tooth Mousse throughout the mouth. Hold in the mouth for as long as possible (a further 1 - 2 minutes) avoiding excretion (spitting out) and delaying swallowing. The longer GC Tooth Mousse and saliva are maintained in the mouth, the more effective the result.
4. Expectorate thoroughly and if possible avoid rinsing. Any GC Tooth Mousse remaining on the surface can be left to gradually dissipate overnight. Do not eat or drink for 30 minutes following application.

FLAVORS

Strawberry, Melon, Vanilla, Mint, Tutti-Frutti
STORAGE
Recommended for optimal performance, store in a cool dry low humidity area away from direct sunlight (8-25°C) (46.4-77.0°F).

PACKAGES

- Tube containing 40g (35mL)
1. Assorted Package
10 x 40g (2 each of Strawberry, Melon, Vanilla, Mint, Tutti-Frutti)
 2. Single Flavor Package
10 x 40g in each flavor Vanilla, Strawberry, Melon**, Tutti-Frutti**, Mint
**available in Europe only
 3. Refill Package
1 x 40g tube in each flavor

NOTE

GC Tooth Mousse with its unique CPP-ACP formulation was developed at The School of Dental Science, The University of Melbourne Victoria Australia.

*RECALDENT is a trademark used under licence. CPP-ACP technology has related patents or patents pending in Australia, NZ, Europe, Canada, and USA.

CAUTION

1. Always replace cap after use and remove any residual paste or moisture around the tube neck or inside the cap.
2. In case of contact with eyes, flush immediately with water and seek medical attention.
3. In case of spillage on clothes, wash off with water.
4. In any angioedema symptoms are experienced, this may indicate sensitivity or allergy to the benzoate preservatives, or to some other component of the product such as a flavoring agent. In this event, discontinue the use of the product and refer to a physician.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the Safety Data Sheets available at: <http://www.gceurope.com>

They can also be obtained from your supplier.

Last revised : 04/2018

IGC **CE** **R** Only
MANUFACTURED BY
GC AMERICA INC.
1000 West 172nd Street, Aliso Viejo, CA 92683 U.S.A.
TEL : +1-708-597-0900
DISTRIBUTED BY
GC AMERICA INC.
3737 West 127th Street, Aliso, IL 60803 U.S.A.
TEL : +1-708-597-0900
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

IGC **EU** **GC EUROPE N.V.**
Researchpark Hasselrode-Leuven 1240, Interleuvenlaan 33,
3001 Leuven, Belgium TEL: +32 16 74 10 00
disponibles en
GC ASIA DENTAL PTE. LTD
11 Tampines Concourse, #03-05 Singapore 528729
TEL: +65 6546 7560
GC AUSTRALASIA DENTAL PTY. LTD.
1753 Botany Rd, Banksmeadow, NSW 2019, Australia
TEL: +61 2 9301 8200
PRINTED IN THE USA

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