



## Why choose GC Tooth Mousse™ / MI Paste Plus™?

GC Tooth Mousse™ and MI Paste Plus™ are deliciously tasting crèmes that support healthy teeth and gums and help to maintain oral balance.

- ✓ They contain calcium and phosphate, the essential minerals that form teeth, delivered through a unique protein called RECALDENT™ (CPP-ACP). It helps to replenish these minerals lost during acid attacks from foods and beverages.
- ✓ They support the growth of beneficial, health-associated bacteria, thereby maintaining a healthy oral balance for teeth and gums.

GC Tooth Mousse™ is a great fluoride-free addition to your daily oral care routine.

GC MI Paste Plus™ contains additional fluoride (900 ppm) that works in synergy with RECALDENT™ to rebuild an even stronger enamel.

### It contains RECALDENT™ (CPP-ACP)

Nature has developed highly efficient protein systems in saliva, blood and milk that stabilise and transport calcium and phosphate, which are the fundamental building blocks of our teeth and bones. Extensive research over the past 25 years has refined the specific milk protein which transports these essential minerals to help protect our teeth – casein phosphopeptide amorphous calcium phosphate – otherwise known as RECALDENT™ (CPP-ACP).



## GC Tooth Mousse™ and MI Paste Plus™ come in a variety of delicious flavours.



Mint • Strawberry • Vanilla • Melon • Tutti Frutti

### GC Tooth Mousse™

Topical crème with calcium and phosphate  
Fluoride-free

### GC MI Paste Plus™

Topical crème with calcium, phosphate, and fluoride  
With 900 ppm fluoride

### Recommended application frequency of

- GC Tooth Mousse™  GC MI Paste Plus™:
- Once a day - at night after toothbrushing
  - Twice a day - morning and night after toothbrushing
  - \_\_\_\_\_



**ALWAYS READ THE LABEL AND FOLLOW THE DIRECTIONS FOR USE.**

RECALDENT™ and RECALDENT™ logo are trademarks of Mondelēz International group used under license.



GC Europe NV  
Interleuvenlaan 33  
B-3001 Leuven

T: +32 16 74 10 00  
E: info.gce@gc.dental  
www.gc.dental/europe

## GC Tooth Mousse™ & MI Paste Plus™

Topical crèmes for enamel strengthening and healthy oral balance



Brush.  
Floss.  
Crème.



# Protect and brighten your smile, every day!

Over time, teeth can lose essential minerals and weaken due to frequent exposure to acidic foods and drinks, whitening treatments, or the natural aging process.

GC Tooth Mousse™ and MI Paste Plus™ help to restore lost minerals, strengthen teeth, and maintain a healthy oral environment.




## Applications



Daily protection of oral health



Prevention of tooth decay

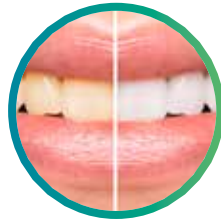

Before and after tooth whitening



Erosion, reflux and morning sickness




Tooth hypersensitivity



Children and adolescents



Dry mouth conditions and xerostomia

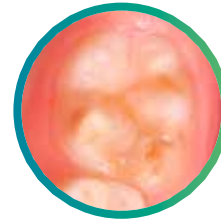

Dental developmental defects and chalky teeth



During and after orthodontics (braces and clear aligners)



After professional tooth cleaning



Chronic disease sufferers and multiple medication use



Aged care and special needs

## Directions for use

You should discuss with your dentist how often and for how long you should apply GC Tooth Mousse™ or MI Paste Plus™. Application should occur after brushing teeth. In most situations, morning and night application is advised.



1 Squeeze a pea-size amount of GC Tooth Mousse™ or MI Paste Plus™ onto a clean finger or cotton tip.



2 Apply to all teeth and use your tongue to spread around evenly.



3 Leave GC Tooth Mousse™ or MI Paste Plus™ on teeth for a minimum of 3 minutes avoiding spitting out and swallowing. The longer it stays in the mouth, the better.



4 Spit out excess. Any GC Tooth Mousse™ or MI Paste Plus™ remaining on the surface can be left to gradually dissipate. Avoid rinsing, and do not eat or drink for 30 minutes following application.



**Protect.  
Prevent.  
Remineralise.**