UNIVERSAL VPS ADHESIVE TRAY ADHESIVE

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

For adhering VPS impression materials or EXA'lence™ to impression trays.

RECOMMENDED PRODUCTS

EXAMIX™ NDS, EXAFAST™ NDS, EXAJET™, EXAFLEX™, EXACLEAR and EXA'lence™

*All packaging types may not be available in all countries.

CONTRAINDICATIONS

In rare cases, the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

- DIRECTIONS FOR USE**
1. Clean the inner surface of the tray and dry thoroughly.
 2. Apply an even layer of adhesive onto the tray using a brush or cotton pellet.
 3. Allow to stand for 5 minutes or air dry thoroughly with an air syringe.
 4. Load the tray with impression materials.

STORAGE

Recommended for optimal performance store in a cool and dry place (4°-25°C / 39.2°F-77.0°F).

PACKAGES

Universal VPS Adhesive 7mL bottle
Universal VPS Adhesive 12-pack (12-7mL bottles)

- CAUTION**
1. In case of contact with skin or oral tissue, immediately remove with a sponge or cotton soaked in alcohol. Flush with water.
 2. In case of contact with eyes, flush immediately with water and seek medical attention.
 3. The material is flammable and volatile. Do not use near naked flame. Keep away from sunlight or sources of ignition. Use in a well ventilated place.
 4. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

CLEANING AND DISINFECTING

MULTI-USE DELIVERY SYSTEMS: To avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.

DO NOT IMMERSE. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:

<http://www.gceurope.com>
or for The Americas
<http://www.gcamerica.com>

They can also be obtained from your supplier.

UNDESIED EFFECTS- REPORTING:

If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:

https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gc.dental

In this way you will contribute to improve the safety of this product.

Last revised: 04/2020

MANUFACTURED by
GC DENTAL PRODUCTS CORP.
2-285 Torimatsu-Cho, Kusagai, Aichi 486-0844, Japan

RESPONSIBLE MANUFACTURER IN CANADA:
GC AMERICA INC.
3737 West 127th Street, Alsip IL 60803 U.S.A.

DISTRIBUTED by
GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
TEL: +1-708-597-0900
www.gcamerica.com

EU: GC EUROPE N.V.
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33,
B-3001 Leuven, Belgium
TEL: +32-16-74-10-00

GC ASIA DENTAL PTE. LTD.
5 Tampines Central 1, #06-01 Tampines Plaza, Singapore 529541
TEL: +65-6546-7588

GC AUSTRALASIA DENTAL PTY. LTD.
1753 Botany Rd, Banksmeadow, NSW 2019, Australia
TEL: +61-2-9301-8200

GC SOUTH AMERICA
Rua Heliodora, 399, Santana - São Paulo, SP - Brasil
CEP: 02022-051 - TEL: +55-11-2925-0965 CNPJ: 08.279.999/0001-61
RESP. TÉCN. Mayara De Santis Ribeiro - CRO/SP: 105.982

UNIVERSAL VPS ADHESIVE TRAY ADHESIVE

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