UNIFAST™ LC

LIGHT CURED ACRYLIC RESIN


For use only by a dental professional in the recommended indications.

- INDICATIONS FOR USE
Direct technique using a hand held visible light curing device (470 nm wavelength).
Indirect technique using a table top curing unit (470 nm wavelength).
By pressure and/or brush on techniques.

- CONTRAINDICATIONS
Avoid use of this product in patients with known allergies to methacrylate monomer or methacrylate polymer.

- DIRECTIONS FOR USE
a) The standard powder/liquid ratio is 1.0 g/0.5 mL (first graduation on the powder measure/21 drops of liquid).

- b) Add powder to liquid in the mixing cup and mix quickly for 10-15 seconds.
The following illustrates the lapsed time stages in polymerisation: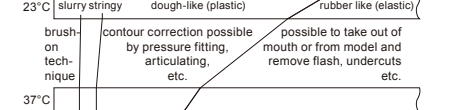

- c) Use the slurry or dough according to application standard techniques.
When all necessary corrections or adjustments have been made, light cure the resin using an appropriate visible light curing device (470 nm wavelength).

Table showing curing depth (mm) for different resin types (Translucent, Ivory, Pink # 3/E, Live Pink # 8) and curing methods (Hand held, Table top).

- The above are typical values and are given for guidance only.
Light cure within 1 hour from starting the mix.
UNIFAST™ LC can also be cured chemically in 5 minutes in water at 60°C (140°F).

STORAGE
Recommended for optimal performance, store in a cool dark place away from direct sunlight (4-25°C) (39.2-77.0°F).

PACKAGES
1. Bottle of 50 g powder.
2. Box of 30 g liquid (2 x 15 g or 14.3 mL).
3. 1-2 packager*: 1 x 50 g Pulver, 2 x 15 g (14.3 mL) liquid, accessories.

SHADES
1. Crown colour Ivory*, A2, A3, B2, B3, C2
2. Gingival colour* Pink # 3, Live Pink # 8 (veined), Pink E
3. Translucent Translucent
* Not available in Europe.

CAUTION
1. The liquid is volatile and highly flammable. Close the bottle immediately after use and keep away from sources of ignition.
2. In case of contact with eyes, flush immediately with water and seek medical attention.


Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: http://www.gceurope.com

CLEANING AND DISINFECTING
MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration.

For the Summary of Safety and Clinical Performance (SSCP) please see EUDAMED database (https://ec.europa.eu/tools/eudamed) or contact us at Regulatory.gce@gc.dental

Undesired effects - Reporting:
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:

UK Responsible Person
GC UNITED KINGDOM Ltd.
Coopers Court Newport Pagnell, Buckinghamshire, MK16 8JS, United Kingdom
Last revised: 08/2022

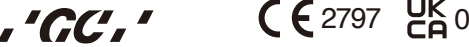MANUFACTURED BY
GC DENTAL PRODUCTS CORP.
2-955 Tomihata-cho, Kagawa-ku, Itatahiki-ku, Tokyo 114-8585, Japan
RESPONSIBLE MANUFACTURER IN CANADA
GC AMERICA INC.
3737 West 127th Street, Aliso, IL, 60803 U.S.A.

DISTRIBUTED BY
GC CORPORATION
76-1 Hiei-yama-cho, Itabashi-ku, Tokyo 174-8585, Japan
EU : GC EUROPE N.V.
Researchpark Haardrode-Leuven 1240, Interleuvenaan 33, B-3001 Leuven, Belgium TEL : +32 16 74 10 40

Prior to use, carefully read the instructions for use. EN

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Table showing curing depth (mm) for different resin types (Translucent, Ivory, Pink # 3/E, Live Pink # 8) and curing methods (Composietlamp, Tafelapparaat).

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Undesired effects - reporting:
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: https://ec.europa.eu/growth/sectors/medical-devices/contacts_en

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