
ACRYLIC RESIN FOR TEMPORARY RESTORATION
For use only by a dental professional in the recommended indications.
UNIFAST Trad is a self curing, fast setting acrylic resin for use in a variety of dental applications. For use in either pressure or paint-on techniques.

RECOMMENDED INDICATIONS
For repair of dentures, construction of temporary crowns, impression trays for individual teeth and orthodontic plates, splints, and check bite.
CONTRAINDICATIONS
Avoid use of this product in patients with known allergies to methacrylate monomer or methacrylate polymer.

DIRECTIONS FOR USE
A. PRESSURE TECHNIQUE
The recommended powder/liquid ratio is 1g of powder (1st graduation of powder measure) to 0.5mL (1st graduation of liquid measure) of liquid.

- Dispense the required amount of liquid into the rubber cup as follows:
 - Remove the protective cap and screw on the dispensing nozzle (fig. 1).
 - Pour into the measure provided (fig. 2) until the bottom of the meniscus is level with the required graduation (fig. 3).
 - Replace the protective cap to prevent evaporation.
- Add the powder and mix thoroughly for 10-15 seconds.
- The mixture reaches a dough-state 20-30 seconds after mixing. All manipulation should be finished before 2 minutes after mixing when setting starts.

B. PAINT-ON TECHNIQUE
Use standard techniques.

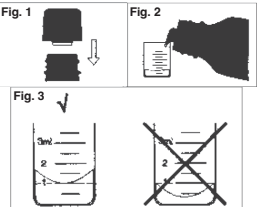STORAGE
Recommended for optimal performance, store in a cool place away from direct sunlight (2-37°C/36-98°F).

NOTES
1. Do not use UNIFAST Trad powder or liquid with components of other brands of acrylic resin.
2. Remove residual resin with a dry gauze after each application.
3. Mop up liquid spills with a dry cloth.

PACKAGES
1. Bottles of powder: 100g; 250g; 1kg
2. Bottles of liquid: 100g (104mL); 250g (260mL)

SHADES
1. Crown color: Ivory
2. Gingival colors: Pink (#3); Live Pink (#8); Pink X
NOTE: Not all shades available in all countries.

CAUTION
1. The liquid is HIGHLY FLAMMABLE. Avoid sources of ignition.
2. In case of contact with eyes, flush immediately with water and seek medical attention.
3. Personal Protective Equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

CLEANING AND DISINFECTING RECOMMENDATION
MULTI-USE DELIVERY SYSTEMS: To avoid cross-contamination between patients the bottles and measuring devices require mid-level disinfection. Immediately after use inspect the bottles, measuring devices and label for deterioration. Discard if damaged. DO NOT IMMERGE. Thoroughly clean bottles and measuring devices to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines. Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the Safety Data Sheets available at: <https://www.gc.dental/europe> or for the Americas <https://www.gc.dental/america>. They can also be obtained from your supplier.

Undesired effects - Reporting:
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gc.dental In this way you will contribute to improve the safety of this product.

UK Responsible Person
GC UNITED KINGDOM Ltd.
Coopers Court Newport Pagnell, Buckinghamshire, MK16 8JS, United Kingdom

Last revised: 06/2024

**UK
CE 0086**

2797

MANUFACTURED BY
GC AMERICA INC.
3737 West 120th Street, Alsip, IL 60803 U.S.A.
TEL: +1-708-597-0900
www.gc.dental/america

DISTRIBUTED BY
GC CORPORATION
70-1 Hasunuma-cho, Itabashi-ku, Tokyo, Japan

Rx Only

RESPONSIBLE MANUFACTURER IN CANADA
GC AMERICA INC.
3737 West 120th Street, Alsip, IL 60803 U.S.A.
TEL: +1-708-597-0900
www.gc.dental/america

GC SOUTH AMERICA
Rua Heliodora, 399, Santana
São Paulo, SP, BRASIL
CEP: 02022-051 - TEL: +55-11-2925-0965
CNPJ: 08.279.999/0001-61
RESP. TEL: Exib. de Lata - CRD/SP/100.866

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