Use only by use carefully read the instructions for use.

UniFil® Bond

LIGHT-CURED BONDING SYSTEM

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

Bonding of light cured composites to tooth structure.

CONTRAINDICATIONS

- Pulp capping.
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

DIRECTIONS FOR USE

1. CAVITY PREPARATION
Prepare tooth in usual manner. Dry the prepared tooth surfaces by gently blowing with an air syringe. Note:

For pulp capping, use calcium hydroxide.

2. APPLICATION OF UniFil® Bond SELF-ETCHING PRIMER

Dispense 1 or 2 drops of UniFil® Bond SELF-ETCHING PRIMER into the dispensing dish provided (Fig. 1). Apply the primer to the enamel and dentin surfaces on the prepared tooth using a pink disposable micro-tip applicator provided (Fig. 2), leave undisturbed for 20 seconds. Note:

- Prior to use, leave the primer bottle at room temperature for 5 minutes after removing from refrigerator.
- After dispensing, use as quickly as possible.
- Do not allow the primer to contact oral tissues. In case of contact, immediately remove with a sponge or cotton. Rinse thoroughly with water after placing composite.
- Rinse the bottle immediately after use.

3. DRYING THE SELF-ETCHING PRIMER

20 seconds after application, gently dry with an air syringe for 5 seconds (Fig. 3) and ensure that the primed surface has a glossy appearance. Do not rinse away.

Should the applied Primer be rinsed away, dry the cavity and re-apply primer.

4. APPLICATION OF UniFil® Bond BONDING AGENT
Dispense 1 or 2 drops of UniFil® Bond BONDING AGENT into the dispensing dish (Fig. 4). Immediately apply to the primed enamel and dentin surfaces using a yellow colored micro-tip applicator (Fig. 5). Blow gently with an air syringe to form a thin film (Fig. 6).

- Prior to use, leave the bottle of BONDING AGENT at room temperature for 5 minutes after removing from refrigerator.
 - Take care not to mix with SELF-ETCHING PRIMER.
 - After dispensing, protect from light and use as quickly as possible.
 - Replace bottle cap immediately after use.
 - Prior to blowing gentle air, make sure that no water or oil is injected from the air syringe as this can adversely affect bonding effectiveness.
 - Be sure to use UniFil Bond Self-etching Primer with Bonding Agent. Do not use other primers.
- 5. LIGHT CURING OF BONDING AGENT**
Light cure for 10 seconds using a visible light curing unit (470nm wavelength)* (Fig. 7). In case of deep cavities, light cure for a minimum of 20 seconds. *Irradiation time by light curing device

Halogen	10 sec.
Plasma arc	3 sec.

Note:
When light curing material, wear eye protection glasses.

6. PLACEMENT OF LIGHT-CURED COMPOSITE
For placement of light-cured composite, follow manufacturer's instructions for use.

STORAGE

Store in a refrigerator (2 - 8°C / 35.6 - 46.4°F) when not in use.
(Shelf life: 2 years from date of manufacture)

PACKAGES

- Kit:
 - Self-Etching Primer 6mL
 - Bonding Agent 6mL
 - 50 micro-tip applicators (25 each pink / yellow), applicator holder, dispensing dish (No. 1) and protective cover (No. 1)
- Replacement packages
 - Self-Etching Primer 6mL
 - Bonding Agent 6mL

Option:
Box of Micro-tip applicator (250 each pink / yellow) with applicator holder (4)
Other items are available separately.

CAUTION

- UniFil® Bond SELF-ETCHING PRIMER is flammable. Do not use near naked flame. Keep away from sources of ignition. Keep away from sunlight. Use in a well ventilated place.
- In case of contact of UniFil® Bond SELF-ETCHING PRIMER and BONDING AGENT with oral tissues or

- skin, immediately remove with a sponge or cotton. Flush with water. Note that the primer affected part may turn white. In this situation, instruct the patient to leave undisturbed until the mark disappears in 2-3 days.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- Take special care that the patient should not swallow the primer or bonding agent.
- Do not mix with other similar products.
- Do not use with chemically cured composite resins

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