SURGICAL INSTRUMENT

Cytrans™ Bone Dish

Cytrans™ Bone Spoon

For use only by a dental professional in the indications for use.

Cytrans™ Bone Dish is a stainless steel mixing cup for combining bone graft material with saline solution or blood. Cytrans™ Bone Spoon is a stainless steel instrument for compacting bone graft material in bone graft site.

INDICATIONS FOR USE

Cytrans™ Bone Dish: For mixing bone graft material with saline or blood.

Cytrans™ Bone Spoon: For compacting bone graft material in bone graft site.

PRECAUTIONS

I. Precautions for use

Do not use this product in patients with a known history of hypersensitivity—such as rash or dermatitis—to this product or to alloys or ingredients of similar composition.

II. Additional basic precautions

- Prior to use, ensure that the product is free from damage, burrs, or other defects.
- Discontinue use if the product is dropped or otherwise damaged or deformed.
- Do not apply excessive force, as it may cause breakage or deformation.
- Use enzyme-based detergents for cleaning. Do not use the following chemical agents for sterilization:
 - Peracetic acid-based solutions
 - Glutaraldehyde-based solutions
 - Phthalaldehyde-based solutions
- As the product is non-sterile at the time of purchase, it must be cleaned and sterilized before use, and the same procedure must be followed for each subsequent use.
- Always take precautions to avoid contamination after sterilization. Do not touch the product directly with your hands or with any non-sterile instruments.
- Regardless of whether the product was used during surgery, perform the prescribed disinfection, cleaning, and sterilization procedures afterward.
- Do not modify or alter the product.
- Do not use for purposes other than those stated in [Indication for use].
- This product must only be used by licensed dental professionals.

DIRECTIONS FOR USE

- As the product is non-sterile at the time of purchase, it must be sterilized prior to use.
Cleaning and Sterilization:
After cleaning and drying, sterilize using an autoclave under one of the following conditions:
 - 115–118°C for 30 minutes
 - 121–124°C for 15 minutes
 - 126–132°C for 10 minutes
 - 134°C for 3 minutes
If the drying process exceeds 135°C, the drying step should be omitted.
- Place the bone graft material into the mortar-shaped concavity of Cytrans™ Bone Dish and mix it with the patient's blood or sterile saline.
- Use Cytrans™ Bone Spoon to fill the bone defect with the bone graft material.

Precautions for maintenance

- After use, immerse the instrument in saline solution and perform preliminary rinsing under running water immediately after surgery. Use a brush to thoroughly remove any residues, especially in grooves.
- Avoid prolonged immersion in saline solution, as it may cause rust or discoloration. Clean the instrument promptly after surgery.
- Do not overload the sterilization chamber when autoclaving; sterilize an appropriate number of instruments at a time.
- Do not sterilize rusted instruments together with other instruments, as they may cause contamination.
- To prevent rust and discoloration, observe the following precautions:
 - Do not allow blood or bone residues to dry and harden on the instrument.
 - Use purified water for final rinsing.
 - Do not sterilize instruments with residues, moisture, or cleaning agents.
 - When autoclaving, use purified water instead of tap water.
- After cleaning, immediately wipe off moisture to prevent rust formation before storage.
- When using a cleaning device, ensure that instruments do not come into contact with each other to avoid damage.
- During cleaning, store instruments in a cassette or similar container to prevent scratches.
- Water marks may appear on the surface after cleaning, disinfection, and sterilization; however, this is a characteristic of stainless steel and does not affect product quality.

STORAGE AND HANDLING

I. Storage

Store at room temperature, avoiding direct sunlight, high temperatures, and high humidity.

II. Precautions for storage

Ensure that the product is stored and managed in a manner that prevents access by individuals other than dental professionals.

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GC ASIA DENTAL PTE. LTD.
GC AUSTRALASIA DENTAL PTY. LTD.
NIMI CHEMICAL INDUSTRIAL CO., LTD.