Prior to use, carefully read the instructions for use.

EN

CAVITON

HYDRAULIC TEMPORARY RESTORATIVE

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

As a temporary filling material.

CONTRAINDICATIONS

In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

DIRECTIONS FOR USE

- Dry the prepared cavity with a small cotton pellet.
- Transfer the material to the cavity using a suitable instrument and form the contour. In the case of occlusal Class I cavities let the patient bite down on the surface, and then form the contour.
- Ask the patient to close his/her mouth. The filling will harden in about 30 minutes by absorbing moisture. Recommend the patient to avoid pressure or abrasion for at least 1 hour.
Notes
 - Do not mix or contaminate with any other material.
 - When removing the filling, clean the cavity with an alcohol soaked cotton pellet.
 - Close the jar immediately.

STORAGE

Store at temperature of 4-25°C (39.2-77.0°F).

(Shelf life: 2 years from date of manufacture)

PACKAGES

Jar 30 g, white or pink.

Last revised : 04/2009

MANUFACTURED by
GC ASAHl CORP.
1-3-3 Kachigawa-cho, Kasugai, Aichi 486-0945, Japan

DISTRIBUTED by
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

 EU: GC EUROPE N.V.
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33,
B-3001 Leuven, Belgium TEL: +32 16 74 10 00

GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
TEL: +1-708-597-0900

GC ASIA DENTAL PTE. LTD.
19 Loyang Way, #06-27 Singapore 508724
TEL: +65 6546 7588

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