**COE**<sup>®</sup>

GC AMERICA INC.  
ALSIP, IL 60803 U.S.A.

**COE**<sup>®</sup>

MANUFACTURED by  
GC AMERICA INC.  
3737 West 127th Street  
Alsip, Illinois 60803 U.S.A.

Tel: +1-708-597-0900

DISTRIBUTED by  
GC EUROPE N.V.  
Interveerlaan 13  
B-3001 Leuven, Belgium  
Tel: +32-16-39-80-50

7831005

9800 

HARD CHAIRSIDE DENTURE RELINE  
HARTES PROTHESEN-UNTERFÜTTERUNGSMATERIAL ZUR  
ANWENDUNG DIREKT IM MUND  
MATERIAU DE REBASAGE DUR AU FAUTEUIL  
RIBASANTE DURO PER DENTIERE DA STUDIO  
MATERIAL PARA REBASE DURO DE APLICACION EN CLINICA  
DIRECTE HARDE VOERING VOOR PROTHESEN  
HÄRDTBLIVENDE DIREKTE REBASERINGSMATERIALE  
HÄRT REBASERINGSMATERIAL FÖR PROTESER, "CHAIRSIDE"

**COOLINER**<sup>TM</sup>

**COOLINER**<sup>TM</sup>

#### **HARD CHAIRSIDE DENTURE RELINE**

Prior to use, carefully read the instructions. For use only by a dental professional in the recommended indications.

#### **CONTRAINDICATIONS:**

Patients who have shown sensitivity to methacrylates. In case of allergy refer to a physician.


#### **RECOMMENDED INDICATIONS:**

A temporary lining for acrylic dentures. For use in chairside procedures.

#### **DIRECTIONS FOR USE:**

- 1. Preparation of the denture:** Relieve and roughen the area of the denture to be relined. Clean and dry the denture thoroughly. Coat labial and buccal surfaces of the denture with COE LUBRICANT. Do not apply coating within 3mm (1/8 inch) of the peripheral border. If the denture has plastic teeth also protect them with COE LUBRICANT.
- 2. Preparation of KOOLINER:** Recommended powder/liquid ratio is 15ml powder to 6ml liquid. Pour liquid into the mixing cup and then add the powder slowly. Stir thoroughly for no more than 30 seconds and avoid the introduction of air bubbles.

ADD POWDER  
TO LIQUID  
AND STIR FOR  
30 SECONDS

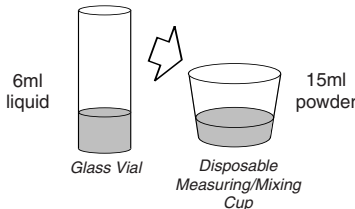- 3. Application.** After approximately 1-2 minutes spread the mixture of KOOLINER over the area to be relined. Seat the denture in the manner of taking an impression and instruct the patient to close lightly into occlusion. After 3 minutes instruct the patient to move lips and cheeks so that a muscle trimmed periphery is obtained. Remove the denture and rinse under cold water. Trim away excess material. Re-seat the denture and instruct the patient to close FIRMLY into occlusion, and to hold this position for 5 minutes. Remove the denture and rinse again in cold water.
- 4. Finishing.** Peak curing temperature: Approximately 43°C/110°F at 7 minutes when tested according to ADA/ANSI specification number 17. In thicker applications, peak temperature may exceed that stated above, possibly producing a hazardous condition in the mouth during curing. When curing is complete (10 minutes), trim away excess. For smoothing the edges use a hot spatula.

#### **HARD CHAIRSIDE DENTURE RELINE STORAGE:**

Store at temperature of 4-25°C (39 - 77°F). (3 year shelf life guarantee).

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