

EXPERIENCE ceramic

Bonding process

- All tooth preparations, bracket placement, and bracket positioning should be conducted in a manner consistent with standard practice protocols.
- All adhesive preparations, applications, and polymerization should be consistent with the manufacturer's directions being used.
- Suggested practice beyond those is:
 1. Liberally apply adhesive to the entire surface of the bracket base.
 2. Press adhesive material into the base to remove pockets of air and ensure uniform coverage.
 3. Remove all flash surrounding the bracket after placement.


Debonding Process




- 1**
Open the clips and
remove the wire




- 2**
Remove excess
adhesive if not done
at fitting stage

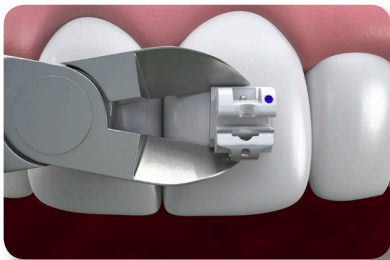

- 3**
Use an appropriatez
tool beneath the
bracket base either
gingival occlusal

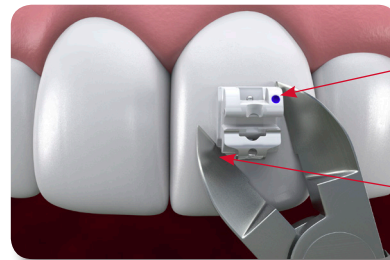

- 3**
OR at opposing
gingival occlusal
corners

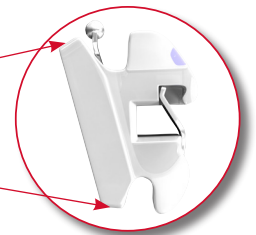

- 4**
Clean up teeth of
remaining adhesive