



**EN**

# Instructions for Use Ortho Clip Opener 50-1032-0000

## 1. CONFIGURATION AND STRUCTURE

Blade: Stainless steel (Fe, Cr, Mn, Ni)

Holder: Polypropylene




## 2. PERFORMANCE, INTENDED PURPOSE, EFFECT OR EFFICACY

In orthodontic treatment, this product is used for opening the clip of Self ligating bracket.


## 3. OPERATING INSTRUCTIONS OR HOW TO USE

1. Insert the short tip into the center notch at the middle of clip. (Fig. 1)

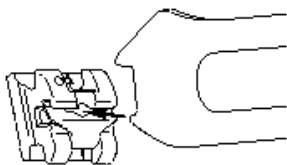


2. Slide the tip of the tool to the occlusal. (Fig. 2) Then the long tip comes in contact with gingival wall of the bracket. This will prevent the clip from opening excessively. (Fig.3)


(Fig. 2)

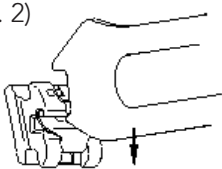

(Fig. 3)

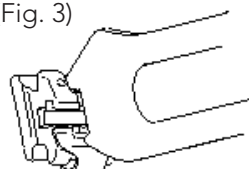

3. When closing the clip, push the curved part with either a finger or the inner side of long tip. (Fig. 4)

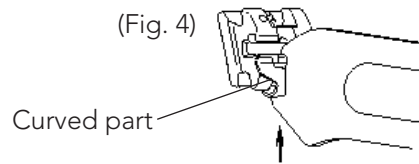

## 4. PRECAUTIONS FOR USE

Important basic cautions


1. Do not use the product for other than the orthodontic purposes.
2. Professional use only.
3. If deformation or deterioration is recognized, do not use. Contact a sales representative.
4. Stainless steel components of instrument contain nickel which may cause allergic reactions.
5. For sterilizing, use an autoclave ( $\leq 121^{\circ}\text{C}$ ) or ethylene oxide gas sterilizer after cleaning as with other common orthodontic instruments. Dry-heat sterilization is not suitable for resin holder.
6. When the clip and the body are bonded with adhesive, remove adhesive material from them before opening the clip.

## 5. STORAGE

1. Keep away from non-professionals.

## 6. PACKAGE

- 1 piece per package

 **GC Orthodontics Europe GmbH**  
Harkortstraße 2  
58339 Breckerfeld  
Germany



Distributed by:  
**GC Orthodontics Europe GmbH**  
Harkortstraße 2  
58339 Breckerfeld  
Germany

**GC Orthodontics America Inc.**  
Alsip, IL 60803  
USA