



EN


Instructions for Use GC ORTHO CHAIN

CONTRAINDICATION

1. Single use only. Do not reuse.
2. Avoid use for patients with an identified allergy to polyurethane, such as skin sensitization effect.

1. CONFIGURATION AND STRUCTURE

Component: Polyurethane

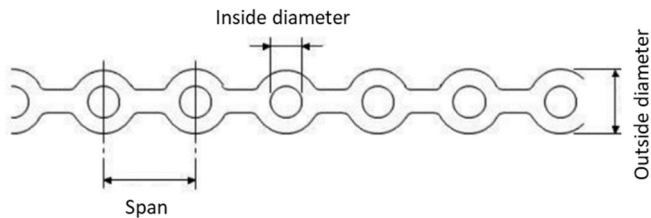

Dimensions

		Thickness	Outside diameter	Inside diameter	Span
SHORT	Clear	0.5mm	2.8mm	1.4mm	3.0mm
	Grey				
MEDIUM	Clear	0.5mm	2.8mm	1.4mm	3.6mm
	Grey				
LONG	Clear	0.5mm	2.8mm	1.4mm	4.0mm
	Grey				

2. PERFORMANCE, INTENDED PURPOSE, EFFECT OR EFFICACY

This product is elastic material used for movement of teeth and retention of orthodontic appliances in orthodontic treatment.

3. OPERATING INSTRUCTIONS OR HOW TO USE

Cut into an appropriate length, tying to the tie wing of bracket or hooking on the hook, the traction force is applied.

4. PRECAUTIONS CONCERNING THE USAGE

1. In order to avoid the risk of infection, use as cutting into an appropriate length beforehand.
2. For traction, use with elongation of 2 -2.5 times the original for the standard, and do not apply too strong traction force to the teeth.

5. PRECAUTIONS FOR USE

Important basic cautions

1. Do not use the product for other than the orthodontic purposes.
2. Professional use only.
3. If deformation or deterioration is recognized, do not use. Contact a sales representative.
4. If an allergic reaction occurs, discontinue use of the product and direct patient to consult physician.

6. Defect and adverse event

Used product is likely to cause deformation, deterioration or reduction of strength and induce infection.

7. STORAGE

[Storage method]

1. Keep away from non-professionals.
2. Store in a dark place, away from the direct sunshine, UV rays, heat and moisture.

8. [Shelf life/expiration date for use]

Please use by expiration date listed on the label.

*Expiration date uses the format YYYY-MM

Manufactured by:

Nimi Chemical Industries Co.,LTD
4-13 Yokoyama-Cho, Kiryu-Shi, Gunma-ken,
376-0055, Japan

Manufactured for:

GC Orthodontics America Inc.
3737 West 127th Street, Alsip, IL 60803
USA