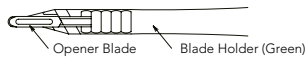
## Component

### Step by step Instructions

#### Clip Opening

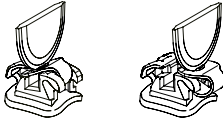
(1) Place the round blade tip of opening tool onto the top surface of the clip parallel to archwire slot, until it gets over a small projection on the clip and clicks into the center of the clip.

Click The tool is stable.

(Position of the beginning to open clip.)

(2) Once the blade is properly positioned, pull the blade tip toward occlusal direction to open.

#### Clip Closing

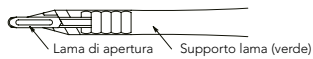
The EXPERIENCE® mini clip closes with pivotally-slide motion. Therefore, pushing straight the clip may cause clip deformation.

When closing the clip, lift the clip up slightly to labial direction and then, push the clip with fingertip.

Alternatively, position blade of opener at the occlusal base and swivel instrument gingivally to close.

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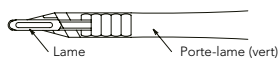
## EM-Tool (Plastic): 50-2130-0000

### Direction for use:

Caution: U.S Federal Law prohibits the sale of this device outside the dental or dental specialty field.

### Indications for use:

The EXPERIENCE® Opening Instrument is indicated for opening the self-ligating clip of the EXPERIENCE® orthodontic bracket

### Indications for sterilization:

Dry-heat sterilization cannot be used for the plastic holder.

### Warnings

Instrument is not sterile, use an autoclave or ethylene oxide gas sterilizer after cleaning this product for sterilizing. Do not exceed a sterilization temperature of 121° C. Stainless steel components of instrument contain nickel which may cause allergic reactions.

### Precautions:

Remove excess adhesive prior to opening the clip. Pushing the clip straight to close rather than using a pivotal motion may cause clip deformation.

### Adverse Reactions:

None known

### Warranty and limited recourse:

1. GC Orthodontics warrants that this product is free from manufactures defects.

2. In the case of a claimed defect, GC orthodontics obligation shall be limited to replacement of merchandise or a refund at the purchase price.

### Limitation of liability:

Except where prohibited by law, GC Orthodontics will not be liable for any loss or damages arising from this product directly or indirectly. This limitation does not apply to third party personal injury claims.

**NICKEL WARNING:** This product contains nickel, a chemical known to the State of California to cause cancer.

Distributed by  
**GC Orthodontics America Inc.**  
Alsip, IL 60803  
USA

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**GC Orthodontics Europe GmbH**