Prior to use, carefully read the instructions for use. **EN**

GC Ortho Connect™

LIGHT-CURED ORTHODONTIC ADHESIVE

Use only by a dental professional in the indications for use.

INDICATIONS FOR USE

- Bonding metal brackets and attachments.
- Bonding ceramic brackets.
- Bonding plastic brackets and acrylic appliances.

CONTRAINDICATIONS

- Avoid use of this product in patients with known allergies to methacrylate monomer or methacrylate polymer.
- Cementation of stainless steel crown-retained appliances (e.g. Herbst Appliances).

COMPOSITION

Dimethacrylate, silicon dioxide, barium glass, phosphoric acid ester monomer, initiator, pigment, stabilizer

DIRECTIONS FOR USE

A. TOOTH PREPARATION

- Clean tooth surfaces to be bonded with pumice and water. Rinse thoroughly with water. Dry by gently blowing with an oil-free air.
- Ethch the tooth surface with GC Ortho Etching Gel 37% phosphoric acid for 30 seconds. Rinse thoroughly with water. Refer to the respective instructions for use.
- Remove excess moisture by blowing with an oil-free air. Prepared surfaces should be dry.

NOTE:
If the surface to be bonded contains porcelain, composite, zirconia or metal, it should be treated accordingly. Porcelain and composite should be sandblasted and pre-treated with silane. Zirconia and metal should be sandblasted. Refer to manufacturer's instructions.

B. BRACKET PREPARATION
Make sure that the bracket is pretreated and handled according to the manufacturer's instructions.

C. SYRINGE PREPARATION

- Hold the syringe upright and remove the cap by turning counterclockwise.
- Promptly and securely attach the dispensing tip to the syringe by turning clockwise.
- Place the light protective cover until use.

NOTE:
Take care not to attach the dispensing tip too lightly. This may damage the screw. In order to ensure a tight connection make sure that the threads are free of remnant material.

D. BONDING PROCEDURE

Brackets

- Remove cover from the dispensing tip. **NOTE:** Prior to syringe material, make sure that the dispensing tip is securely attached to the syringe. Any possible air from the dispensing tip is removed by gently pushing forward the syringe plunger with the tip pointing upwards until material reaches the opening of the tip.
- Slowly push the syringe material. If plugged, remove the dispensing tip and extrude a small amount of material directly from the syringe.
- Coat the bonding surface of the bracket completely with the paste.
- Position the coated bracket on the tooth.
- Press the bracket firmly against the enamel surface. Using an explorer or a scaler, remove the excess adhesive at this time. Press the bracket again. Continue placing additional brackets. Place all brackets in a quadrant or in the full arch. **NOTE:**
 - After removing excess adhesive, be careful not to disturb the brackets before the adhesive is cured. Movement of the brackets at this point might decrease the bond strength.
 - Minimize exposure to ambient light. Ambient light can shorten the main curing time.
 - Light cure each bracket for 10 seconds each from the mesial and distal aspects with an LED light curing device (>500 mW/cm², 460-480 nm).
 - NOTE:
 - It is very important that the curing light be tested periodically with a light meter to ensure adequate light output.
 - Keep light guide as close as possible to the bonding and brackets.

Acrylic Appliances

- Using a microetcher or a rotary instrument, roughen and place retention holes in the internal surfaces of the appliance.
- Prepare the teeth as mentioned in section #1.
- To facilitate removal of the appliance, place petroleum jelly on the occlusal surface of the appliance.
- Fill the internal surfaces of the appliance with GC Ortho Connect.
- Seal into proper position.
- Immediately remove excessive adhesive from the periphery of the appliance.
- Light-cure on the buccal, palatal and occlusal surfaces for 10 seconds each with an LED light curing device (>500 mW/cm², 460-480 nm).
- Appliance activation can begin at the patient's bedtime the same evening.




E. DEBONDING PROCEDURE

Brackets

Grasp bracket with a ligature cutter and, while supporting the tooth with fingers from the lingual side, give a sharp clockwise rotation to the bracket. If resistance is met, try again twisting in the opposite direction. Remove any residual adhesive with a rotary instrument.

Acrylic Appliances

To remove bonded appliances, flex the appliance in multiple sites with band removing pliers. If this does not remove the appliance, section the appliance and remove.

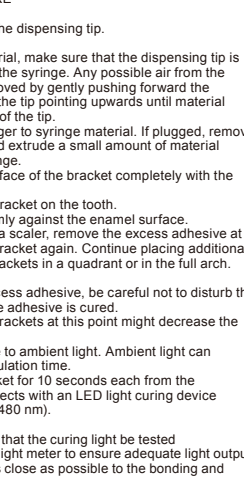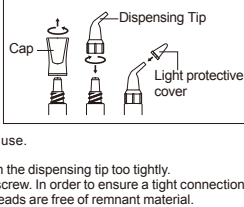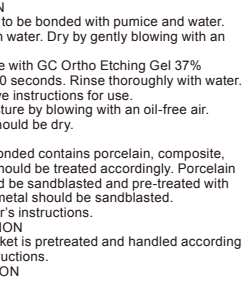SHADE

Clear

STORAGE

Recommended for optimal performance, store at room temperature (4-25°C / 39.2-77.0°F) away from direct sunlight.

PACKAGE

2 × 2.9 (0.1 mL) syringe,
4 x dispensing tip for GC Ortho Connect,
2 x light protective cover

CAUTION

- In case of contact with oral tissue or skin, remove immediately with cotton or a sponge soaked in alcohol. Flush with water. To avoid contact, a rubber dam and/or COCOA BUTTER can be used to isolate the operation field from the skin or oral tissue.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- Take care to avoid ingestion of the material.
- In case of contact with unintended areas of tooth or prosthetic appliances, remove with instrument, sponge or cotton pellet before light curing.
- Do not use GC Ortho Connect in combination with eugenol containing materials as eugenol may hinder GC Ortho Connect from setting or bonding properly.
- Do not mix with other similar products.
- The dispensing tips cannot be sterilized in an autoclave or chemically.
- Do not wipe the dispensing tip with excessive alcohol as this may cause contamination of the material.
- Wear plastic or rubber gloves during operation to avoid direct contact with air inhibited resin layers in order to prevent possible sensitivity.
- Wear protective eye glasses during light curing.
- When polishing the polymerized material, use a dust collector and wear a dust mask to avoid inhalation of cutting dust.
- Avoid getting material on clothing.
- In respect of all uses according to local regulations.
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:
www.gc.dental/ortho

They can also be obtained from your supplier.

CLEANING AND DISINFECTING

MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use respect device and label for deterioration. Discard device if damaged.

DO NOT IMMERSE. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional / national guidelines.

For the Summary of Safety and Clinical Performance (SSCP) please see EUDAMED database (<https://ec.europa.eu/tools/eudamed>) or contact us at Regulatory.gce@gc.dental

Undesired effects - Reporting:
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:
https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gc.dental. In this way you will contribute to improve the safety of this product.

UK Responsible Person
GC UNITED KINGDOM LTD.
Coopers Court Newport Pagnell, Buckinghamshire, MK16 8JS
United Kingdom

Last revised: 03/2024

REX Only

UK CA 0086

REGULATORY GCE

MANUFACTURED BY
GC DENTAL PRODUCTS CORP.
2-285 Torimitsu-cho, Kasugai, Aichi 486-0844, Japan

EU : GC EUROPE N.V.
Researchpark Hasselt/Luizen 1240, Interleuvenlaan 33,
B-3001 Leuven, Belgium TEL: +32 16 74 10 00

RESPONSIBLE MANUFACTURER IN CANADA:
GC Orthodontics (USA) LLC
3737 West 127th Street, Alsip, IL 60803 U.S.A.

DISTRIBUTED BY
GC Orthodontics Europe GmbH
Harkortstrasse 2, 58339 Breckerfeld, Germany

GC Orthodontics America Inc.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
TEL: +1-688-332-4883
www.gc.dental/ortho

GC SOUTH AMERICA
Rua Heliodora, 399, Santana - São Paulo, SP, BRASIL
CEP: 02022-051 - TEL: +55-11-2925-0965
CNPJ: 08.279.990-0001
RESP. TÉCN. Erick de Lima - CRO/SP 100.866

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