Prior to use, carefully read the instructions for use. **EN**

GC Ortho Connect™ Flow

LIGHT-CURED ORTHODONTIC ADHESIVE
For use only by a dental professional in the indications for use.

INDICATIONS FOR USE
Bonding lingual retainers

CONTRAINDICATIONS
Avoid use of this product in patients with known allergies to methacrylate monomer or methacrylate polymer.


COMPOSITION
Dimethacrylate, barium glass, silicon dioxide, phosphoric acid ester monomer, initiator, stabilizer

DIRECTIONS FOR USE
A. **TOOTH PREPARATION**
1. Clean tooth surfaces to be bonded with pumice and water. Rinse thoroughly with water. Dry by gently blowing with an oil-free air.

2. Etch the tooth surface with GC Ortho Etching Gel 37% phosphoric acid for 30 seconds. Rinse thoroughly with water. Refer to the respective instructions for use.
3. Remove excess moisture by blowing with an oil-free air. Prepared surfaces should be dry.

NOTE:
If the surface to be bonded contains porcelain, composite, zirconia or metal, it should be treated accordingly. Porcelain and composite should be sandblasted and pre-treated with silane. Zirconia and metal should be sandblasted. Refer to manufacturer's instructions.

B. **SYRINGE PREPARATION**
1. Hold the syringe upright and remove the cap by turning counterclockwise.

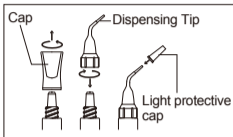2. Promptly and securely attach the dispensing tip to the syringe by turning clockwise.
3. Place the light protective cap until use.
NOTE:
Take care not to attach the dispensing tip too tightly. This may damage its screw. In order to ensure a tight connection make sure that the threads are free of remnant material.

C. **BONDING PROCEDURE**
1. Remove cap from the dispensing tip.
NOTE:

Prior to syringe material, make sure that the dispensing tip is securely attached to the syringe. Any possible air from the dispensing tip is removed by gently pushing forward the syringe plunger with the tip pointing upwards until material reaches the opening of the tip.

2. Slowly push the plunger to syringe material. If plugged, remove the dispensing tip and extrude a small amount of material directly from the syringe.
3. Set the pre-adapted wire on the teeth. Apply GC Ortho Connect Flow paste to the desired area of the wire and tooth. Let the material flow over and around the wire.

NOTE:
Ambient light can shorten the manipulation time of the paste.
4. Light cure paste on each tooth for 10 seconds using an LED light curing device (> 700 mW/cm²).
NOTE:
a. It is very important that the curing light be tested periodically with a light meter to ensure adequate light output.
b. Keep light guide as close as possible to the paste.

SHADE
Clear

STORAGE
Recommended for optimal performance, store at room temperature (4-25°C / 39-2-77°F) away from direct sunlight.

PACKS
2 x 2.7 g (2.0 mL) syringe
30 x DISPENSING TIP III (plastic) for GC Ortho Connect Flow
2 x light protective cap

CAUTION
1. In case of contact with oral tissue or skin, remove immediately with cotton or a sponge soaked in alcohol.

2. Flush with water. To avoid contact with a rubber dam and/or COCOA BUTTER can be used to isolate the operation field from the skin or oral tissue.
3. In case of contact with eyes, flush immediately with water and seek medical attention.

4. Take care to avoid ingestion of the material.
5. In case of contact with unintended areas of tooth or prosthetic appliances, remove with instrument, sponge or cotton pellet before light curing.
6. In case of contact with skin, remove immediately with water and seek medical attention.

7. Use of this product is for single use only. To prevent cross contamination between patients, do not reuse the dispensing tip. The dispensing tips cannot be sterilized in an autoclave or chemically.
8. Do not wipe the dispensing tip with excessive alcohol as this may cause contamination of the material.
9. Do not use GC Ortho Connect Flow in combination with eugenol containing materials as eugenol may hinder GC Ortho Connect Flow from setting or bonding properly.

10. Do not mix with other similar products.
11. Wear plastic or rubber gloves during operation to avoid direct contact with air inhibited resin layers in order to prevent possible sensitivity.
12. Wear protective eye glasses during light curing.
13. When polishing the polymerized material, use a dust collector and wear a dust mask to avoid inhalation of escaping dust.

14. Avoid getting material on clothing.
15. Dispose of all wastes according to local regulations.
16. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

17. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: www.gc.dental/ortho

They can also be obtained from your supplier.

CLEANING AND DISINFECTING
MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.

DO NOT IMMERGE. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional / national guidelines.

For the Summary of Safety and Clinical Performance (SSCP) please see EUDAMED database (<https://ec.europa.eu/tools/eudamed>) or contact us at Regulatory.gce@gc.dental

Undesired effects: Reporting: If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gc.dental In this way you will contribute to improve the safety of this product.

UK Responsible Person
GC UNITED KINGDOM Ltd.
Coopers Court Newport Pagnell, Buckinghamshire, MK16 8US
United Kingdom
Last revised: 03/2024

MANUFACTURED by
GC DENTAL PRODUCTS CORP.
2-295 Torimatsu-cho, Kasugai, Aichi 486-0844, Japan
EU : GC EUROPE N.V.
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33, B-3001 Leuven, Belgium TEL: +32 16 74 10 00

RESPONSIBLE MANUFACTURER IN CANADA:
GC Orthodontics America Inc.
3737 West 127th Street, Alsip, IL, 60803 U.S.A.

DISTRIBUTED by
GC Orthodontics Europe GmbH
Harkortstrasse 2, 58339 Breckerfeld, Germany
GC Orthodontics America Inc.
3737 West 127th Street, Alsip, IL, 60803 U.S.A.
TEL: +1-888-332-4883
www.gc.dental/ortho

GC SOUTH AMERICA
Rua Heliodora, 399, Santana - São Paulo, SP, BRASIL
CEP: 02022-051 - TEL: +55-11-2925-0965
CNPJ: 08.279.999/0001-61
RESP. TEC: Erick de Lima - CRO/SP 106.866

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