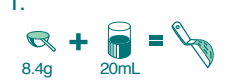
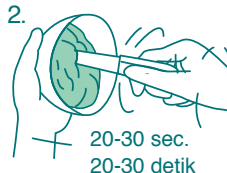
1.  
  
8.4g + 20mL =  

2.  
  
20-30 sec.  
20-30 detik

3.  
  
90 sec.  
90 detik

4.  

5.  

Rinse well. Disinfect if necessary.  
Eliminate excess water before pouring.  
For the most accurate result pour cast immediately after rinsing.

Recommended gypsum:  
GC Fujirock EP (type 4)  
GC NEW PLASTONE II (type 5)

AROMA FINE PLUS  
FAST SET

ASIA

ALGINATE IMPRESSION MATERIAL  
BAHAN CETAK ALGINATE

**CAUTION**  
Store in a cool dry place, in tightly closed container.  
Loosen the powder well before every use.

Mixing time: 20-30 sec.  
Total working time: 70 sec.  
Setting time: 150 sec.  
The setting time includes the mixing time and the minimum manipulation time. This is based on the mixing at 23°C with distilled or deionized water.

Higher temperatures of the mixing water will shorten the setting- and total working time.  
Variations in hardness of the water may influence the setting time.

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23°C

4°C

STORAGE  
Recommended for optimal performance,  
store in a cool and dark place  
(4~25°C) (39.2~77.0°F).

**GC**

GC CORPORATION  
TOKYO, JAPAN

AROMA FINE PLUS  
FAST SET

ASIA

ALGINATE IMPRESSION MATERIAL  
BAHAN CETAK ALGINATE

(NOTES)

1. Prior to use, carefully read the instructions.  
2. For use only by a dental professional in the indications of taking dental impressions.  
3. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.  
4. In case of contact of material or mixture with eyes, immediately flush with water and seek medical attention.  
5. When using material, wear a dust mask to avoid inhaling powder dust.  
6. After opening, transfer the contents into a container for exclusive use of the material.  
7. Do not mix with any other product.

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NET WT. 500g

**GC**

MANUFACTURED by  
GC CORPORATION  
76-1 HASUNUMA-CHO, ITABASHI-KU, TOKYO, JAPAN  
DISTRIBUTED by  
GC ASIA DENTAL PTE. LTD.  
SINGAPORE 529541

MADE IN JAPAN

131627  
270693TP