CERASMART™ T20

FORCE ABSORBING HYBRID CERAMIC CAD/CAM BLOCK

DESCRIPTION
CERASMART™ T20 is a hybrid ceramic for fixed all-ceramic dental restorations.

INTENDED PURPOSE
Indirect anterior and posterior restorations of missing or damaged teeth.

INDICATIONS FOR USE
Missing tooth structure in anterior and posterior teeth.

Areas of application:
1. Metal free indirect restorations: full crown, inlay, onlay, laminated veneer
2. Implant supported crown

CLINICAL BENEFITS
1. Restoring and/or improving masticatory function.
2. Improving the teeth aesthetics.

CONTRAINDICATIONS
Avoid use of this product in patients with known allergies to methacrylate polymer.

INTENDED USERS/SPECIAL TRAINING
• Dentists
• Dental technicians
• No special training required

PATIENT TARGET GROUP
Patients needing to restore missing or damaged tooth tissue in anterior and posterior teeth.

COMPOSITION
Barium glass (70–80%), methacrylate polymer (20–30%), pigment (trace), silicon dioxide (trace)

PERFORMANCE CHARACTERISTICS
The product's performance characteristics satisfy the requirements of the intended use.

DIRECTION FOR USE
1. PREPARATION DESIGN
When designing restorations, the following minimum dimensions should be maintained:

• Minimum wall thickness should be 1.5 mm, 1.0 mm at margins.
Prepare margins with deep chamfer or rounded shoulder. (Figure 1)

• For Inlays and onlays, all internal edges and angles should be rounded. Avoid beveling margins in direct occlusal contact with the opposing tooth.

• Minimum thickness of the restoration should be 1.5 mm in all areas of fissure areas and 1.5 mm in cusp areas.

2. MILLING
Compatibility with CAD/CAM systems:

The material needs to be used with an official milling strategy/program in one of the CAM systems such as CEREC/InLab. The complete list is available at www.gc.dental

CEREC/InLab is not registered trademark of GC. Please refer to respective instructions of the milling unit. Select the milling program for CERASMART™ T20 or composite resin block materials.

After milling, check the restoration for the following:

• Discolorations linked to the milling process
• Protruberances
• Cracks or chips

If the restoration is defective, discontinue the manufacture of restoration.

3. FINISHING AND POLISHING
Finish and polish with coarse and fine silicone point.

DIAPOLUSHER PASTE can be used for final polishing to obtain high gloss NOTE:

When adjusting or polishing the restoration, use a dust collector and wear a dust mask to avoid inhaling dust. Wear protective glasses.

4. CEMENTATION
With Sandblasting technique:

1) Sandblasting with 25-50 µm alumina 0.15 MPa / 1.5 bar is recommended.
2) Blow the restoration with an oil-free air syringe or clean with an ultrasonic cleaner and dry.

3) Clean with alcohol to remove oil film.
4) Treat the surface with a silane coupling agent such as CERAMIC PRIMER II or G-Multi PRIMER.

5) Cement with an adhesive resin cement such as G-CEM LinkForce.

Prior to using CERAMIC PRIMER II, G-Multi PRIMER or G-CEM LinkForce, refer to the respective instructions for use of the manufacturer.

With hydrofluoric acid etching:

1) Treat with hydrofluoric acid (5%) for 60 seconds.
2) Wash with water spray or an ultrasonic cleaner and dry.

3) Treat the surface with a silane coupling agent such as CERAMIC PRIMER II or G-Multi PRIMER.

4) Cement with an adhesive resin cement such as G-CEM LinkForce.

NOTE:

Prior to using CERAMIC PRIMER II, G-Multi PRIMER or G-CEM LinkForce, refer to the respective instructions for use of the manufacturer.

With hydrofluoric acid (35-37%) can be also used for the purpose of cleaning the surface, preferably scrubbing with a microbrush for 10-15 seconds.

5. ADDITIONAL APPLICATION AND REPAIR
Repair the bonding area using diamond point or carbide bur.

Apply CERAMIC PRIMER II or G-Multi PRIMER to improve adhesion and dry the surface.

Apply a bonding agent following the manufacturer's instruction. Proceed with the application of the chosen composite restorative material. G-anal Universal Flo or G-anal Universal Injectable is recommended in order to make the surface of restorative homogeneous as it consists of the same flexible nano ceramic technology as CERASMART™ T20. In this case, the intermediate use of a bonding agent is not necessary.

6. CHARACTERIZATION
When appropriate, OPTIGLAZE color or GRADIA PLUS Lustre Paint is recommended as surface characterization and glazing technique. Sandblast with 25-50 µm alumina (0.15 MPa / 1.5 bar), clean and dry. Apply CERAMIC PRIMER II or G-Multi PRIMER, dry and apply OPTIGLAZE color or GRADIA PLUS Lustre Paint. OPTIGLAZE color and GRADIA PLUS Lustre Paint can only be cured with a light-curing device with light source having wavelength lower than 430 nm. Refer to the respective instructions for use.

DISPOSAL
Dispose of all waste according to local regulations.

STORAGE
Recommended for optimal performance, store in a cool and dark place (4-25°C / 39-77°F).

SIDE EFFECTS
There are no known side effects to date.

RESIDUAL RISKS
• Technical complications: chipping, marginal fracture
• Biological complications: bleeding, pulpitis, abrasion

PRECAUTIONS, WARNINGS AND LIMITATIONS OF USE
Do not use if the prescribed application technique cannot be applied.

1. In rare cases this product may cause sensitivity in some people. If such reactions are experienced, discontinue the use of product and consult a physician.

2. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

3. Do not re-use. Milling partially or fully processed blocks will result in an inaccurate restoration.

Some products referenced in this Instructions for Use may be classified as hazardous according to GHS. Always familiarize yourself with the Safety Data Sheets available at:

<https://www.gc.dental/europe> or for the Americas:

<https://www.gc.dental/americas>

You can also be obtained from your supplier.

For the Summary of Safety and Clinical Performance (SSCP) please see EUDAMED database (<https://ec.europa.eu/tools/eudamed>) or contact us at Regulatory@gc.dental (Basic UDI-DI: +J022MD0034K3)

SHADE
5 Vita shades™ available in two translucencies:

- High Translucency (HT): A1 HT, A2 HT, A3 HT, A3.5 HT, B1 HT

- Low Translucency (LT): A1 LT, A2 LT, A3 LT, A3.5 LT, B1 LT

- One bleach shade (BL)

*Vita® is a registered trademark of Vita Zahnfabrik, Bad Säckingen, Germany.

SIZE
Available in 3 sizes: 12, 14, 14L

REPORTING INFORMATION ACCORDING TO EU MDR
In the case of serious adverse events to the device, please report it to GC EUROPE N.V. (vigilance@gc.dental) and to the competent authority of the Member State in which the user/patient resides.

NOTE:
The current instructions for Use are available in the download section of the <https://www.gc.dental/> website. To access the appropriate version, please select your region from the footer.

UK Responsible Person
GC UNITED KINGDOM Ltd.
Angene Court Newport Pagnell, Buckinghamshire, MK16 8JS
United Kingdom

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