"TISSUE CONDITIONER" FÖR TANDLÖSA KÄKAR
KONDITIONER TIL TANDLÖSE GUMMER
TISSUE CONDITIONER VOOR TANDELLOZE KAK
ACONDICIONADOR DE TEJIDO EDENTADO
CONDIZIONATORE TESSUTI PER EDENTULI
CONDITIONNEUR DE TISSU
(SCHLEIMHAUT-CONDITIONER)
EDENTULOUS TISSUE CONDITIONER
THERAPEUTISCHE, EXTREM WEICHES UNTERFÜLLERUNGSMATERIAL

COE-COMFORTTM

MANUFACTURED by
GC AMERICA INC.
3737 West 127th Street, Alsip, Illinois 60803 U.S.A.

DISTRIBUTED by
GC EUROPE N.V.
Research-Park, Interleuvenlaan 13, B-3001 Leuven, Belgium

COE-COMFORT™

GB

EDENTULOUS TISSUE CONDITIONER

Prior to use, carefully read the instructions. For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS:

Lining of ill fitting dentures as a tissue conditioner to facilitate recovery of damaged soft tissue. Functional impression material for full dentures. Functional impression material for relining the edentulous area of distal extension partials. Impression material for construction of obturators. Lining of surgical and periodontal splints. Diagnostic aid to locate pressure areas on dentures. Liner to stabilize base tray during recording of jaw relations.

CONTRAINdications:

Patients who have shown sensitivity to the material. In case of allergy refer to physician.

DIRECTIONS FOR USE:

TISSUE CONDITIONING:

1. **Pre-treatment considerations:** The restoration of sound condition to damaged soft tissue takes time, requiring several applications of COE-COMFORT. It is recommended that before starting treatment and at each subsequent visit an impression is taken and a study model poured. Comparison of such casts will demonstrate how well treatment is progressing. If the existing dentures do not provide good tissue coverage and are not in proper centric and vertical relationship, they should be rebased and corrected before beginning treatment. If tissues beneath both upper and lower dentures need treatment consider separate treatments. If not practical then prepare the upper first and seat it. Then prepare the lower and seat it. Use separate mixes of COE-COMFORT.

2. **Preparation of the denture:** Using a handpiece, remove undercuts to relieve the denture to prevent impingement. Wash and dry the denture thoroughly. Coat buccal and labial flanges with COE LUBRICANT to prevent adhesion of COE-COMFORT.

3. **Mixing:** Dispense 5ml of COE-COMFORT liquid into a disposable mixing cup. Fill the red measure level with COE-COMFORT powder (6g) and add slowly to the liquid. Mix thoroughly for 30 seconds. Set aside the mixture for approximately 1 minute until it flows sluggishly from the spatula. Note: Using less liquid gives a thicker, faster setting mix and vice versa.

4. **Lining the denture:** Spread the mixture evenly on the surface to be coated, including the peripheral borders. During this time ask the patient to rinse the mouth thoroughly with cold water. The reduced temperature of the mouth provides a more favorable working time. Insert the denture if taking an impression. Instruct the patient to close VERY GENTLY into occlusion. After the denture has been secured, encourage the patient to talk for a few minutes thus stimulating the tissues. After four or five minutes remove the denture and rinse thoroughly in cold water. Trim away excess material. Inspect the contact area of the denture for areas of contact impingement where the pink denture resin shows through the white COE-COMFORT. In such cases relieve the area with a handpiece and wash and dry the denture fully. Make up a small quantity of COE-COMFORT mixture and apply to the freshly relieved area. Instruct the patient to rinse with cold water, reinsert the denture and tell the patient to close VERY GENTLY into occlusion.

5. **Further treatment:** Full restoration of tissue to a sound condition takes time and several repeat applications of COE-COMFORT may be necessary. Ask the patient to return 2-4 days after the initial treatment, and each subsequent treatment. At each visit remove all the COE-COMFORT from the denture using a handpiece and place a new COE-COMFORT lining as described above. Two or three treatments may be sufficient, but continue as often as necessary until you judge that further treatment will bring no further improvement. At this stage reline the denture using a hard chairside reline material.

OTHER INDICATIONS:

In other indications the same basic procedures as used for tissue conditioning are employed, being simply adapted to the particular application.

STORAGE:

Store in a dry location at room temperature (70° to 77°F; 21° to 25°C). (3 year shelf-life guarantee).

PACKAGES:

COE-COMFORT Professional Package

COE-COMFORT Powder, 6oz (170g)

COE-COMFORT Liquid, 6oz (177ml)

WARNING:

COE-COMFORT liquid is flammable. Avoid sources of ignition. COE-COMFORT contains denatured alcohol. In case of contact with skin wash thoroughly with water. In case of contact with eyes, flush immediately with water and seek medical attention.

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