Prior to use, carefully read the instructions for use. (**EN**)

DIAPOLISHER PASTE

DIAMOND POLISHING PASTE FOR COMPOSITE AND PORCELAIN

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

Final high luster polishing of direct, indirect composite resin and porcelain restoration

CONTRAINDICATION

In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician

DIRECTIONS FOR USE

I. Preparation
1) Finish and polish restoration using standard techniques.

2) Hold the syringe upright and remove the wing cap by turning counterclockwise.
3) Attach the dispensing tip correctly to the syringe by turning clockwise.

Note :

Take care not to screw the dispensing tip too tightly. This may damage its screw. In order to ensure a tight connection make sure that the threads are free of remnant material.

4) After attaching the dispensing tip, protect it with the cover until ready for use in order to avoid contamination or drying-out.

II. Dispensing

Slowly push the plunger and dispense sufficient material into a clean dispensing dish or on a mixing pad. The syringe can be stored either with the original cap or with the cover on the dispensing tip.

III. Polishing

Apply a small amount of paste onto an appropriate composite/porcelain polishing brush or cup. Polish all surfaces using moderate speed and pressure (up to 10,000 rpm). Refer to the respective manufacturer’s instructions for use of the brush or cup. After polishing, rinse with water to remove the paste.

Note:
In case of contact with adjacent natural tooth structure, use moderate speed and pressure up to 5,000 rpm.

STORAGE

Store at room temperature away from heat, moisture and direct sunlight (4-25°C).
(Shelf life: 3 years from date of manufacture.)

PACKAGES

Kit contents: Diapolisher Paste 2g x 1, Dispensing tip (needle type) x 1, Protective cover x 1
Refill: Dispensing tip (needle type) 20 pieces

CAUTION

1. To prevent splatter of paste and/or excessive heat generation always use rotary instruments at speeds below 10,000 rpm for direct/indirect composite and porcelain, and below 5,000 rpm for natural tooth. Wear protective eye glasses during use.

2. In case of contact with eyes, flush immediately with water and seek medical attention.

3. Wear plastic or rubber gloves during use to prevent possible sensitivity.

4. When polishing the polymerized material, use a dust collector and wear a dust mask to avoid inhalation of dust particles.

5. The dispensing tip cannot be sterilized in an autoclave or chemiclave.

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