

TECHNIQUE GUIDE






EQUIA
FORTE



EQUIA
FORTE



Class II Cavity Preparation Guide

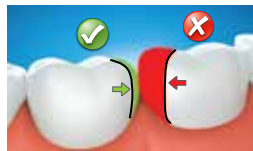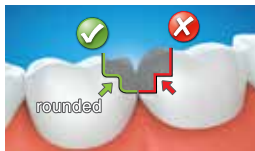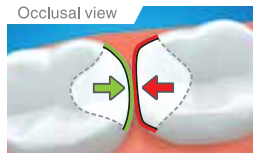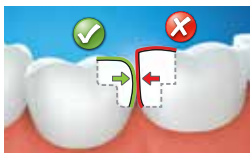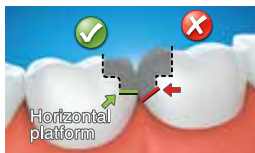

Use sectional matrix system

1-1,5mm distance from cusp peaks

No over-contouring

3M

951708
W9701





Clinical Step by Step



1
Apply petroleum jelly
or GC Cocoa butter
inside the matrix



2
Use anatomical shaped
wedges for better
adaptation and
contact points



3
Use of tight rings
from sectional matrix
systems act as separator
of teeth to ensure good
contact points



4
OPTIONAL STEP:
Apply GC Cavity
Conditioner(10 sec.)
or Dentin Conditioner
(20 sec.)



5
Rinse and gently dry, do
not desiccate



6
Shake or tap.
Depress plunger



7
Insert on
Capsule Applier.
Click once to activate



8
Mix for **10 sec.**
Working time is
1 min.15 sec.
from start of mix



9
Insert on Capsule
Applier. Click twice
to prime capsule



10
IMMEDIATELY
Dispense within 10 sec.



11
Pack and contour.
Avoid moisture
contamination and
dry-out



12
Ensure complete set of
EQUIA® Forte Fil and
carefully remove the ring.
Use a probe to separate
the bond between matrix
and EQUIA® Forte Fil



13
Final finishing after
2 min 30 sec. from
start of mix



14
Finish the restoration
by applying the
EQUIA® Forte coat.
DO NOT AIR BLOW



15
Light cure for **20 sec.**

Note: Steps except for 1, 2, 3 and 12 are common for Class I and II