G-Multi PRIMER
PRIMER FOR GLASS CERAMIC HYBRID CERAMICS, ZIRCONIA, ALUMINA, COMPOSITES AND METAL BONDING

G-Multi PRIMER is a priming agent designed to treat the inherent surface of glass and glass/ceramic hybrids (e.g. CERASAMARTZ70), zirconia, alumina, composites and metal restorations. This is to promote a strong bond to the restorations. It is used in conjunction with the following steps of curing procedures or direct composites in case of repairs.

For use only by a dental professional in the indications for use.

INDICATIONS FOR USE

1. Priming of adherent surface of indirect restorations made of glass ceramics, hybrid ceramics (e.g. CERASAMARTZ70), zirconia, alumina, composites and metal before luting procedure.
2. Intra-oral and extra-oral repair of fractured indirect restorations made of glass or hybrid ceramics, zirconia, alumina, and composites.

CONTRAINDICATIONS

Avoid use of this product in patients with known allergies to methacrylate monomer, methacrylate polymer or ethanol.

COMPOSITION

Ethanol, dimethacrylate, MDP, silane coupling agent, MDP7

DIRECTIONS FOR USE

- A. For luting procedure.
 1. Treat adherent surface of restoration according to the respective manufacturer's instructions and then clean it using a suitable cleaning method. Rinse and dry with oil-free air syringe.
 2. Dispense one drop of G-Multi PRIMER into a dispensing dish.
 3. Apply the primer to the adherent surface of the restoration using a micro-pip applicator and dry with oil-free air syringe.
 4. Apply suitable resin-based luting cement to the primed surface of the restoration. For these steps, refer to the manufacturer's instructions for use of the corresponding luting cement.
- B. For intra-oral Repair.
 1. Roughen adherent surface using a coarse diamond bur / a carbundum point and/or with sandblaster.
 2. Roughen the surface of the restoration according to the respective manufacturer's instructions and then clean it using a suitable cleaning method. Rinse and dry with oil-free air syringe.
 3. Dispense one drop of G-Multi PRIMER into a dispensing dish.
 4. Apply a thin layer to the fractured surface of the restoration using a micro-pip applicator and dry with oil-free air syringe.
 5. Continue the repair by using a light-cured adhesive (e.g. G-Premio BOND), zirconium oxide primer (e.g. G-Genial, Essentia), referring to their respective manufacturer's instructions for use.

If treated surface is contaminated with saliva or blood, mix with water and use to treat the surface according to the manufacturer's instructions and re-apply G-Multi PRIMER according to the instructions above.

STORAGE

Recommended for optimal performance, store in a cool and dark place (4-25°C / 39-77°F).

CAUTION

1. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
2. G-Multi PRIMER is flammable and volatile. Do not use in open flame, keep away from heat sources.
3. Keep away from direct sunlight. Use in a well ventilated space.
4. In case of contact with oral tissues, immediately remove with a cotton pellet or a sponge. Rinse immediately with water.
5. In case of contact with skin, remove immediately with a cotton pellet or a sponge. Rinse with soap and water.
6. In case of contact with eyes, flush immediately with water and seek medical attention.
7. Replace the cap of both bottles immediately after use.
8. If condensation (water) is present on the bottle of G-Multi PRIMER, immediately wipe it off.
9. Avoid inhalation or ingestion of the product.
10. Do not mix with any other primers.
11. If you experience any adverse sensitivity in some people, if any such reactions are experienced, discontinue the use of the product and refer to a physician.

Some products referenced in the present IAU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <https://www.gdental.europa.com> or ask us for The Americas <https://www.gdentalamerica.com>

They can also be obtained from your supplier.

CLEANING AND DISINFECTING

MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires low-level disinfection. Immediately after use please remove and label for deterioration. Discard device if damaged. DO NOT IMMERGE: This device is not intended to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional / national guidelines.

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Ethanol, Dimethacrylat, MDP, Silan-Haftvermittler, MDP7

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