Master all your challenges

*initial*TM
Spectrum Stains

- EN Technical Manual
- DE Technisches Handbuch
- FR Manuel Technique
- IT Manuale tecnico
- ES Manual técnico
- PL Instrukcja techniczna

JUNE 2019

GC

E	DE	FR	IT	
Table of Contents	Inhaltsverzeichnis	Table des matières	Indice	Page
1. Intended Use & Set Composition	1. Bestimmungsgemäße Verwendung und Set-Zusammensetzung	1. Indications & Composition des coffrets	1. Destinazione d'uso e composizione del set	4 - 5
2. Description of GC Initial Spectrum Stains	2. Beschreibung der GC Initial Spectrum Stains	2. Description des GC Initial Spectrum Stains	2. Descrizione di GC Initial Spectrum Stains	6
3. The Glaze & Glaze Liquids	3. Glasurflüssigkeit und Glasurpaste	3. Poudre Glaze & Liquides Glaze	3. La polvere e i liquidi per glasura	7
4. GC Initial Spectrum Stain Application	4. Anwendung der GC Initial Spectrum Stains	4. Application des GC Initial Spectrum Stains	4. Applicazione di GC Initial Spectrum Stain	
External Characterization	Äußere Charakterisierung	Caractérisation externe	Caratterizzazione esterna	8 - 11
Internal Characterization	Innere Charakterisierung	Caractérisation interne	Caratterizzazione interna	12
Combined Characterization	Kombinierte Charakterisierung	Caractérisation combinée	Caratterizzazione combinata	14 - 15
5. Firing Chart	5. Brenntabelle	5. Tableau de cuisson	5. Tabella dei parametri di cottura	16
6. Physical Properties & Shelf Life	6. Physikalische Eigenschaften und Haltbarkeit	6. Propriétés physiques & Péremption	6. Proprietà fisiche e durata utile del prodotto	17
7. FAQ	7. Häufig gestellte Fragen	7. FAQ	7. Domande frequenti	18 - 22
8. Related Products	8. Verwandte Produkte	8. Produits associés	8. Prodotti collegati	42

1

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EN One for All

GC Initial Spectrum Stains are universal stains & glaze for the individualized staining and characterization of ceramic materials.

- **A wide spectrum of use**

The stains and glaze are developed for the external staining and glazing of tooth coloured ceramic materials and internal individualization of layering ceramics.

Compatible to a wide CTE-range, the GC Initial Spectrum Stains are perfectly adapted to fit on nearly all types of dental ceramics.

In particular they are adapted to the aesthetical colour concept of the GC Initial ceramic portfolio.

- **A wide spectrum of colours**

GC Initial Spectrum Stains are powder stains available in a variety of 16 colours empowering the aesthetics of your restorations.

- **Adaptable consistency of the stains & glaze**

By using either the Glaze or the Glaze Paste Liquid the stains & glaze can be adapted to the preferred consistency of the user.

Regardless the way of application a very fine end result will be obtained maintaining the surface texture and assuring an optimal gloss level.

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GC Initial Spectrum Stain Set – Set Composition

- 16x GC Initial Spectrum Stain SPS1 – SPS16, 3g
- 1x GC Initial Spectrum Glaze Powder GL, 10g
- 1x GC Initial Spectrum Glaze Liquid, 25ml
- 1x GC Initial Spectrum Glaze Paste Liquid, 8ml
- 1x GC Initial Spectrum Stain Shade Guide

*We advise you to thoroughly check the Q&A for further clarification on application details.
 *Wir empfehlen Ihnen, die Fragen und Antworten sorgfältig durchzulesen, um weitere Erklärungen zu den Anwendungsdetails zu erhalten.
 *Nous vous conseillons de bien lire les questions et réponses pour plus de précisions sur les détails de l'application
 *Per ulteriori chiarimenti sui dettagli relativi all'applicazione consigliamo di consultare attentamente la sezione Domande e risposte.

2

- EN Description of the GC Initial Spectrum Stains
- DE Beschreibung der GC Initial Spectrum Stains
- FR Description des GC Initial Spectrum Stains
- IT Descrizione di GC Initial Spectrum Stains

- | | | | |
|--|---|---|---|
|  | Initial Spectrum Stain SPS-1, 3g - ivory white |  | Initial Spectrum Stain SPS-9, 3g - brown |
|  | Initial Spectrum Stain SPS-2, 3g - melon yellow |  | Initial Spectrum Stain SPS-10, 3g - dark walnut |
|  | Initial Spectrum Stain SPS-3, 3g - signal yellow |  | Initial Spectrum Stain SPS-11, 3g - coral |
|  | Initial Spectrum Stain SPS-4, 3g - light terracotta |  | Initial Spectrum Stain SPS-12, 3g - denim blue |
|  | Initial Spectrum Stain SPS-5, 3g - savannah gold |  | Initial Spectrum Stain SPS-13, 3g - twilight |
|  | Initial Spectrum Stain SPS-6, 3g - caramel |  | Initial Spectrum Stain SPS-14, 3g - deep orange |
|  | Initial Spectrum Stain SPS-7, 3g - camel |  | Initial Spectrum Stain SPS-15, 3g - sunset |
|  | Initial Spectrum Stain SPS-8, 3g - olive |  | Initial Spectrum Stain SPS-16, 3g - midnight |

- EN The Glaze & Glaze Liquids
- DE Glasurflüssigkeit und Glasurpaste
- FR Poudre & Liquides Glaze
- IT La polvere e i liquidi per glasura

• EN **GC Initial Spectrum Glaze Powder GL, 10g**

Fine Glaze powder to be mixed with the Glaze or Glaze Paste Liquid.

The Glaze Powder can be adapted to the preferred consistency of the user by using the Glaze or the Glaze Paste Liquid.

GC Initial Spectrum Glaze Liquid, 25ml

The standard low viscosity type of mixing liquid allowing a fine application.

GC Initial Spectrum Glaze Paste Liquid, 8ml

The high viscosity type of mixing liquid allowing a gel-type application.

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GC Initial Spectrum Liquide Glaze, 25ml

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• IT **GC Initial Spectrum Glaze Powder GL, 10g**

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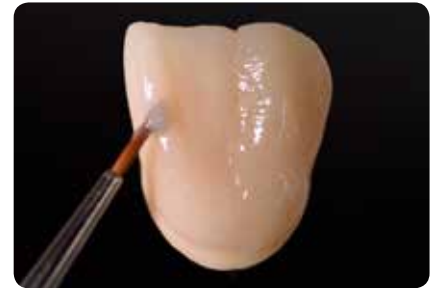
EN Initial situation

Ceramic crown with individual micro-texture, ready to be stained & glazed. *Note: before staining and glazing, the ceramic needs to be sand-blasted with 50 microns Alu-Oxid at 1,5 bar pressure and steam cleaned or cleaned by ultra-sonic bath.*

By using either the Glaze or the Glaze Paste Liquid the stains can be adapted to the preferred consistency.

Left: "Gel-type" mixture with Glaze Paste Liquid.

Right: "Fine" mixture with Glaze Liquid.

Coat the entire surface of the restoration with the Spectrum Glaze in the preferred consistency.

Left part of the crown: application of the Glaze using the Glaze Paste Liquid.

Right part of the crown: Glaze applied using the Glaze and Glaze Liquid.

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EN Initial LiSi Press Crown
As connection firing Lustre Pastes NF have been used on the entire surface.
For extra internal individualization the Initial Spectrum Stains are applied and fired.

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EN Layering of the Initial LiSi Press Crown with the Initial LiSi veneering ceramic.
Initial Spectrum Stains are used for internal individualization during the layering process.

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EN Please consult the firing parameters of the dedicated layering materials.

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EN External staining of the Initial Lisi Press Crown to mimic nature analogue ageing of the tooth.

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EN Final result after internal and external staining.

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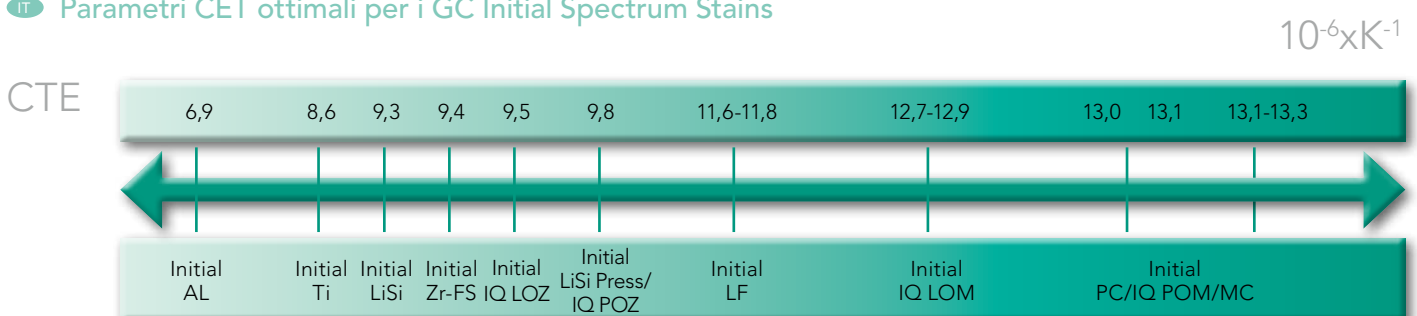
EN *All values mentioned in the physical properties overview meet the performance criteria as defined in EN ISO 6872:2008 Dentistry – Ceramic Materials ISO 6872:2008).

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EN Important to know:

- GC Initial LRF Block
It is advised to use the GC Initial LRF Glaze Paste as dedicated glazing material.
The GC Initial Spectrum Stains can be added in limited amounts to the GC Initial LRF Glaze Paste for individualisation.
- Layering materials can be individually toned using the Spectrum Stains.
When mixed into Dentin, Enamel, Transpa powders it is advised to use small amounts.
To assure the right mixing ratio we advise to create a small firing sample.
- When used during build-up on an unfired ceramic small amounts can be added.
- When mixing layering material and stains use the recommended modelling liquid not the Initial Spectrum Stain Liquid.
- Always consult the firing parameters of the dedicated layering materials.

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EN Q2. Are the GC Initial Spectrum Stains & Glaze fluorescent?

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EN Similar to natural teeth the GC Initial Spectrum Stains offer a certain level of fluorescence. The Glaze Powder however is a non-fluorescent glazing material. It protects and glazes your surface. Fluorescence is given by the underlying ceramic powders / core material.

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EN Q3. Can the GC Initial Spectrum Stains be mixed with the GC Initial IQ One Body Lustre Pastes NF?

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EN Yes they can be mixed with moderation to intensify the Lustre Pastes NF. Mix the required amount of stain powder with the LP NF Dilution Liquid and mix it with the required LP NF or directly mix the stain powders with e.g. the Lustre Paste Neutral (L-N).

Initial Spectrum Stains and Lustre Pastes NF are compatible products.

Initial Lustre Pastes NF are ready-to-use 3-dimensional dynamic shades and are based on a special mix of fine ceramic particles.

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EN Q4. Are there any contraindications for use of the GC Initial Spectrum Stains?

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EN Everything what is not explicitly listed as indication – refer to Intended Use & Shelf Life.

Do not mix with other stains.

The stains are not suitable for staining, characterization and glazing of partially crystallized CAD restorations (e.g. IPS e.max® CAD – Ivoclar Vivadent).

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EN Q5. I do not have a shiny surface after firing the Glaze, what might be the cause?

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EN Check the firing temperatures, increase the final temperature. When the GC Initial Spectrum Stains are heavily diluted or applied in a too thin layer the glossy effect is tempered, so apply a more thick layer/dilute less the pastes.

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EN Yes, you can do further firings, keeping the same firing temperatures.

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PL Opis GC Initial Spectrum Stains

- | | |
|--|---|
|  Initial Spectrum Stain SPS-1, 3g - ivory white |  Initial Spectrum Stain SPS-9, 3g - brown |
|  Initial Spectrum Stain SPS-2, 3g - melon yellow |  Initial Spectrum Stain SPS-10, 3g - dark walnut |
|  Initial Spectrum Stain SPS-3, 3g - signal yellow |  Initial Spectrum Stain SPS-11, 3g - coral |
|  Initial Spectrum Stain SPS-4, 3g - light terracotta |  Initial Spectrum Stain SPS-12, 3g - denim blue |
|  Initial Spectrum Stain SPS-5, 3g - savannah gold |  Initial Spectrum Stain SPS-13, 3g - twilight |
|  Initial Spectrum Stain SPS-6, 3g - caramel |  Initial Spectrum Stain SPS-14, 3g - deep orange |
|  Initial Spectrum Stain SPS-7, 3g - camel |  Initial Spectrum Stain SPS-15, 3g - sunset |
|  Initial Spectrum Stain SPS-8, 3g - olive |  Initial Spectrum Stain SPS-16, 3g - midnight |

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PL GC Initial Spectrum Glaze Powder GL, 10 g

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GC Initial Spectrum Glaze Liquid, 25 ml

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GC Initial Spectrum Glaze Paste Liquid, 8 ml

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GC Initial LiSi Press

GC Initial MC, Zr-FS, LiSi

EN Medical Device Regulation Requirement

Undesired effects - Reporting: If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gc.dental In this way you will contribute to improve the safety of this product.

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GC NORDIC AB - Finnish Branch
GC NORDIC AB - Danish Branch
GC NORDIC AB
GC AUSTRIA GmbH - Swiss Office
GC AUSTRALASIA DENTAL PTY LTD
GC ASIA DENTAL PTE. LTD.