Prior to use, carefully read the instructions for use.

EN

## GC Tooth Mousse™

TOPICAL CREME WITH CALCIUM AND PHOSPHATE

For use by or under the supervision of a dental professional in the indications for use.

**Description**  
GC Tooth Mousse is a water based creme containing RECALDENT™ (CPP-ACP: Casen Phosphopetite - Amorphous Calcium Phosphate). When CPP-ACP is applied in the oral environment, it will bind to biofilms, plaque, bacteria, hydroxyapatite and soft tissue localising bio-available calcium and phosphate.

**Important:**  
Saliva will enhance the effectiveness of CPP-ACP and the flavor will help stimulate saliva flow. The longer CPP-ACP and saliva are maintained in the mouth, the more effective the result.

**A topical paste containing bio-available calcium and phosphate**  
1. Provides extra protection for teeth.  
2. Helps neutralize acid challenges from acidogenic bacteria in plaque.  
3. Helps neutralize acid challenges from other internal and external acid sources.

**CONTRAINDICATIONS**  
RECALDENT™ (CPP-ACP) is derived from milk casien. Do not use this material on patients with a proven or suspected milk protein allergy and/or with a sensitivity or allergy to benzoate preservatives.

**DIRECTIONS FOR USE**  
**INSTRUCTIONS FOR USE**  
a. Prior to and following in-office bleaching.  
b. After ultrasonic, hand scaling or root planing.  
c. Following professional tooth cleaning (P.T.C.)  
d. After application of topical fluoride.  
e. To provide a topical coating for patients suffering from erosion, xerostomia or Sjögrens syndrome.  
f. During orthodontic treatment.  
g. For high risk caries patients.  
h. For special needs patients.

**In-office application:**

**1. Custom tray application**  
1. Before use rinse the custom tray thoroughly under running water.  
2. Extrude a generous layer of GC Tooth Mousse into the tray and apply to the upper and/or lower teeth.  
3. Leave the tray undisturbed in the mouth for a minimum of 3 minutes.  
4. Remove the tray.  
5. Instruct the patient to use the tongue to spread the remaining GC Tooth Mousse throughout the mouth. Instruct the patient to retain for as long as possible (1- 2 minutes) avoiding expectoration and delaying swallowing. The longer GC Tooth Mousse and saliva are maintained in the mouth, the more effective the result.  
6. Ask the patient to expectorate and if possible avoid rinsing. Any GC Tooth Mousse remaining on the surface can be left to gradually dissipate. Advise the patient not to eat or drink for 30 minutes following application.  
7. Any residual GC Tooth Mousse remaining on the tray should be rinsed or brushed off under running water immediately after use.

**II. Non tray application:**

1. If necessary, remove any excess saliva on the tooth surface with a cotton roll or pellet. However, it is NOT necessary to dry the teeth with compressed air.  
2. Apply a sufficient amount of GC Tooth Mousse to the tooth surfaces using an application swab, gloved finger or in difficult interproximal areas use an Interproximal Tooth Cleaning Brush.  
3. Leave GC Tooth Mousse undisturbed for a minimum of 3 minutes.  
4. Then instruct the patient to use the tongue to spread the remaining GC Tooth Mousse throughout the mouth. Request the patient to hold in the mouth for as long as possible (an additional 1 - 2 minutes) avoiding expectoration and delaying swallowing. The longer GC Tooth Mousse and saliva are maintained in the mouth, the more effective the result.  
5. Ask the patient to expectorate and if possible avoid rinsing. Any GC Tooth Mousse remaining on the surface can be left to gradually dissipate. Advise the patient not to eat or drink for 30 minutes following application.

**At home application:**

**III. Day time application after tooth brushing as recommended by a dental professional**  
1. Apply a sufficient amount of GC Tooth Mousse to upper and lower teeth. A pea size amount for each arch is the minimum required. The material should be applied to the tooth surfaces using a clean dry finger or cotton tip. For difficult areas (between the teeth) use an Interproximal Tooth Cleaning Brush or dental floss coated with GC Tooth Mousse.  
2. Leave GC Tooth Mousse on the teeth undisturbed for a minimum of 3 minutes.  
3. Then use your tongue to spread the remaining GC Tooth Mousse throughout the mouth. Hold in the mouth for as long as possible (a further 1 - 2 minutes) avoiding expectoration (spitting out) and delaying swallowing. The longer GC Tooth Mousse and saliva are maintained in the mouth, the more effective the result.  
4. Expectorate thoroughly and if possible avoid rinsing. Any GC Tooth Mousse remaining on the surface can be left to gradually dissipate. Do not eat or drink for 30 minutes following application.

**IV. Night time application after tooth brushing as recommended by a dental professional**  
1. Apply a sufficient amount of GC Tooth Mousse to upper and lower teeth. A pea size amount for each arch is the minimum required. The material should be applied to the tooth surfaces using a clean dry finger or cotton tip. For difficult areas (between the teeth) use an Interproximal Tooth Cleaning Brush or dental floss coated with GC Tooth Mousse.  
2. Leave GC Tooth Mousse on the teeth undisturbed for a minimum of 3 minutes.  
3. Then use your tongue to spread the remaining GC Tooth Mousse throughout the mouth. Hold in the mouth for as long as possible (a further 1 - 2 minutes) avoiding expectoration (spitting out) and delaying swallowing. The longer GC Tooth Mousse and saliva are maintained in the mouth, the more effective the result.  
4. Expectorate thoroughly and if possible avoid rinsing. Any GC Tooth Mousse remaining on the surface can be left to gradually dissipate overnight. Do not eat or drink for 30 minutes following application.

### FLAVOR

Milkskake

### STORAGE

Recommended for optimal performance, store in a cool dry low humidity area away from direct sunlight (8-25°C) (46.4-77.0°F).

### PACKAGES

Tube containing 40 g (35 mL)  
1. Single Flavor Package  
10 x 40 g in Milkshake  
2. Refill Package  
1 x 40 g tube in Milkshake

### NOTE

\*RECALDENT and RECALDENT logo are trademarks of Mondelz International group used under license.

### CAUTION

1. Always replace cap after use and remove any residual paste or moisture around the tube neck or inside the cap.  
2. In case of contact with eyes, flush immediately with water and seek medical attention.  
3. In case of spillage on clothes, wash off with water.  
4. If any angioedema symptoms are experienced, this may indicate sensitivity or allergy to the benzoate preservatives, or to some other component of the product such as a flavoring agent. In this event, discontinue the use of the product and refer to a physician.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <https://www.gc-dental.de>

They can also be obtained from your supplier.

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**III. Tägliche Anwendung nach dem Zähneputzen, wie vom zahmmedizinischen Fachpersonal empfohlen**  
1. Apply a sufficient amount of GC Tooth Mousse to upper and lower teeth. A pea size amount for each arch is the minimum required. The material should be applied to the tooth surfaces using a clean dry finger or cotton tip. For difficult areas (between the teeth) use an Interproximal Tooth Cleaning Brush or dental floss coated with GC Tooth Mousse.  
2. Leave GC Tooth Mousse on the teeth undisturbed for a minimum of 3 minutes.  
3. Then use your tongue to spread the remaining GC Tooth Mousse throughout the mouth. Hold in the mouth for as long as possible (a further 1 - 2 minutes) avoiding expectoration (spitting out) and delaying swallowing. The longer GC Tooth Mousse and saliva are maintained in the mouth, the more effective the result.  
4. Expectorate thoroughly and if possible avoid rinsing. Any GC Tooth Mousse remaining on the surface can be left to gradually dissipate. Do not eat or drink for 30 minutes following application.

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