KOOLINER™

HARD CHAIRSIDE DENTURE RELINE

Prior to use, carefully read the instructions. For use only by a dental professional in the recommended indications.

CONTRAINDICATIONS:

Patients who have shown sensitivity to methacrylates. In case of allergy refer to a physician.

RECOMMENDED INDICATIONS:


A temporary lining for acrylic dentures. For use in chairside procedures.

DIRECTIONS FOR USE:

1. Preparation of the denture: Relieve and roughen the area of the denture to be relined. Clean and dry the denture thoroughly. Coat labial and buccal surfaces of the denture with COE LUBRICANT. Do not apply coating within 3mm (1/8 inch) of the peripheral border. If the denture has plastic teeth also protect them with COE LUBRICANT.

2. Preparation of KOOLINER: Recommended powder/liquid ratio is 15ml powder to 6ml liquid. Pour liquid into the mixing cup and then add the powder slowly. Stir thoroughly for no more than 30 seconds and avoid the introduction of air bubbles.

ADD POWDER TO LIQUID AND STIR FOR 30 SECONDS

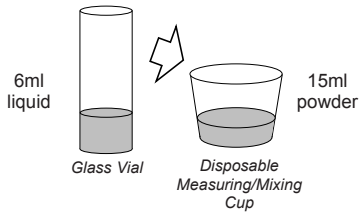3. Application. After approximately 1-2 minutes spread the mixture of KOOLINER over the area to be relined. Seat the denture in the manner of taking an impression and instruct the patient to close lightly into occlusion. After 3 minutes instruct the patient to move lips and cheeks so that a muscle trimmed periphery is obtained. Remove the denture and rinse under cold water. Trim away excess material. Re-seat the denture and instruct the patient to close FIRMLY into occlusion, and to hold this position for 5 minutes. Remove the denture and rinse again in cold water.

4. Finishing. Peak curing temperature: Approximately 43°C/110°F at 7 minutes when tested according to ADA/ANSI specification number 17. In thicker applications, peak temperature may exceed that stated above, possibly producing a hazardous condition in the mouth during curing. When curing is complete (10 minutes), trim away excess. For smoothing the edges use a hot spatula.

STORAGE CONDITIONS:

Keep dry. Excessive heat and freezing temperatures may impact product performance. For optimal performance store in a cool, dark place (2°C to 37°C / 36°F to 98°F). (Shelf life: 3 years from date of manufacture).

PACKAGES:

345001 KOOLINER Professional Package
345002 KOOLINER Powder 3oz (80g)
345091 KOOLINER Liquid (55mL)

Last revised: 11/2013

MANUFACTURED BY
GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803, U.S.A.

DISTRIBUTED BY:
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-Ku, Tokyo 174-8585, Japan

EU: GC EUROPE N.V.
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33,
B-3001 Leuven, Belgium
TEL: +32 16 74 10 00 7831113

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