Prior to use, carefully read the instructions for use.

EN

TEMPRON

TEMPORARY CROWN AND BRIDGE RESIN

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

For the preparation of temporary crowns, bridges or dentures. Space maintenance.

CONTRAINDICATIONS

In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

DIRECTIONS FOR USE

- A. CROWNS
- PREPARATION
 - Fill defects in the tooth with GC CAVITON or equivalent temporary filling material.
 - Take an impression using a heavy (i.e. less water) alginate impression material. Wrap the impression in wet gauze to prevent distortion.
 - Prepare the tooth to be crowned.
 - Cut V notches in the impression from gingival to buccal and lingual sides to allow escape of excess resin.
 - MIXING
 - The standard powder/liquid ratio is 1.0 g/0.5 mL (up to the first graduation on the powder measure and the first graduation of the liquid pipette).

Note:

To minimize shrinkage use as little liquid as possible.
 - Measure the liquid into the mixing jar, and slowly add the corresponding quantity of powder. Mix for 20-30 seconds.
 - APPLICATION
 - Pour the creamy paste into the prepared impression, and seat tightly in the mouth.
 - After 3 minutes in the mouth remove the impression, leaving the dough like resin on the tooth.
 - Remove the resin crown before it is fully set and dip in and out of hot water (± 50°C) several times in order to prevent shrinkage.
 - Trim any excess using a bur and then polish.
 - Cement the temporary crown in position using GC FREEGENOL or other appropriate temporary cement.

B. BRIDGES

- PREPARATION
 - Select a suitable artificial tooth and fix between the abutment teeth using a suitable wax. Check the bite.
 - Take a full arch impression using an alginate impression material. Carefully remove the pontic and wrap the impression in wet gauze.
 - Prepare the abutment teeth as appropriate. Subsequent procedures are carried out as described above for crowns A. 1 (a) to A. 3 (e).

STORAGE

Store in a cool place away from direct sunlight. (Shelf life: 3 years from date of manufacture)

PACKAGES

- Bottle of 100 g powder.
- Bottle of 100 g (104 mL) liquid.
- 1-1 package : 100 g powder, 100 g (104 mL) liquid, accessories.

WARNING

- The liquid is volatile and highly flammable. Close the bottle immediately after use and keep away from sources of ignition.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- Use of polysulfide rubber based impression materials is not recommended.

Last revised : 08/2008

MANUFACTURED by
GC DENTAL PRODUCTS CORP.
2-285 Toriimatsu-cho, Kasugai, Aichi 486-0844, Japan

DISTRIBUTED by
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

 EU : GC EUROPE N.V.
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33,
B-3001 Leuven, Belgium TEL: +32. 16. 39. 80. 50.

GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
TEL: +1-708-597-0900.

GC ASIA DENTAL PTE. LTD.
19 Loyang Way, #06-27 Singapore 508724
TEL: +65 6546 7588

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DIRECTIONS FOR USE

- CROWNS**
 - PREPARATION**
 - Fill defects in the tooth with GC CAVITON or equivalent temporary filling material.
 - Take an impression using a heavy (i.e. less water) alginate impression material. Wrap the impression in wet gauze to prevent distortion.
 - Prepare the tooth to be crowned.
 - Cut V notches in the impression from gingival to buccal and lingual sides to allow escape of excess resin.
 - MIXING**
 - The standard powder/liquid ratio is 1.0 g/0.5 mL (up to the first graduation on the powder measure and the first graduation of the liquid pipette).

Note:

To minimize shrinkage use as little liquid as possible.
 - Measure the liquid into the mixing jar, and slowly add the corresponding quantity of powder. Mix for 20-30 seconds.
 - APPLICATION**
 - Pour the creamy paste into the prepared impression, and seat tightly in the mouth.
 - After 3 minutes in the mouth remove the impression, leaving the dough like resin on the tooth.
 - Remove the resin crown before it is fully set and dip in and out of hot water (± 50°C) several times in order to prevent shrinkage.
 - Trim any excess using a bur and then polish.
 - Cement the temporary crown in position using GC FREEGENOL or other appropriate temporary cement.

B. BRIDGES

- PREPARATION**
 - Select a suitable artificial tooth and fix between the abutment teeth using a suitable wax. Check the bite.
 - Take a full arch impression using an alginate impression material. Carefully remove the pontic and wrap the impression in wet gauze.
 - Prepare the abutment teeth as appropriate. Subsequent procedures are carried out as described above for crowns A. 1 (a) to A. 3 (e).

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