

醫療器材仿單標籤粘貼表

產品中文名稱	“而至”齒頸部黏固劑	申請廠商	台灣而至股份有限公司
--------	------------	------	------------

“而至”齒頸部黏固劑 “GC” Cervical Cement

許可證：衛署醫器輸字第 012646 號

製造日期：批號：

有效期限：如外包裝所示

台灣而至股份有限公司

台北市民生東路三段 156 號 4 樓

“而至”齒頸部黏固劑 “GC” Cervical Cement

衛署醫器輸字第 012646 號

注意：使用前請務必詳讀原廠之使用說明書並遵照指示使用
本案產品使用方法、禁忌症、警語及注意事項請參照原廠資料

產品敘述與特性

本產品因其玻璃離子對牙齒結構的黏著性，相容性與美觀性，主要用於齒頸部與根管表面的密封。它可提供快速攪拌時間，低水敏感性，表面的高強度與抗蝕性。使用本產品於有牙齦退縮的牙周病人，可減低其對牙齒的敏感反應。

1. 當您按照使用說明製備齒頸黏固粉(GC CERVICAL CEMENT)時，因為混合料的流動性極佳，所以很容易就可以將之塗在牙齒表面上。
2. 塗上之後很快就會固定。
3. 可穩固黏著於牙齒表面。
4. 可提供優異的生物相容性，幾乎不會對牙髓或組織產生刺激。
5. 在塗上並固定之後，黏固粉的表面會保持光滑，事實上，即使放在口腔內一段很長的時間，依然不會變色或染色。

用途

填封齒頸與牙根表面的過敏部位。

物理性質

()：標準差

粉/液比 (公克/公克)	1.0/1.0	1.2/1.0 (標準)	1.4/1.0
作用時間 (分/秒)	2 分 00 秒	1 分 30 秒	1 分 00 秒
固定時間 (分/秒)	3 分 00 秒	2 分 30 秒	2 分 00 秒
堅硬度 (公釐) (負重 120 公克, 1 分鐘後)	40	33	27
抗壓強度 (MPa, 1 天後)	160 (9.8)	182 (3.9)	189 (5.9)
黏著強度 (MPa, 1 天後)			
牛體牙釉質	2.5 (0.4)	3.0 (0.4)	3.2 (0.6)
牛體牙本質	1.8 (0.4)	2.2 (0.4)	2.2 (0.5)
崩解率 (%) (1 天後)	0.29 (0.02)	0.23 (0.01)	0.19 (0.02)

測試條件：溫度 (23±1℃) — 相對濕度 (50±5%)

色調

牙齦色、淺棕色

包裝

1. 000016 10 公克粉末，瓶裝。
2. 000126 10 公克液體，瓶裝。
3. 000018 1-1 包裝：1 瓶 10 公克粉末，1 瓶 10 公克液體與附件。

製造廠名稱：GC CORPORATION Fuji Oyama Factory
製造廠地址：584-1, Nakahinata, Oyama-Cho, Sunto-Gun, Shizuoka-Ken 410-1307, Japan

藥商名稱：台灣而至股份有限公司

CERVICAL CEMENT

GLASS IONOMER CEMENT FOR CERVICAL & ROOT SURFACES

CIMENT VERRE IONOMERE POUR ZONE CERVICALE ET SURFACE RADICULAIRE

GLASIONOMERZEMENT FÜR ZAHNHALS-UND WURZELBEREICHENVERSIEGELUNG

CEMENTO VETRO IONOMERICO PER SUPERFICIE CERVICALI E RADICOLARI

CEMENTO DE VIDRIO IONOMERO PARA SUPERFICIES CERVICALES Y RADICULARES

0.70 (0.02)	0.50 (0.01)	0.20 (0.01)	0.10 (0.01)
----------------	----------------	----------------	----------------

Test Conditions: Temperature 23±1°C, Relative Humidity 50±5%

2. Do not get the liquid on the teeth or the tongue.

DIRECTIONS FOR USE

1. Apply GC DENTIN CONDITIONER to the area to be treated.

2. Mix the powder and liquid in a small amount of water or saliva.

3. Apply the cement to the area to be treated using a brush or applicator.

4. Allow the cement to set for 1 minute.

5. Rinse the area with water and dry it.

6. The cement is now ready for use.

7. The cement is now ready for use.

8. The cement is now ready for use.

9. The cement is now ready for use.

10. The cement is now ready for use.

11. The cement is now ready for use.

12. The cement is now ready for use.

13. The cement is now ready for use.

14. The cement is now ready for use.

15. The cement is now ready for use.

16. The cement is now ready for use.

17. The cement is now ready for use.

18. The cement is now ready for use.

19. The cement is now ready for use.

20. The cement is now ready for use.

21. The cement is now ready for use.

22. The cement is now ready for use.

- ※ 裝訂原廠之仿單、使用說明書及其中文仿單
- ※ 中文仿單稿包括適應症、禁忌症、型號 (規格製造廠名稱、地址及藥商名稱、地址，如有附仿單標籤等實物過大或印於玻璃金屬容器等)