

而至伊尼雪牙科用金屬瓷粉系列 “GC”Initial MC

衛署醫器輸字第 023254 號

注意：使用前請務必詳閱原廠之使用說明書並遵照指示使用

【產品敘述】

1) 形狀與結構

- 本產品有粉末狀與膏糊狀，與多種完整的色樣系列更接近天然齒的色澤。
- 膏糊狀是先用粉末與專用液調拌而成之預先混合膏。
- 亦有調拌液可用於粉末的調拌。

*本產品含下列瓷粉與調拌液，其主要成分如下：

瓷粉	形狀	主要成分
遮色瓷粉	粉末狀	長石，二氧化矽
遮色修飾瓷粉		
肩台遮色瓷粉		
遮色象牙質瓷粉		
象牙質瓷粉		
漂白象牙質瓷粉		
螢光象牙質瓷粉		
遮色象牙質修飾瓷粉		
螢光透明瓷粉		
玻璃透明瓷粉		
牙釉質瓷粉		
漂白牙釉質瓷粉		
半透色瓷粉		
半透色修正瓷粉		
牙釉質加強瓷粉		
牙釉質咬合瓷粉		
牙釉質歐柏瓷粉		
齒頸部半透明瓷粉		
肩台透明瓷粉		
內插色瓷粉		
牙齦色瓷粉		
一般牙齦色瓷粉		
修補用瓷粉	膏糊狀	長石，醇類
釉粉		
切端色瓷粉		
金屬黏著劑	形狀	主要成分
遮色瓷膏		
遮色修飾瓷膏		
調拌液	液狀	水分
遮色用調拌液		
肩台用調拌液		
堆築用調拌液		
釉粉調拌液		
稀釋液	液狀	醇類
遮色膏液		

2) 操作原則

用調拌液將粉末或是膏糊充分調拌後，經過鑄模、成形、乾燥與金屬燒結成金屬陶瓷復形物。

【產品用途與功能】

本產品用於牙科的金屬陶瓷復形物。

【規格】

【1】瓷粉材料

項目	規格		
一致性	使用蒸餾水可避免色素分離。		
外觀	不得含有雜質		
彎曲強度	> 50 MPa		
溶解度	<100µg/cm ²		
膨脹係數 單位 ($\times 10^{-6} K^{-1}$)	遮色瓷粉、遮色膏	2次燒結	13.5±0.5
		4次燒結	13.5±0.5
	象牙質牙釉瓷粉	2次燒結	13.1±0.5
		4次燒結	13.3±0.5
玻璃轉換溫度 (°C)	遮色瓷粉		600±20
	遮色瓷膏		585±20
	象牙質牙釉瓷粉		575±20
破裂、剝落強度	> 25 MPa		

【2】調拌用液、稀釋液

外觀	不得含有雜質
----	--------

<燒結時間表> (參考用)

燒結	1	2	3	4	5	6	表面
遮色膏首次燒結	550	6.00	80	X	940	1.00	光滑
遮色瓷粉首次燒結	600	2.00	80	X	940	1.00	光滑
遮色膏二次燒結	550	6.00	80	X	930	1.00	略光滑
遮色瓷粉二次燒結	600	2.00	80	X	930	1.00	光滑
首次、二次肩台燒結	550	2.00	80	X	930	1.00	略光滑
首次象牙質燒結	580	6.00	55	X	890	1.00	略光滑
二次象牙質燒結	580	6.00	55	X	880	1.00	略光滑
釉燒結	600	2.00	55	-	890	1.00	光滑
含釉粉的釉燒結	480	2.00	45	-	850	1.00	光滑
修補用瓷粉燒結	450	4.00	45	X	770	1.00	光滑

1. 預熱溫度 (°C)
2. 烘乾時間 (分鐘)
3. 加熱速度 (°C/min)
4. 抽真空
5. 燒結溫度 (°C)
6. 維持時間 (分鐘)

以上為一般燒結的標準情況；須視高溫爐本身的狀況作調整。主要的癥結在於如何達到最理想的燒結效果，燒結情況則是最基本的考量因素。

- 適用之金屬：所有適用於與瓷粉燒結的金屬。

「操作之注意事項」

- 1) 使用膏糊前先用抹刀調拌均勻。如膏糊使用前已儲存相當時間，須加入遮色膏調拌液以增加其稠度。

- 2) 膏糊使用後立即將蓋子蓋緊。

【使用時注意事項】

1) 注意

- ① 為避免瓷粉飛塵傷害身體，在進行切割或是碾磨時應使用局部淨空器或是戴上合格的面具，以避免誤吸。
- ② 切割或是碾磨本產品時應戴上護目鏡，以避免傷及眼睛。
- ③ 欲將產產品置入口內時，須為患者戴上橡皮障。
- ④ 將產產品置入口內時可視情況使用黏著劑，如樹脂黏著劑等。
- ⑤ 勿擅將本產品與其他產品合併使用。
- ⑥ 勿超出上述【產品用途與效能】之使用範圍。
- ⑦ 本產品限由合格的牙科專業人員使用。

2) 重要的基本注意事項

- ① 本系列產品(瓷粉，瓷膏糊，調拌液)與其調拌後的泥狀物，應避免與眼睛接觸。如有誤觸，請即用水大量沖洗後就醫治療。
- ② 本產品於置入患者口內前，應先去除雜質，清理乾淨。

【儲存方式與有效期間】

〔儲存方式〕

- 瓷膏或是調拌液不得儲存於溫度高的地方，或是蓋子未栓緊的狀態。
- 本產品須適當保存，並置於非牙科專業人員隨手可取得之處。

〔有效期間〕

本產品限於有效期間內使用※如外包裝所示。

※ 如外包裝顯示 EXP. 2012-02，代表有效期間至 2012 年 2 月份止。

【包裝】

不同顏色的瓷粉與調拌液均為個別包裝或是部分為套裝組合
〔色調〕

項目	色樣代碼
金屬黏著劑	IN.MB
遮色瓷粉、 遮色瓷膏	O-A1、O-A2、O-A3、O-A3.5、O-A4、 O-B1、O-B2、O-B3、O-B4、O-C1、 O-C2、O-C3、O-C4、O-D2、O-D3、 O-D4
遮色修飾瓷粉、遮色修 是瓷膏	OM-1、OM-2、OM-3、OM-4、OM-5、 OM-6、GO/U
肩台遮色瓷粉	SO-37、SO-38、SO-39
遮色象牙質瓷粉	OD-A1、OD-A2、OD-A3、OD-A3.5、 OD-A4、OD-B1、OD-B2、OD-B3、 OD-B4、OD-C1、OD-C2、OD-C3、 OD-C4、OD-D2、OD-D3、OD-D4
象牙質瓷粉	D-A1、D-A2、D-A3、D-A3.5、D-A4、 D-B1、D-B2、D-B3、D-B4、D-C1、 D-C2、D-C3、D-C4、D-D2、D-D3、 D-D4
漂白象牙質瓷粉	BLD-1、BLD-2、BLD-3
螢光象牙質瓷粉	FD-91、FD-92、FD-93
遮色象牙質修飾瓷粉	ODM-1、ODM-2

螢光透明瓷粉	CL-F
玻璃透明瓷粉	CL-W
牙釉質瓷粉	E-57、E-58、E-59、E-60
漂白牙釉質瓷粉	BL-E
半透色瓷粉	TN、TO
半透色修正瓷粉	TM-01、TM-02、TM-03、TM-04、 TM-05
牙釉質加強瓷粉	EI-11、EI-12、EI-13、EI-14
牙釉質咬合瓷粉	EO-15、EO-16、EO-17
牙釉質歐柏瓷粉	EOP1、EOP2、EOP3、EOP4
齒頸部半透明瓷粉	CT-21、CT-22、CT-23、CT-24、CT-25
肩台透明瓷粉	ST-30、ST-31、ST-32、ST-33、ST-34、 ST-35、ST-36
內插色瓷粉	IN-41、IN-42、IN-43、IN-44、IN-45、 IN-46、IN-47、IN-48、IN-49、IN-50、 IN-51
牙齦色瓷粉	GM-23、GM-24、GM-34、GM-35、 GM-36
一般牙齦色瓷粉	GU
修補用瓷粉	COR
釉粉調拌液	GL
切端色瓷粉	INC

- 瓷粉材料：3g、10g、20g、50 g、250 g
- 瓷膏材料：3g、4g
- 遮色調拌液：25mL、50mL
- 堆築用調拌液：25mL、50mL、250mL、1000 mL
- 肩台用調拌液：25mL、50mL
- 釉粉調拌液：25m
- 遮色瓷膏用調拌液：8mL

**型號與規格
套裝組**

1	877001	GC Initial MC, Basic Set (Paste Opaque): 16 x Paste Opaque 3g 16 x Dentin 20g 4 x Enamel (E57, 58, 59, 60) 20g 3 x INside (IN43, 44, 46) 20g 1 x Clear Fluorescence (CL-F) 20g 2 x Translucent (TN, TO) 20g 1 x Glaze 10g 1 x INCisio 3g 1 x 25 ml Glaze Liquid 2 x 50ml Modelling Liquid 2 x Shade Guide 1 x Flip Chart
2	877002	GC Initial MC, Basic Plus Set (Paste Opaque): 6 x Paste Opaque Modifier (OM1~6) 3g 16 x Opaqus Dentin (ODA1~ODD4) 20g 2 x Opaqus Dentin Modifier (ODM1, 2) 20g 7 x Shoulder Transparent (ST30~36) 20g 3 x Shoulder Opaque (SO37, 38, 39) 20g 5 x Translucent Modifier (TM01~05) 20g 4 x Enamel Intensive (EI11~14) 20g 1 x Correction Powder 20g 1 x 25 ml Glaze Liquid 1 x 50 ml Shoulder Liquid 1 x 50 ml Modelling Liquid 3 x Shade Guide
3	877003	GC Initial MC, Advanced Set: 11 x Inside (IN41~51) 20g 5 x Cervical Translucent (CT21~25) 20g 3 x Enamel Occlusal (EO15, 16, 17) 20g 4 x Enamel Opal (EOP1~4) 20g 3 x Fluo Dentin (FD91, 92, 93) 20g 1 x Gingival Universal 20g 1 x 25 ml Glaze Liquid 2 x 50 ml Modelling Liquid 2 x Shade Guide
4	877004	GC Initial MC, Paste Opaque Set: 16 x Paste Opaque (OA1~ODA4) 3g 6 x Paste Opaque Modifier (OM1~OM6) 3g
5	877016	GC Initial, Stain Set IN vivo / IN situ MC & LF & PC: 13 x INVivo (Fluorescence)3g 3 x INSitu (Non fluorescence)3g 1 x Glaze Powder10g 1 x 25 ml Glaze Liquid 1 x shade tab
6	877035	GC Initial MC, Entrance Kit (Paste opaque): 1 x Paste Opaque OPA3 3g 1 x Opaqus Dentin ODA3 20g 1 x INside IN-44 20g, 1 x Dentin DA3 20g 1 x Clear Fluorescence CL-F 20g 1 x Enamel E58 20g 1 x Enamel E59 20g 1 x Enamel Opal EOP2 20g 1 x MC, LF INVivo/INSitu Glaze Powder 10g 1 x Modelling Liquid 25ml 1 x MC, LF INVivo/INSitu Glaze Liquid 25ml 1 x Flip Chart

7	877045	GC Initial MC, Basic Set (Powder Opaque) : 16 x Powder Opaque 20g 16 x Dentin 20g 4 x Enamel (E57, 58, 59, 60) 20g 3 x INside (IN43, 44, 46) 20g 1 x Clear Fluorescence (CL-F) 20g 2 x Translucent (TN, TO) 20g 1 x Glaze 10g 1 x INCisio 3g 1 x 25 ml Glaze Liquid 1 x 50 ml Opaque Liquid 1 x 50ml Modelling Liquid 2 x Shade Guide 1 x Flip Chart
8	877046	GC Initial MC, Basic Plus Set (Powder Opaque): 6 x Powder Opaque Modifier (OM1~6) 20g 16 x Opaqus Dentin (ODA1~ODD4) 20g 2 x Opaqus Dentin Modifier (ODM1, 2) 20g 7 x Shoulder Transparent (ST30~36)20g 3 x Shoulder Opaque (SO37, 38, 39) 20g 5 x Translucent Modifier (TM01~05) 20g 4 x Enamel Intensive (EI11~14) 20g 1 x Correction Powder 20g 1 x 25 ml Opaque Liquid 1 x 50 ml Shoulder Liquid 1 x 50 ml Modelling Liquid 3 x Shade Guide
9	877048	GC Initial MC, Entrance Kit (Powder Opaque) : 1 x Powder Opaque OPA3 20g 1 x Opaqus Dentin ODA3 20g 1 x INside IN-44 20g 1 x Dentin DA3 20g 1 x Clear Fluorescence CL-F 20g 1 x Enamel E58 20g 1 x Enamel E59 20g 1 x Enamel Opal EOP2 20g 1 x MC, LF INVivo/INSitu Glaze Powder 10g 1 x Modelling Liquid 25 ml 1 x MC, LF INVivo/INSitu Glaze Liquid 25ml 1 x Opaque Liquid 25 ml 1 x Flip Chart
10	877065	GC Initial MC, Bleach Set: 1 x Powder Opaque Modifier OM-1 (white) 1 x Bleach Dentin BLD-1 (light) 1 x Bleach Dentin BLD-2 (white) 1 x Bleach Dentin BLD-3 (Xwhite) 1 x Bleach Enamel BL-E 1 x Enamel Opal EOP-1 (bleached white) 1 x Fluo Dentin FD-91 (light) 2 x Modelling Liquid 25ml 1 x Opaque Liquid 25ml 7 x Shade Guide Tabs
11	877070	GC Initial MC, Gum Shade Set (Powder Opaque): 1 x Opaque Modifier GO-U (gum opaque universal) 20g, 1 x GM-23 (base light) 20g 1 x GM-24 (base dark) 20g 1 x GM-34 (intensive violet) 20g 1 x GM-35 (intensive cream) 20g 1 x GM-36 (intensive red) 20g 1 x Modelling Liquid 25ml 1 x Opaque Liquid 25ml 1 x 6 shade guide tabs
12	877071	GC Initial MC, Gum Shade Set (Paste Opaque) 1 x Opaque Modifier GO-U (gum opaque universal) 20g 1 x GM-23 (base light) 20g 1 x GM-24 (base dark) 20g 1 x GM-34 (intensive violet) 20g 1 x GM-35 (intensive cream) 20g 1 x GM-36 (intensive red) 20g 1 x Modelling Liquid 25ml 1 x Opaque Liquid 25ml 6 x Shade Guide Tabs

3g 補充包

13	870001	GC Initial MC, Paste Opaque OA1
14	870002	GC Initial MC, Paste Opaque OA2
15	870003	GC Initial MC, Paste Opaque OA3
16	870004	GC Initial MC, Paste Opaque OA3.5
17	870005	GC Initial MC, Paste Opaque OA4
18	870006	GC Initial MC, Paste Opaque OB1
19	870007	GC Initial MC, Paste Opaque OB2
20	870008	GC Initial MC, Paste Opaque OB3
21	870009	GC Initial MC, Paste Opaque OB4
22	870010	GC Initial MC, Paste Opaque OC1
23	870011	GC Initial MC, Paste Opaque OC2
24	870012	GC Initial MC, Paste Opaque OC3
25	870013	GC Initial MC, Paste Opaque OC4
26	870014	GC Initial MC, Paste Opaque OD2
27	870015	GC Initial MC, Paste Opaque OD3
28	870016	GC Initial MC, Paste Opaque OD4
29	870017	GC Initial MC, Paste Opaque Modifier OM-1
30	870018	GC Initial MC, Paste Opaque Modifier OM-2
31	870019	GC Initial MC, Paste Opaque Modifier OM-3
32	870020	GC Initial MC, Paste Opaque Modifier OM-4
33	870021	GC Initial MC, Paste Opaque Modifier OM-5
34	870022	GC Initial MC, Paste Opaque Modifier OM-6
35	870023	GC Initial MC, Paste Opaque Modifier GO-U
36	876001	GC Initial IN vivo, IV-1 MC& LF& PC stain
37	876002	GC Initial IN vivo, IV-2 MC& LF& PC stain
38	876003	GC Initial IN vivo, IV-3 MC& LF& PC stain
39	876004	GC Initial IN vivo, IV-4 MC& LF& PC stain
40	876005	GC Initial IN vivo, IV-5 MC& LF& PC stain
41	876006	GC Initial IN vivo, IV-6 MC& LF& PC stain
42	876007	GC Initial IN vivo, IV-7 MC& LF& PC stain
43	876008	GC Initial IN vivo, IV-8 MC& LF& PC stain
44	876009	GC Initial IN vivo, IV-9 MC& LF& PC stain
45	876010	GC Initial IN vivo, IV-10 MC& LF& PC stain
46	876011	GC Initial IN vivo, IV-11 MC& LF& PC stain
47	876012	GC Initial IN vivo, IV-12 MC& LF& PC stain
48	876013	GC Initial IN vivo, IV-13 MC& LF& PC stain
49	876020	GC Initial IN situ, IS-B MC& LF stain
50	876021	GC Initial IN situ, IS-R MC& LF stain
51	876022	GC Initial IN situ, IS-C MC& LF stain
52	876030	GC Initial Incisio, IN-C MC& LF stain

20g 補充包

53	870031	GC Initial MC, Opaqus Dentin ODA1
54	870032	GC Initial MC, Opaqus Dentin ODA2
55	870033	GC Initial MC, Opaqus Dentin ODA3
56	870034	GC Initial MC, Opaqus Dentin ODA3.5
57	870035	GC Initial MC, Opaqus Dentin ODA4
58	870036	GC Initial MC, Opaqus Dentin ODB1
59	870037	GC Initial MC, Opaqus Dentin ODB2
60	870038	GC Initial MC, Opaqus Dentin ODB3
61	870039	GC Initial MC, Opaqus Dentin ODB4
62	870040	GC Initial MC, Opaqus Dentin ODC1
63	870041	GC Initial MC, Opaqus Dentin ODC2
64	870042	GC Initial MC, Opaqus Dentin ODC3
65	870043	GC Initial MC, Opaqus Dentin ODC4
66	870044	GC Initial MC, Opaqus Dentin ODD2

67	870045	GC Initial MC, Opaqus Dentin ODD3
68	870046	GC Initial MC, Opaqus Dentin ODD4
69	870047	GC Initial MC, Opaqus Dentin Modifier ODM-1
70	870048	GC Initial MC, Opaqus Dentin Modifier ODM-2
71	870051	GC Initial MC, Dentin DA1
72	870052	GC Initial MC, Dentin DA2
73	870053	GC Initial MC, Dentin DA3
74	870054	GC Initial MC, Dentin DA3.5
75	870055	GC Initial MC, Dentin DA4
76	870056	GC Initial MC, Dentin DB1
77	870057	GC Initial MC, Dentin DB2
78	870058	GC Initial MC, Dentin DB3
79	870059	GC Initial MC, Dentin DB4
80	870060	GC Initial MC, Dentin DC1
81	870061	GC Initial MC, Dentin DC2
82	870062	GC Initial MC, Dentin DC3
83	870063	GC Initial MC, Dentin DC4
84	870064	GC Initial MC, Dentin DD2
85	870065	GC Initial MC, Dentin DD3
86	870066	GC Initial MC, Dentin DD4
87	870067	GC Initial MC, Enamel E-57
88	870068	GC Initial MC, Enamel E-58
89	870069	GC Initial MC, Enamel E-59
90	870070	GC Initial MC, Enamel E-60
91	870071	GC Initial MC, Clear Fluorescence CL-F
92	870072	GC Initial MC, Translucent TN
93	870073	GC Initial MC, Translucent TO
94	870074	GC Initial MC, Translucent Modifier TM-01
95	870075	GC Initial MC, Translucent Modifier TM-02
96	870076	GC Initial MC, Translucent Modifier TM-03
97	870077	GC Initial MC, Translucent Modifier TM-04
98	870078	GC Initial MC, Translucent Modifier TM-05
99	870080	GC Initial MC, Clear Window CL-W
100	870081	GC Initial MC, Enamel Intensive EI-11
101	870082	GC Initial MC, Enamel Intensive EI-12
102	870083	GC Initial MC, Enamel Intensive EI-13
103	870084	GC Initial MC, Enamel Intensive EI-14
104	870085	GC Initial MC, Enamel Occlusal EO-15
105	870086	GC Initial MC, Enamel Occlusal EO-16
106	870087	GC Initial MC, Enamel Occlusal EO-17
107	870091	GC Initial MC, Fluo-Dentin FD-91
108	870092	GC Initial MC, Fluo-Dentin FD-92
109	870093	GC Initial MC, Fluo-Dentin FD-93
110	870101	GC Initial MC, Enamel Opal EOP1
111	870102	GC Initial MC, Enamel Opal EOP2
112	870103	GC Initial MC, Enamel Opal EOP3
113	870104	GC Initial MC, Enamel Opal EOP4
114	870121	GC Initial MC, Cervical Translucent CT-21
115	870122	GC Initial MC, Cervical Translucent CT-22
116	870123	GC Initial MC, Cervical Translucent CT-23
117	870124	GC Initial MC, Cervical Translucent CT-24
118	870125	GC Initial MC, Cervical Translucent CT-25
119	870130	GC Initial MC, Shoulder Transpa ST-30
120	870131	GC Initial MC, Shoulder Transpa ST-31
121	870132	GC Initial MC, Shoulder Transpa ST-32
122	870133	GC Initial MC, Shoulder Transpa ST-33
123	870134	GC Initial MC, Shoulder Transpa ST-34
124	870135	GC Initial MC, Shoulder Transpa ST-35
125	870136	GC Initial MC, Shoulder Transpa ST-36

126	870137	GC Initial MC, Shoulder Opaque SO-37
127	870138	GC Initial MC, Shoulder Opaque SO-38
128	870139	GC Initial MC, Shoulder Opaque SO-39
129	870141	GC Initial MC, Inside IN-41 Flamingo
130	870142	GC Initial MC, Inside IN-42 Terracota
131	870143	GC Initial MC, Inside IN-43 Sun
132	870144	GC Initial MC, Inside IN-44 Sand
133	870145	GC Initial MC, Inside IN-45 Havanna
134	870146	GC Initial MC, Inside IN-46 Brasil
135	870147	GC Initial MC, Inside IN-47 Sienna
136	870148	GC Initial MC, Inside IN-48 Kurkuma
137	870149	GC Initial MC, Inside IN-49 Maracuja
138	870150	GC Initial MC, Inside IN-50 Curry
139	870151	GC Initial MC, Inside IN-51 Olive
140	870155	GC Initial MC, GM-23 (base light)
141	870156	GC Initial MC, GM-24 (base dark)
142	870157	GC Initial MC, GM-34 (intensive violet)
143	870158	GC Initial MC, GM-35 (intensive cream)
144	870159	GC Initial MC, GM-36 (intensive red)
145	870160	GC Initial MC, Gingival Universal GU
146	870165	GC Initial MC, Correction Powder COR
147	870171	GC Initial MC, Powder Opaque O-A1
148	870172	GC Initial MC, Powder Opaque O-A2
149	870173	GC Initial MC, Powder Opaque O-A3
150	870174	GC Initial MC, Powder Opaque O-A3,5
151	870175	GC Initial MC, Powder Opaque O-A4
152	870176	GC Initial MC, Powder Opaque O-B1
153	870177	GC Initial MC, Powder Opaque O-B2
154	870178	GC Initial MC, Powder Opaque O-B3
155	870179	GC Initial MC, Powder Opaque O-B4
156	870180	GC Initial MC, Powder Opaque O-C1
157	870181	GC Initial MC, Powder Opaque O-C2
158	870182	GC Initial MC, Powder Opaque O-C3
159	870183	GC Initial MC, Powder Opaque O-C4
160	870184	GC Initial MC, Powder Opaque O-D2
161	870185	GC Initial MC, Powder Opaque O-D3
162	870186	GC Initial MC, Powder Opaque O-D4
163	870187	GC Initial MC, Powder Opaque Modifier OM-1 (White)
164	870188	GC Initial MC, Powder Opaque Modifier OM-2 (olive/kaki)
165	870189	GC Initial MC, Powder Opaque Modifier OM-3(ocker/orange)
166	870190	GC Initial MC, Powder Opaque Modifier OM-4 (yellow gold)
167	870191	GC Initial MC, Powder Opaque Modifier OM-5 (red brown)
168	870192	GC Initial MC, Powder Opaque Modifier OM-6 (gingival)
169	870193	GC Initial MC, Powder Opaque Modifier GO-U (gum opaque universal)
170	870202	GC Initial MC, Bleach Dentin BLD-1(light)
171	870203	GC Initial MC, Bleach Dentin BLD-2(white)
172	870204	GC Initial MC, Bleach Dentin BLD-3 (Xwhite)
173	870205	GC Initial MC, Bleach Enamel, BL-E

50g 補充包

174	870252	GC Initial MC, Bleach Dentin – light, BLD-1
175	870253	GC Initial MC, Bleach Dentin – white, BLD-2
176	870254	GC Initial MC, Bleach Dentin – Xwhite, BLD-3
177	870255	GC Initial MC, Bleach Enamel, BL-E 50 g
178	870501	GC Initial MC, Powder Opaque O-A1

179	870502	GC Initial MC, Powder Opaque O-A2
180	870503	GC Initial MC, Powder Opaque O-A3
181	870504	GC Initial MC, Powder Opaque O-A3,5
182	870505	GC Initial MC, Powder Opaque O-A4
183	870506	GC Initial MC, Powder Opaque O-B1
184	870507	GC Initial MC, Powder Opaque O-B2
185	870508	GC Initial MC, Powder Opaque O-B3
186	870509	GC Initial MC, Powder Opaque O-B4
187	870510	GC Initial MC, Powder Opaque O-C1
188	870511	GC Initial MC, Powder Opaque O-C2
189	870512	GC Initial MC, Powder Opaque O-C3
190	870513	GC Initial MC, Powder Opaque O-C4
191	870514	GC Initial MC, Powder Opaque O-D2
192	870515	GC Initial MC, Powder Opaque O-D3
193	870516	GC Initial MC, Powder Opaque O-D4
194	870517	GC Initial MC, Powder Opaque Modifier OM-1 (white)
195	870518	GC Initial MC, Powder Opaque Modifier OM-2 (olive/kaki)
196	870519	GC Initial MC, Powder Opaque Modifier OM-3(ocker/orange)
197	870520	GC Initial MC, Powder Opaque Modifier OM-4 (yellow gold)
198	870521	GC Initial MC, Powder Opaque Modifier OM-5 (red brown)
199	870522	GC Initial MC, Powder Opaque Modifier OM-6 (gingival)
200	870531	GC Initial MC, Opaqus Dentin ODA1
201	870532	GC Initial MC, Opaqus Dentin ODA2
202	870533	GC Initial MC, Opaqus Dentin ODA3
203	870534	GC Initial MC, Opaqus Dentin ODA3.5
204	870535	GC Initial MC, Opaqus Dentin ODA4
205	870536	GC Initial MC, Opaqus Dentin ODB1
206	870537	GC Initial MC, Opaqus Dentin ODB2
207	870538	GC Initial MC, Opaqus Dentin ODB3
208	870539	GC Initial MC, Opaqus Dentin ODB4
209	870540	GC Initial MC, Opaqus Dentin ODC1
210	870541	GC Initial MC, Opaqus Dentin ODC2
211	870542	GC Initial MC, Opaqus Dentin ODC3
212	870543	GC Initial MC, Opaqus Dentin ODC4
213	870544	GC Initial MC, Opaqus Dentin ODD2
214	870545	GC Initial MC, Opaqus Dentin ODD3
215	870546	GC Initial MC, Opaqus Dentin ODD4
216	870547	GC Initial MC, Opaqus Dentin Modifier ODM-1
217	870548	GC Initial MC, Opaqus Dentin Modifier ODM-2
218	870551	GC Initial MC, Dentin DA1
219	870552	GC Initial MC, Dentin DA2
220	870553	GC Initial MC, Dentin DA3
221	870554	GC Initial MC, Dentin DA3.5
222	870555	GC Initial MC, Dentin DA4
223	870556	GC Initial MC, Dentin DB1
224	870557	GC Initial MC, Dentin DB2
225	870558	GC Initial MC, Dentin DB3
226	870559	GC Initial MC, Dentin DB4
227	870560	GC Initial MC, Dentin DC1
228	870561	GC Initial MC, Dentin DC2
229	870562	GC Initial MC, Dentin DC3
230	870563	GC Initial MC, Dentin DC4
231	870564	GC Initial MC, Dentin DD2
232	870565	GC Initial MC, Dentin DD3
233	870566	GC Initial MC, Dentin DD4
234	870567	GC Initial MC, Enamel E-57
235	870568	GC Initial MC, Enamel E-58
236	870569	GC Initial MC, Enamel E-59

237	870570	GC Initial MC, Enamel E-60
238	870571	GC Initial MC, Clear Fluorescence CL-F
239	870572	GC Initial MC, Translucent TN
240	870573	GC Initial MC, Translucent TO
241	870574	GC Initial MC, Translucent Modifier TM-01
242	870575	GC Initial MC, Translucent Modifier TM-02
243	870576	GC Initial MC, Translucent Modifier TM-03
244	870577	GC Initial MC, Translucent Modifier TM-04
245	870578	GC Initial MC, Translucent Modifier TM-05
246	870580	GC Initial MC, Clear Window CL-W
247	870581	GC Initial MC, Enamel Intensive EI-11
248	870582	GC Initial MC, Enamel Intensive EI-12
249	870583	GC Initial MC, Enamel Intensive EI-13
250	870584	GC Initial MC, Enamel Intensive EI-14
251	870585	GC Initial MC, Enamel Occlusal EO-15
252	870586	GC Initial MC, Enamel Occlusal EO-16
253	870587	GC Initial MC, Enamel Occlusal EO-17
254	870591	GC Initial MC, Fluo-Dentin FD-91
255	870592	GC Initial MC, Fluo-Dentin FD-92
256	870593	GC Initial MC, Fluo-Dentin FD-93
257	870601	GC Initial MC, Enamel Opal EOP1
258	870602	GC Initial MC, Enamel Opal EOP2
259	870603	GC Initial MC, Enamel Opal EOP3
260	870604	GC Initial MC, Enamel Opal EOP4
261	870621	GC Initial MC, Cervical Translucent CT-21
262	870622	GC Initial MC, Cervical Translucent CT-22
263	870623	GC Initial MC, Cervical Translucent CT-23
264	870624	GC Initial MC, Cervical Translucent CT-24
265	870625	GC Initial MC, Cervical Translucent CT-25
266	870630	GC Initial MC, Shoulder ST-30
267	870631	GC Initial MC, Shoulder ST-31
268	870632	GC Initial MC, Shoulder ST-32
269	870633	GC Initial MC, Shoulder ST-33
270	870634	GC Initial MC, Shoulder ST-34
271	870635	GC Initial MC, Shoulder ST-35
272	870636	GC Initial MC, Shoulder ST-36
273	870637	GC Initial MC, Shoulder Opaque SO-37
274	870638	GC Initial MC, Shoulder Opaque SO-38
275	870639	GC Initial MC, Shoulder Opaque SO-39
276	870641	GC Initial MC, Inside IN-41 Flamingo
277	870642	GC Initial MC, Inside IN-42 Terracota
278	870643	GC Initial MC, Inside IN-43 Sun
279	870644	GC Initial MC, Inside IN-44 Sand
280	870645	GC Initial MC, Inside IN-45 Havanna
281	870646	GC Initial MC, Inside IN-46 Brasil
282	870647	GC Initial MC, Inside IN-47 Sienna
283	870648	GC Initial MC, Inside IN-48 Kurkuma
284	870649	GC Initial MC, Inside IN-49 Maracuja
285	870650	GC Initial MC, Inside IN-50 Curry
286	870651	GC Initial MC, Inside IN-51 Olive
287	870660	GC Initial MC, Gingival Universal GU

250g 補充包

288	870801	GC Initial MC, Dentin D-A1
289	870802	GC Initial MC, Dentin DA2
290	870803	GC Initial MC, Dentin DA3
291	870804	GC Initial MC, Dentin DA3.5
292	870805	GC Initial MC, Dentin DA4
293	870806	GC Initial MC, Dentin D-B1
294	870807	GC Initial MC, Dentin DB2

295	870808	GC Initial MC, Dentin DB3
296	870809	GC Initial MC, Dentin DB4
297	870810	GC Initial MC, Dentin D-C1
298	870811	GC Initial MC, Dentin D-C2
299	870812	GC Initial MC, Dentin D-C3
300	870813	GC Initial MC, Dentin D-C4
301	870814	GC Initial MC, Dentin D-D2
302	870815	GC Initial MC, Dentin D-D3
303	870816	GC Initial MC, Dentin D-D4
304	870817	GC Initial MC, Enamel E-57
305	870818	GC Initial MC, Enamel E-58
306	870819	GC Initial MC, Enamel E-59
307	870820	GC Initial MC, Enamel E-60
308	870821	GC Initial MC, Clear Fluorescence CL-F
309	870822	GC Initial MC, Translucent TN
310	870823	GC Initial MC, Translucent TO

調拌用液、釉粉、結合膏等

311	870700	GC Initial MC, Paste Opaque Thinner, 8ml
312	870701	GC Initial MC, Opaque Liquid, 25ml
313	870702	GC Initial MC, Modelling Liquid, 25ml
314	870703	GC Initial MC, Shoulder Liquid, 25ml
315	870711	GC Initial MC, Opaque Liquid, 50ml
316	870712	GC Initial MC, Modelling Liquid, 50ml
317	870713	GC Initial MC, Shoulder Liquid, 50ml
318	870722	GC Initial MC, Modelling Liquid, 250ml
319	870732	GC Initial MC, Modelling Liquid, 1000ml
320	876000	GC Initial MC, LF – Invivo/Insitu Glaze Powder GL, 10g
321	876035	GC Initial MC, LF – Invivo/Insitu, Glaze Liquid, 25 ml
322	877030	GC Initial, MC, LF Connector Paste, 6g
323	877050	GC Initial, INMentalBond , 2 x 4g bonder

比色板組

324	877020	GC Initial, MC, LF, Ti Shade Guide Basic Set (2 CD)
325	877021	GC Initial, MC, LF, Ti Shade Guide Basic Plus Set (3 CD)
326	877022	GC Initial, MC, LF, Ti Shade Guide Advanced Set (2 CD)
327	877026	GC Initial, INvivo / INsitu Shade Guide
328	877027	GC Initial, MC Shade Guide Standard Set

製造廠名稱及地址：

Manufactured by KLEMA Dentalprodukte GmbH
(地址:Koblacherstrasse 3a, A-6812 Meiningen, Austria)

for GC Europe N.V.

(地址: Research Park, Interleuvenlaan 33, B-3001 Leuven, Belgium)

藥商名稱：台灣而至股份有限公司

藥商地址：台北市松山區民生東路三段156號14樓