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“而至”軟敷龍襯底樹脂 “GC” SOFT-LINER Soft Denture Reline Material

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Prior to use, carefully read
the instructions for use.

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SOFT-LINER

SOFT DENTURE RELINE MATERIAL

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

For the temporary rebasing (relining) of acrylic dentures.
Tissue conditioning.

CONTRAINDICATIONS

In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

DIRECTIONS FOR USE

1. PREPARATION

Clean the contact surface thoroughly using pumice and water and abrade the area to be relined.

2. MIXING

a) The standard powder/liquid ratio is 2.2 g/1.8 g, (first graduation on powder measure to 4 graduations of the liquid syringe).

Note:

Consistency may be varied by changing powder/ liquid ratio.

b) Measure into the glass jar and mix for 30-60 seconds.

3. RELINING

a) Coat the mixture onto the denture as evenly as possible.

b) Insert the denture in the mouth, muscle trim and retain it in the centric occlusal position for 4-5 minutes.

c) Remove and inspect for defects.

d) Dip the denture into cold water, and trim away excess at the periphery with a heated sharp instrument.

STORAGE

Store in a cool dark place away from direct sunlight.
(Shelf life : 3 years from date of manufacture)

PACKAGES

1-1 package : 100 g powder, 100 g (97 mL) liquid, plus accessories.

NOTE

Close bottles immediately after use.

WARNING

In case of contact with eyes, flush immediately with water and seek medical attention.

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GC Dental Products Corporation Kachigawa Plant
GC KOREA CO., LTD.
GC DENTAL (SUZHOU) CO., LTD.