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94. 12. 19

FREEGENOL TEMPORARY PACK

Prior to use, carefully read the instructions for use.

EUGENOL FREE TEMPORARY CEMENT

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

Cementation of temporary crowns and bridges.

CONTRAINDICATIONS

In rare cases the product may cause sensitivity to some persons. If such reactions are experienced, discontinue the use of the product and refer to a physician.

DIRECTIONS FOR USE

1. Squeeze equal lengths of Base and Accelerator pastes onto the mixing pad.
2. Mix thoroughly for 20-30 seconds.
3. Insert paste into (temporary) crown or bridge and seat onto the prepared dry tooth.
4. Allow to set (3 minutes from start of mixing) and trim excess.

NOTES

1. Setting time can be adjusted by using more or less Base (More Base-faster set ; less Base-slower set). Do not go outside Base / Accelerator ratio range 3 : 2 to 2 : 3).
2. If material sticks to tooth or metal, Vaseline may be added up to 25% of total mix.

STORAGE

Store at temperature of 4-25°C (39.2-77.0°F).
(Shelf life : 3 years)

PACKAGES

1-1 pkg : 55g Base, 20g Accelerator, Cleaner and mixing pad.

CAUTION

In case of contact with eyes, flush immediately with water and seek medical attention.

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“GC”FREEGENOL TEMPORARY PACK
Temporay Cement

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