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“而至”優尼特自凝壓克力樹脂
“GC” UNIFAST TRAD Acrylic Resin

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製造廠名稱：GC Dental Products Corporation Kachigawa Plant

Prior to use, carefully read the instructions for use.

GB

UNIFAST Trad
GENERAL PURPOSE ACRYLIC RESIN

GC UNIFAST Trad is a self curing, fast setting acrylic resin for use in a variety of dental applications. For use in either pressure or paint-on techniques.

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

1. Repair of dentures.
2. Construction of temporary crowns.
3. Impression trays for individual teeth.
4. Orthodontic plates, splints, check bite and other general purposes.

CONTRAINDICATIONS

Patients with known allergy to acrylic resins. In case of allergy, refer to a physician.


DIRECTIONS FOR USE

A. PRESSURE TECHNIQUE



The recommended powder / liquid ratio is 1 g of powder (1st graduation of powder measure) to 0.5ml (1st graduation of liquid measure) of liquid.

1. Dispense the required amount of liquid into the rubber cup as follows:

- a) Remove the protective cap and screw on the dispensing nozzle (fig. 1)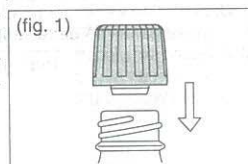

- b) Pour into the measure provided (fig. 2) until the bottom of the meniscus is level with the required graduation (fig. 3).

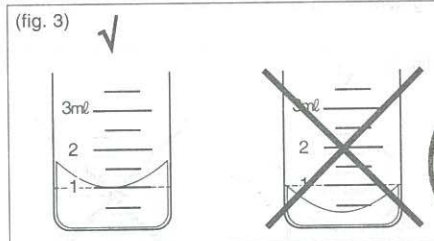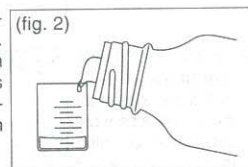- c) Replace the protective cap to prevent evaporation.
2. Add the powder and mix thoroughly for 10-15 seconds.
3. The mixture reaches a dough state 20-30 seconds after mixing. All manipulation should be finished before 2 minutes after mixing when setting starts.

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